

Wrap-Around Bath Support Instructions

PVC frame with padded and non-padded configurations


Model Numbers:

7200, 7300, 7400 (Lo-Back without Padding)


7210, 7310, 7410 (Lo-Back with Padding)

7600, 7700, 7900 (Hi-Back without Padding)

7610, 7710, 7910 (Hi-Back with Padding)

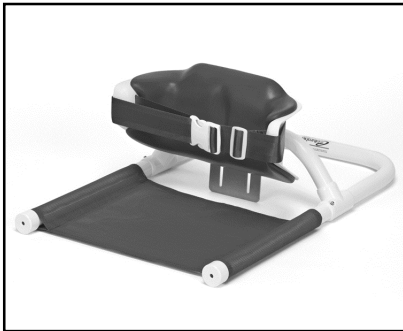

Lo-Back with Padding

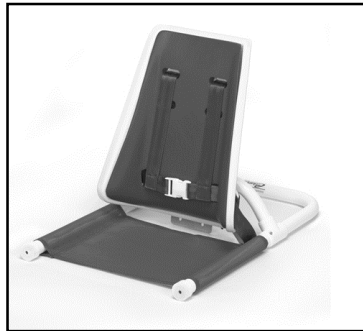

Hi-Back with Padding



Lo-Back without Padding



Hi-Back without Padding

ATTENTION

Your Columbia Wrap-Around Bath Support has been built to the highest standards of quality to ensure years of superior service. Please take the time to read these instructions and pay particular attention to the safety information provided.

Some additional precautions you can take to reduce the risk of injury:

1. Read and understand all instructions.
2. Determine, with a physician or therapist, the suitability of this product for the intended user.
3. Never leave a user in the Wrap-Around Bath Support unattended in the bath. The Wrap-Around Bath Support provides support but does not replace adult supervision while bathing.
4. When this support is used in or around water, uninterrupted supervision is necessary to reduce the risk of injury or drowning.
5. Regularly check that all locking knobs, buckles and other parts are tight at all times.
6. Use the Wrap-Around Bath Support on level surfaces only.

If you require additional assistance, please call Columbia Customer Service at **(800) 454-6612**, or e-mail **info@columbiamedical.com**

Thank you for choosing Columbia.

Replacement Parts

Replacement parts can be ordered by calling our customer service department at 800-454-6612, Monday – Friday, between 7:00 am and 5:00 pm PST/PDT. You can also order online at:
www.columbiamedical.com.

Other Uses

- The Wrap-Around Bath Support can be used indoors on the floor, outdoors on the lawn, in a wading pool, or wherever a stable support is desired.
- The Wrap-Around Bath Support disassembles easily for storage or for traveling.

Cleaning Your Bath Support

The Bath Support may be cleaned with mild cleansers or soap. Harsh abrasive cleaners may dull the finish. The metal parts of the bath support are made of stainless steel and will not rust.


Assembling the Wrap-Around Bath Support

Step 1: Attach the Legs

The front horizontal legs are shipped detached from the frame. To assemble, depress the metal button on the legs and insert the legs into the frame. The metal button will click into the holes of the frame, locking the legs in place. *See diagram below.*

Step 2: Attach the Back Support

Place the metal plate which is attached to the back support against the front side of the curved arch of the frame. Bolt the metal plate to the arch by inserting the two bolts provided through the frame, from front to back. Place the plastic washers on each bolt on the back side of the frame, then screw on the plastic locking knobs. *See diagram below.*

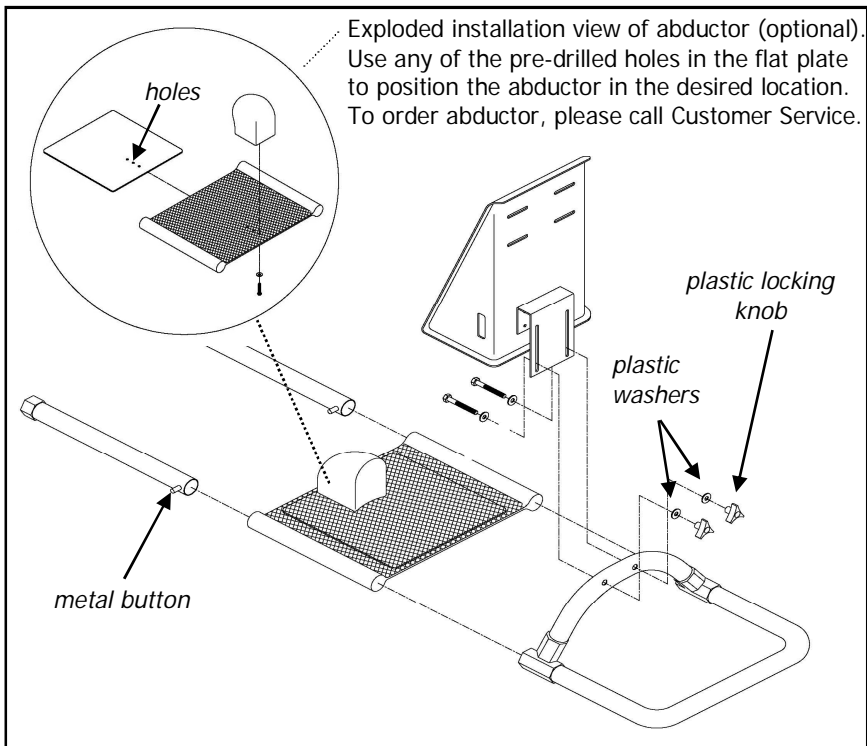

Adjusting the Wrap-Around Bath Support

- Adjust the back height by loosening the plastic locking knobs and sliding the back up or down on the metal plate to the desired height.
- Adjust the lap belt to provide a snug, but comfortable fit.
 - If using a Hi-Back model, adjust the shoulder straps.

CE

BS Partnership
41 Dan-y-Coed Road
Cardiff
CF23 6N
UK



Solutions for Daily Living

11724 Willake Street
Santa Fe Springs, CA 90670
(800) 454-6612

Fax: (310) 305-1718

Email: info@columbiamedical.com

Website: www.columbiamedical.com

© 2012 Columbia Medical Mfg., LLC

COLUMBIA MEDICAL MFG. LLC warrants this product to be free of defects in material and workmanship. Our obligation under this warranty is limited to repair or replacement of any part or entire unit at our option for a period of one year from date of delivery to the original purchaser. The warranty does not include cost of inconvenience, property damage, misuse, abuse, accident or similar incidents. The warranty will not apply if product has been tampered with or repaired by unauthorized individuals.