

USER MANUAL

Bariatric Heavy Duty 8" Wheel Rollator



AML25073

Parts of the product



Specification of the product

Product dimension	Product dimension folded	Handle height adjustment	Weight Capacity	Seat Height	Seat Dimensions	Backrest Height	Wheel diameter
680*738*1020mm	755*225*900mm	823-1020mm	500 LBS	550mm	550*380mm	828mm	8 inch

Assembly Instructions

Assemble handles and height adjustment

The rollator is delivered largely pre-assembled remove the lever from the frame, insert the handles into the tube on the frame position of the handles at the desired height.

The handles are correctly adjusted when you are at the same height as your wrists, fix the height by inserting the lever and tighten it, and make sure that both handles are at the same height.



Assemble the shopping basket

Folding the seat forward, hang the hooks of the shopping basket in the frame, and Do not transport too heavy loads (Maximum weight for the basket is 5kg).

Assemble the backrest

The backrest should only be installed when the rollator is in a safe position and the brakes are activated for the backrest, proceed in reverse order.





Parking brakes

To apply the parking brake, press the brake lever down. To release it, apply the brake as usual. Caution! Never apply the parking brake while in motion.



To reduce walking speed, use the lever on the handle

To do this, pull the handle upwards towards you with your fingers.

Folding rollator

To transport or store your rollator to save space, you can fold it up and fold the seat up. Take out the shopping basket and pull up the cross braces, the rollator will fold up.



Unfold rollator

Position your rollator in front of you and apply the brakes.

The rollator unfolds by gently pressing the handles
Press the cross brace down and fold the seat surface down.