


# Push-Button Buckle Covers

ART NOS. FS018C, FS019C, FS018D, FS019D,  
FS018E & FS019E

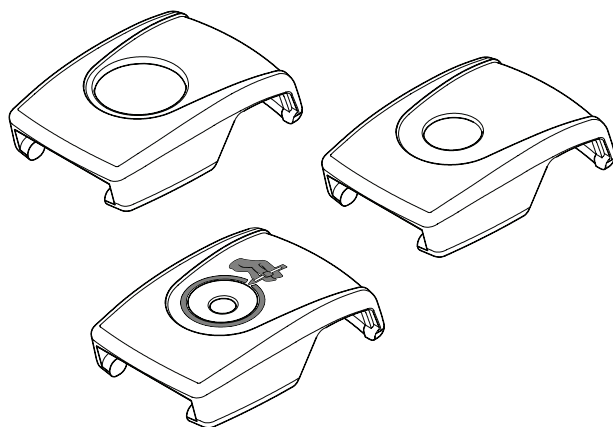

**⚠ WARNING!** Accidental release of this pelvic support belt can allow the user to slip down or fall from the wheelchair. If the user's movements or cognitive abilities could lead to accidental release, a caregiver must be present at all times during its use. Ensure that all caregivers know how to unfasten the product. Failure to do so may delay release in an emergency.

**WARRANTY:** This product carries a limited lifetime warranty against defects in workmanship and materials arising under normal use by the original consumer. Contact your supplier or Bodypoint for warranty claims.

For more information on Bodypoint products, and a list of distributors outside the USA, go to [www.bodypoint.com](http://www.bodypoint.com)

## FS018C-1 & FS019C-1 SECURITY BUCKLE COVERS


**⚠ WARNING!** This product is intended to prevent unintentional release of a belt by the user. Improper use of this product could result in a user being trapped in the wheelchair in an emergency. A caregiver with a release tool should be present at all times during use of this product. Avoid the use of this product as a restraint unless a physician has weighed the risks and determined that it will increase the user's safety.

**⚠ WARNING!** Avoid using tools that may break off inside buckle. Periodically remove buckle cover to check for any obstructions.




Bodypoint Europe BV  
Kerkstraat 29  
7396 PD Terwolde  
The Netherlands



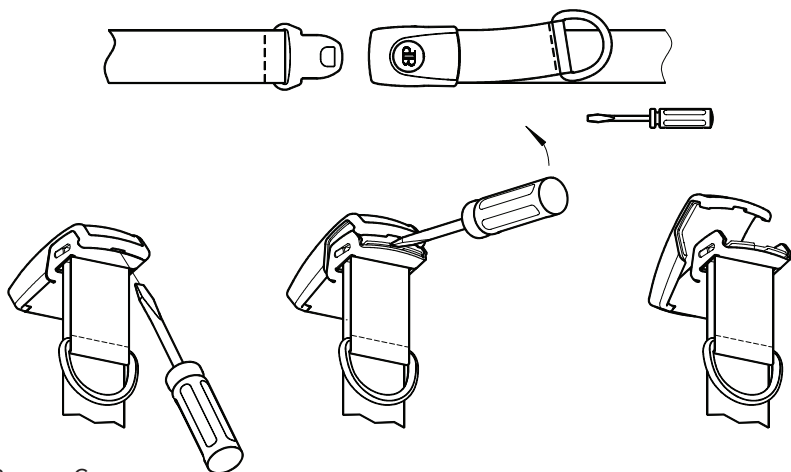

Remove Cover.

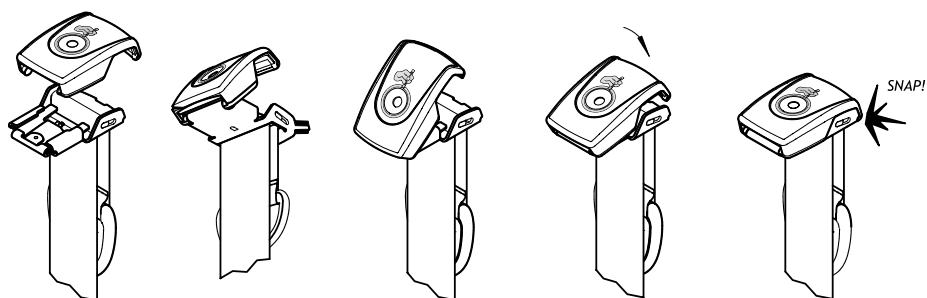

Assemble new cover.

