



**ALLIED
MEDICAL**
LIMITED

USER MANUAL


Buckingham Walking Frame Caddy

NRSL29877



INTENDED USES

The Buckingham Walking Frame Caddy consists of a tray and container, which allows you to safely carry personal items and meals easily and with confidence. Simple to attach to virtually all non-folding wheeled walking frames it is light but sturdy, easy to clean and will not upset the balance of the frame.

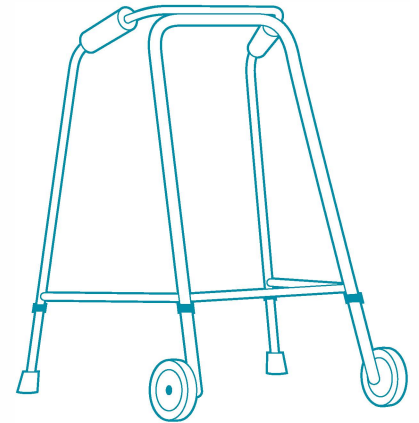


FITTING THE CADDY TO A WALKING FRAME

This product has been designed for use with non-folding, wheeled walking frames.



FOR FRAMES WITH A DOUBLE BAR ACROSS FRONT

- Position the Caddy on the frame with the groove over the bars
- Push fit the Caddy onto the frame until it clicks into position

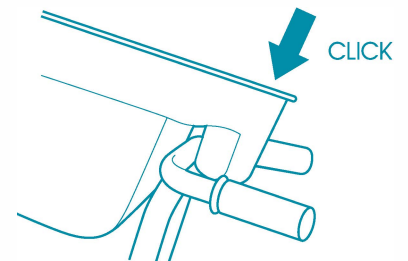


FOR FRAMES WITH A SINGLE BAR ACROSS THE FRONT


The supplied clips are required to secure the Caddy to the frame

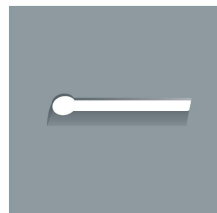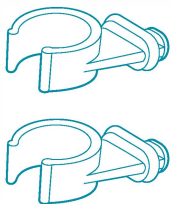

Prior to fitting – the clips supplied will need to be fitted into the channel at the rear of the Caddy

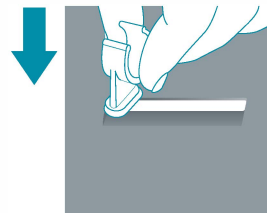

1. Locate the circular pin into the circular hole at the rear of the Caddy

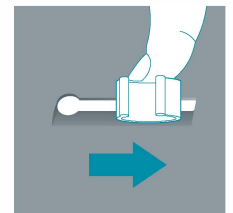


2. Slide along the channel

TO FIT THE CLIP



FOR STANDARD FRAMES

The clips should be angled outwards

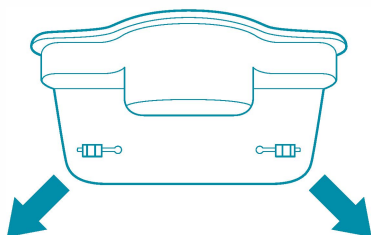


FOR NARROW FRAMES

The clips should be angled inwards, towards the centre of the Caddy

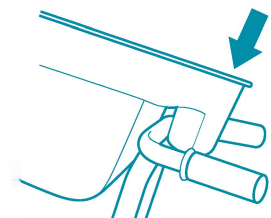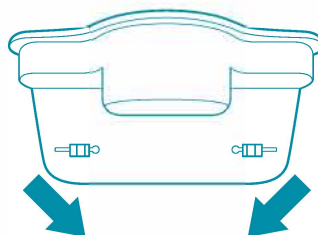


1. Push fit the Caddy onto the frame

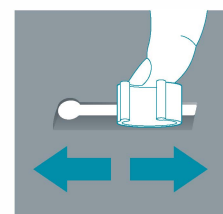



2. Slide the two clips along the channel so that they can be located onto the two front legs of the frame.

USING THE CADDY

- Once the Caddy is fitted, it can be used with or without the tray or cup holder.
- To fit the tray, simply position on top of the Caddy.
- There is no locking mechanism, it simply sits in position on top of the Caddy, secured by the lip on the tray..

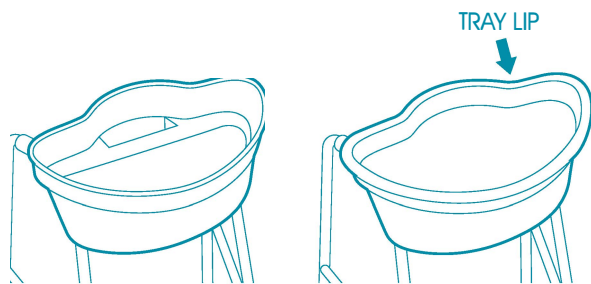

- When the tray is fitted, the cup holder can be fitted if required.
- The cup holder sits on the tray secured in position by the moulded rim on the tray.

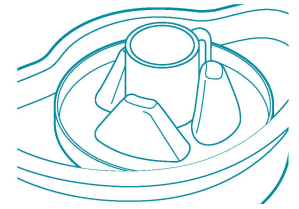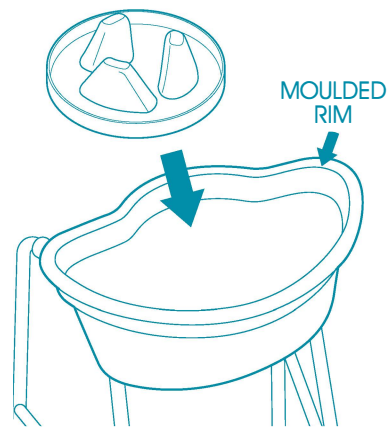

Note, the mug holder should always be used with the circular base positioned within the tray.

RESTRICTIONS FOR USE

The Buckingham Walking Frame Caddy has been designed solely to carry light, everyday personal items and should not be used to carry pets, small children, fuel of any kind, large volumes of liquid, heavy tools, domestic appliances or equipment.

- **Do not carry items that overhang the tray or are larger than the container.**
- **Do not to carry hot drinks on the tray without the Buckingham Mug Holder.**
- **Boiling drinks and teapots should not be carried using the Caddy.**
- **Only for use on wheeled walking frames.**
- **Maximum carrying weight 2kg (4.4lb).**

CLEANING

- All parts of the Caddy may be washed up to temperatures of 75°C
- All components of the Buckingham Caddy can be easily cleaned using either mild detergent and a cloth or a antibacterial wipe.