



Scooter Service Schedule _____ **Date** _____

Make **Model**..... **Serial No.**.....

Batteries & Charger

Load Test Battery 1.....	%
Battery 2.....	%
Check Terminals & Connectors	
Check Charger Cables & Plugs.....	
Check Charger Operation & Indicators	
Check Battery Location	

Control System

Check Throttle Control Operation.....	
Check Speed Control Operation	
Check Control Cable & Connectors.....	
Check High / Low Switch if Installed	
Check Tiller Head Display.....	

Ancillaries

Check Headlights	
Check Tail Lights	
Check Trafficators	
Check Hazard Light Operation	
Check Horn	
Other	
.....	

Motor / Transaxle

Check Drive / Neutral Operation.....	
Check Brake Operation	
Check Motor Brushes Condition	
Dismantle Motor Check Commutator	
And Clean Carbon Dust as Necessary.....	
Check Transaxle Operation.....	
Check Transaxle & Motor Mountings	
Check Condition & security of Rear Wheels.....	

Steering and Suspension

- Check Steering Components
- Condition & Adjustments
- Check Front Wheel Bearings
- Check Condition of Front Wheels
- Check Tiller Head Adjustments.....
- Check Front Suspension Components.....
- Check Rear Suspension Components
- Check Rear Wheel Condition & Security

Chassis & Frame Components

- Check for Mechanical Damage
- Check for Corrosion
- Check Security of Bolts & Fasteners.....

Tyres

- Check Condition
- Check Inflation
- Recommendation

Cowlings & Shrouds

- Check Condition & Attachments

--

Seat

- Check Security of Mountings
- Check Adjustments & Position Locks

Accessories

- Check Condition & Security of Accessories
-
-
-

Road Test & Report

Serviced by

Date: / /

Your Next Service will be due - / /