

# USER MANUAL

## Compact Easywipe Folding Bottom Wiper





BHCPEW

# buckingham compacteasywipe

Personal hygiene aid



Simply unfold the compacteasywipe and follow instructions for use below

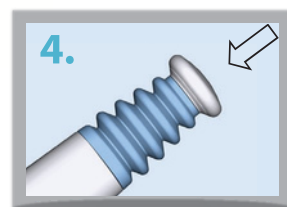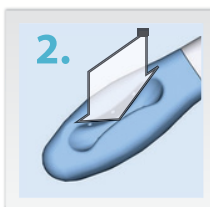

The easywipe has a soft, rounded silicone end (A) to which toilet tissue or wipes are attached and a push button end (B) which releases the tissue or wipe after use. It is simple to use and can be carried discreetly in the travel pouch provided. It is light, sturdy and easy to keep clean.

## Instructions for use

Select the length of toilet tissue which should be approximately 6 sheets (60 cms) long. Fold the length of tissue in half. Hold the easywipe and tuck one end of the toilet tissue into the bean shaped recess (1). Wrap the tissue paper around the head of the wiper end and tuck the loose end into the same recess (2). The toilet tissue will be held in place for cleaning.

The easywipe may be used to clean the vaginal and anal area following toileting. It is recommended to be used between the legs, from the front (3) or from behind, whilst seated on the toilet.

The soiled material is ejected by pushing the button at the end of the easywipe and disposed of into the toilet. Repeat the above process as required.

Use a moist wipe or a disposable cloth with mild soapy water to clean the soft end of the easywipe.

## Specifications:

The easywipe is ergonomically designed to be comfortable, safe to use and to assist reaching when wiping is difficult. It is latex free.

## Please read this:

FOR EXTERNAL USE ONLY.

DO NOT USE THIS PRODUCT IF THERE IS ANY SIGN OF DAMAGE TO THE END MARKED (A) ABOVE. YOU MUST DISCARD THE PRODUCT.

Only to be used for the purposes of external, personal cleansing and wiping.

Do not use in shower or bath.

To maintain personal hygiene the easywipe should not be shared between people.

Not to be used on animals or young children.

Only use the easywipe for the intended purpose.

Any excessive force applied to the hinge (C) could result in damage to the product.

If the hinge is damaged then you MUST DISCARD THE PRODUCT.

DO NOT use the easywipe if any damage to any part of the product is noticed.

## Cleaning

Do not immerse the easywipe in water.

When cleaning wipe all over with a soft disposable cloth in mild soapy water.

Dry with a soft disposable cloth.

It is recommended that the easywipe is cleaned after each use.

Due to hygiene concerns the product, once removed from the packaging, cannot be returned to us.