# daytot joey

USER MANUAL

Please read all instructions carefully before using Joey. All instructions can also be found on our website [www.daytotjourneys.com](http://www.daytotjourneys.com) along with videos and FAQs.

---

## Contents

Section 1:	Intended Use	02
Section 2:	Certification	02
Section 3:	Warnings & Cautions	03
Section 4:	Terms of Warranty	06
Section 5:	Safety Information	06
Section 6:	Precautions & Contraindications	07
Section 7:	Cleaning & Care	08
Section 8:	Daily Inspection	09
Section 9:	Reissuing Daytot Products	10
Section 10:	Instructions for Use	11
	10.1 How to unpack and Assemble Joey	11
	10.2 Attaching the Stabiliser	12
	10.3 Inserting the Cushion	13
	10.4 Adjusting the Insert – Size 2 Only	14
	10.5 Attaching the Covers	16
	10.6 Attaching the Bib	18
	10.7 Preparation and Set-Up	19
	10.8 Using Joey	22
	10.9 Disassembling Joey	23
	10.10 Accessories	23
Section 11:	Technical Information	24
	11.1 Product Dimensions	24
	11.2 User Sizing	25
	11.3 Sizing Table	26
Section 12:	Table of Symbols	27
Section 13:	Frequently Asked Questions	28

## SECTION 1: Intended Use

### **Intended use**

Joey is an early intervention crawler for infants at risk of motor delay.

For use during their first 6 months, Joey helps enhance early motor skills, by encouraging independent movement while positioned in prone. Joey supports the infant, allowing them to independently move, while building the important foundations for their future physical & cognitive development.

### **Indications for use**

Joey is indicated as a low assessment product available direct to parents or prescribed for use by the infants' therapist, paediatrician, or doctor. Users may include, but are not limited to, premature infants, non-ambulatory infants with a neuromuscular, congenital, or chromosomal disorder, or global developmental delay. These conditions typically affect the infants' gross motor skills and posture.

Intended to support the infant in a prone position, Joey enables the hands and feet to be free to move. In this position, the infant will work to develop head and trunk control, shoulder stability and upper limb weight-bearing.

Use of Joey for therapeutic outcomes would likely coincide with ongoing intervention from a variety of multidisciplinary teams to develop the infant's physical, cognitive, social, and sensory skills.

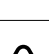
---

## SECTION 2: Certification

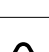
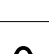
Joey is a Class I Medical Device under EU and UK regulations. As the manufacturer, Daytot Ltd, declares that this product conforms to the Medical Device Regulation (2017/745) and UK Medical Device Regulations 2002.

## SECTION 3: Warnings & Cautions

### Unpacking

	<p>Do not leave plastic packaging in reach of children – Risk of suffocation. Store or dispose of appropriately.</p>
	<p>Do not leave small parts in reach of children – Choking Hazard. Store appropriately.</p>
	<p>Take care to read all instructional information fully before assembly or use. Digital copies of all information related to the product are available on the Daytot website.</p>
	<p>If in any doubt as to the correct operation of the product, please discontinue use and contact Daytot Customer Services at <a href="mailto:hi@daytotjourneys.com">hi@daytotjourneys.com</a></p>
	<p>We recommend that you air the Activity Mat for 24 hrs and wash both sides with warm soapy water before first use.</p>

### Assembly

	<p>Apply the bib to the cover neatly ensuring there is no excess material gathered in the head area. Failure to do so could result in the obstruction of airways.</p>
	<p>Ensure all parts are assembled correctly as per <i>Section 10 'Instructions for Use'</i>.</p>
	<p>Avoid finger trap when assembling and/or disassembling any of the detachable parts. Take care to hold parts away from the connecting areas.</p>
	<p>Ensure cover is securely attached to the body prior to use, as shown in <i>Section 10 'Instructions for Use'</i>. Failure to do so could result in the infant falling out of the product.</p>

## Setup For Use

	The product is to be checked daily to ensure that all components are securely attached and safe for use. See <i>Section 8 'Daily Inspection'</i> .
	Ensure the infant is always supported and positioned correctly in product prior to use, as shown in <i>Section 10 'Instructions for Use'</i> . Failure to do so could result in the infant falling out of the product.
	Ensure the harness is firmly secured around the infant, paying attention to the Velcro areas. Failure to do so could result in the infant falling out of the product.

## Operation

	The product must only be used indoors on a flat, firm and level surface. Product should not be used on high surfaces (e.g. tables).
	Infants should not be left unattended at any time while using the product.
	Pay close attention to the infant's head position. Ensure their airways are always clear.
	Ensure the product always remains stable during use and cannot tip over with the infant in the product.
	Do not lift the product up from the floor surface when the infant is attached.
	Do not let the infant sleep on the product. Sleeping on their front can pose a suffocation risk and is thought to increase the risk of sudden infant death syndrome (SIDS). <i>Ref: NHS Guidelines and US Safe to Sleep® campaign.</i>

	The Daytot Activity Mat can be used to provide a clean and consistent surface. Alternative mats have not been tested and are not recommended for use with the product.
	No other devices or products should be used in conjunction with Joey.
	Ensure the infant's hands / fingers / feet and toes are always kept away from the underside of the product to avoid injury.
	Note the key positioning requirements for sizing and ensure the right configuration is being used for the infant. See <b>Section 10 'Instructions for Use'</b> .
	The Joey covers contain latex.

### Cleaning & Storage

	Do not remove the wheel units from the product.
	Store out of reach of children.
	Do not store the product in direct sunlight or in extreme temperatures.

### Disposal

	If disposing of the product, please follow the guidance of the local waste management facility. Constituent parts can be separated to aid recycling where possible.
---	---

## SECTION 4: Terms of Warranty

Products can be returned to Daytot for warranty repair or replacement up to 12 months from purchase. In all cases, it is the customer's responsibility to ensure the goods being returned are suitably packaged, preferably in the original packaging.

Please contact Daytot Customer Services to notify us and obtain a case reference ID before returning your product.

The customer is responsible for the cost of returning the goods. Once returned to us we will repair or replace your goods and return them to you within 14 days free of charge.

---

## SECTION 5: Safety Information

Notice to the user and/or operator: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or operator is established.

### General

- The Daytot Joey complies with flammability test EN1021 Part 1 & 2. However, the product should be kept away from naked flames and all sources of ignition.

### Assembly

- Do not modify or incorporate any components other than those supplied. Spare parts are available by contacting Daytot Customer Services at [hi@daytotjourneys.com](mailto:hi@daytotjourneys.com)

### Set-up for Use

- Ensure there is a small amount of clearance between the shoulder straps and the infant's neck.
- Keep Velcro hook areas and unpadded hardware surfaces away from the infant as they may cause minor injury.

### Operation

- If the infant exhibits any form of distress beyond what is considered normal for that infant, remove them from the product as quickly as is safely possible. The infant may take time to become accustomed to the product.

- In the case of skin irritation or adverse reaction, remove the infant and notify Daytot Customer Services.
- Ensure the harness remains securely fastened at all times during use. If the harness becomes loose, stop the training, re-position and re-secure the infant.
- Ensure the infant can always move their head freely during use.
- Ensure all wheel units are in contact with the floor surface during use. The two outer stabiliser wheels may lift momentarily during use. If in any doubt about the stability of the infant or product, discontinue use and contact Daytot Customer Services.

### **Cleaning & Storage**

- Ensure regular cleaning of all parts as per *Section 7 'Cleaning & Care'*. Failure to follow these instructions could lead to accelerated wear / malfunction of the product or unwanted bacterial build-up.
- Always ensure that the product is fully cleaned before using with different users.
- Store in an appropriate place to stop trip or fall hazards and keep out of reach of children.

## **SECTION 6: Precautions & Contraindications**

Joey can be used from 37 weeks gestational age (full term). Joey can be used at home, in a hospital, early intervention, or clinical setting.

### **Precautions**

- Fixed spinal deformities such as scoliosis or kyphosis
- Severe high tone (very tight muscles)
- Severe muscle contractures
- Severe muscular or skeletal asymmetry including hip dislocation
- Progressive and muscle wasting conditions
- High risk or recent history of bone fractures
- Strong, uncontrolled movement patterns
- Poorly controlled seizure activity
- Infants' dependent on the use of percutaneous endoscopic gastrostomy (PEG)
- Medical instability including compromised respiratory or cardiovascular function
- Dependant on 3rd party life sustaining apparatus such as oxygen tanks.

## Contraindications

There are no contraindications associated with the use of Joey. All precautions are taken at the discretion of the prescribing clinician or parent.

Daytot recommends that a trained and experienced clinician is present during initial assessments, set-ups and when the product is reissued, to minimise risk.

Therapists should use the precautions & contraindications list as a guide. Where the prescribing therapist feels that an infant is suitable, we ask that they do a risk assessment and proceed based on their clinical expertise.

## SECTION 7: Cleaning & Care

- Sessional, daily, and weekly product cleaning is recommended.
- Disassemble the product into its various parts and follow the specific guidance for each part.
- Do not use bleach to clean the product or any of its parts.
- Do not load product unnecessarily by storing it under other devices.
- Store in an appropriate place to stop trip or fall hazards.

### Hardware

- Do not use abrasive cleaning agents or appliances.
- If required, disassemble, and clean the plastic parts with warm water, domestic anti-bacterial soap, and a clean dry cloth.
- Do not attempt to remove wheel units to clean.
- If required, clean the moving surface of the wheel units with a clean dry cloth. If required use a small amount of warm water to dampen the cloth. Ensure all wheel units can run freely and are not obstructed by dirt or grime.

- Do not fully submerge the product in water.
- Make sure all hardware parts are completely dry before reassembling / use.

### **Upholstery**

- Do not tumble dry or iron any parts.
- Limit staining as quickly as possible with an absorbent cloth, towel, or sponge. Soap and warm water sponging are effective for ordinary soiling and minor spills. Be careful not to saturate the fabric as it could cause the stain to spread.
- The upholstery covers can be machine washed at 40 degrees. Remove the cushion before washing.
- The bibs can be machine washed at 40 degrees.
- To ensure that Velcro remains 'sticky', remove any 'fluff' build up on a regular basis by using either a dry toothbrush, wire cat/dog brush or an adhesive lint roller.
- Make sure all upholstery items including foam parts are fully dry before reassembling.

### **Cushion**

- If required, use a cloth with a small amount of warm water and domestic anti-bacterial soap to clean the foam. Ensure the cushion is completely dry before use.
- Do not machine wash or tumble dry.

### **Activity Mat**

- Wipe-clean with warm soapy water or disinfect with household disinfectant. Do not use bleach. Wash before first use.
- 

## **SECTION 8: Daily Inspection**

The following guidance should be followed before each session. If you have any doubt as to the safety of the product, stop using immediately and contact Daytot Customer Services at [hi@daytotjourneys.com](mailto:hi@daytotjourneys.com).

- Ensure all plastic parts are not damaged. Check for large chips or cracks.
- Ensure all upholstery parts are not damaged. Check for tears, loose stitching, excessive soiling, and that elastic parts still have integrity.

- Check that the stabiliser is attached securely and that the retaining knob is finger tight and secure. If screw threads become crossed, please contact Daytot Customer Services.
- Check that all wheel units can move freely in all directions and that the moving surfaces are free from large chips or dents.
- Ensure all wheel units are in contact with the floor when set on a flat surface, excluding the central stabiliser wheel (1mm clearance).
- Check that the cushion and covers are fitted correctly to the plastic parts. If they have become damaged, please contact Daytot Customer Services.
- Check that the bibs are not damaged and are fitted neatly ensuring there is no excess material gathered in the head area. Failure to do so could result in the obstruction of airways.
- Keep an eye on the key positioning requirements for sizing and ensure the right configuration is being used for the child.

## SECTION 9: Re-Issuing Daytot Products

Before re-issuing a Daytot product, we recommend that it is disinfected using the following procedure:

- ***Note: Always clean the product first to remove accumulated soiling prior to disinfecting/ sanitising.***
- All hardware surfaces should be wiped over with disinfectant surface wipes or liquid disinfectant on a soft cloth. Suitable products are 70% IPA, 70% Alcohol or antibacterial wash and warm water. The manufacturer's instructions for the chosen product should be followed. Product upholstery and fabrics should be cleaned thoroughly in accordance with ***Section 7, 'Cleaning and Care'***.
- Ensure a copy of the user manual is supplied with the product. If in any doubt of the continued safe use of your Daytot product or if any parts should fail, please cease using the product and contact Daytot Customer Services.

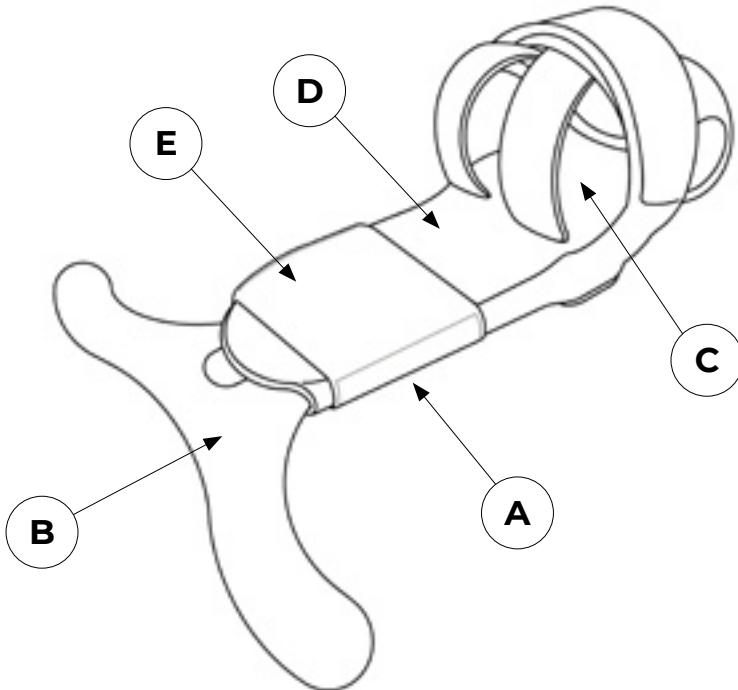
## SECTION 10: Instructions for Use

### 10.1: How to Unpack and Assemble Joey

Take care not to cut through the box as you may damage some of the parts inside. The parts will be contained in polythene bags. Carefully remove them from the box and check you have all the parts required.

#### **Included:**

- A. Body
- B. Stabiliser inc. Screw Knob
- C. Cover Pack
- D. Cushion
- E. Set of Bibs (x3)
- F. Instructions for Use


## 10.2: Attaching the Stabiliser

10.2.1 Place the body on top of the stabiliser.

10.2.2 Fix both parts together by tightening the screw knob by hand.

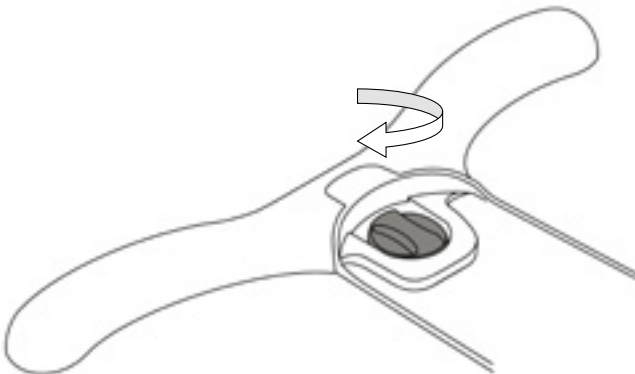10.2.1


10.2.2

### 10.3: Inserting the Cushion

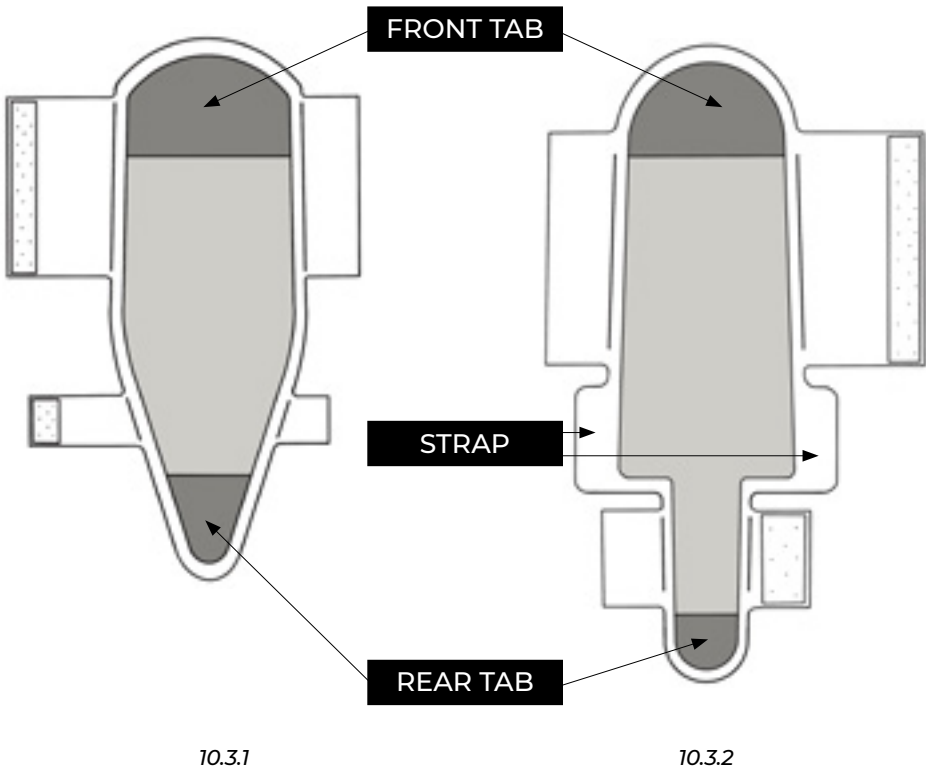10.3.1 Size 1 – Slide the cushion under the tabs at the top and the bottom of the cover.


10.3.2 Size 2 – Slide the cushion under the tabs at the top and bottom of the cover and inside both middle straps (!).

## 10.4: Adjusting the Insert \*Size 2 Only

10.4.1 Joey Size 2 comes with an insert which can be used to raise the infant's pelvis up off the floor. This will be required as they grow.

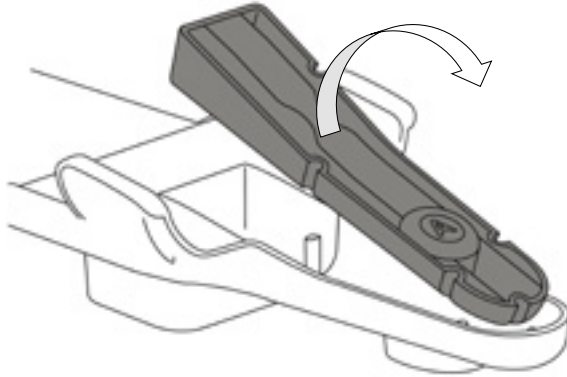
The insert has two sides: **A** and **B**.

10.4.1

**Side A** should be used in the first instance, when transitioning from the Joey Size 1. This is to accommodate infants aged 2-4 months (approx.). This setting keeps the pelvis low to the ground, allowing the infants' feet to make contact with the surface of the floor.

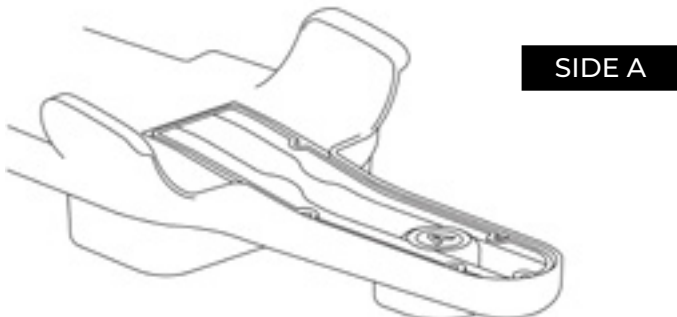10.4.2 **Side A** – The insert should be flush within the product. You will see the letter 'A' facing upwards.

10.4.2

**Side B** should be introduced at approx. 4 months old. You should progress to Side B when the infants' knees touch the floor surface which in turn prevents full flexion and extension of the legs and/or knees (crawling movement).

10.4.3 **Side B** – Simply turn the insert upside down and place it back into the product as shown below. You will see the letter 'B' facing upwards. This will raise the infants pelvis by 15mm.

10.4.4 If using insert B, you should also fix the popper on the underside of the harness to the popper at the rear of the cover.

10.4.3

10.4.4


If you have any questions regarding the insert, please refer to the sizing guide on our website, consult the infant's therapist, or contact Daytot Customer Services.


## 10.5: Attaching the Covers
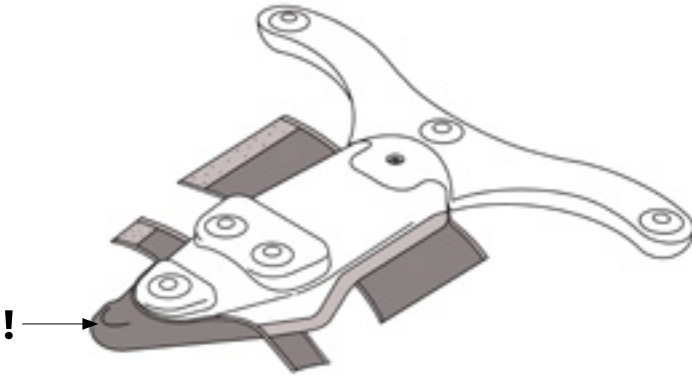
### Size 1 Cover:

- 10.5.1 Pull the cover (with the cushion already inside) over the tail at the rear of the product as shown below.
- 10.5.2 Fasten the Velcro tabs at the front and rear of the product as shown. Ensure the Velcro is straight, flat and the underside of the product is neat and tidy. This will stop the cover contacting the floor.

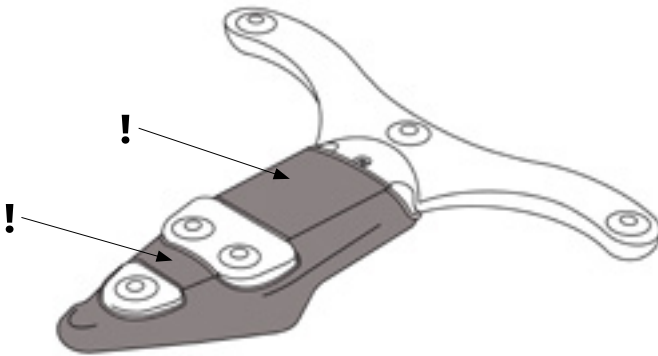10.5.2


10.5.2


**Size 2 Cover:**

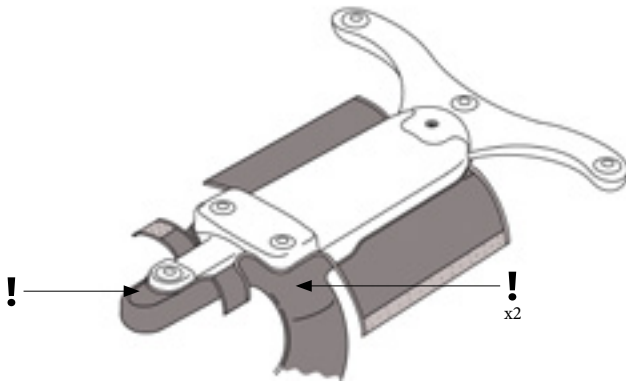10.5.3 Pull the cover (with the cushion already inside) over the tail at the rear of the product as shown below. Then slip the middle straps of the cover over the product on each side (!x2). This will make it easier to fasten the Velcro.

10.5.4 Fasten the Velcro tabs at the front and rear of the product as shown. Ensure the Velcro is straight, flat and the underside of Joey is neat and tidy. This will stop the cover contacting the floor.

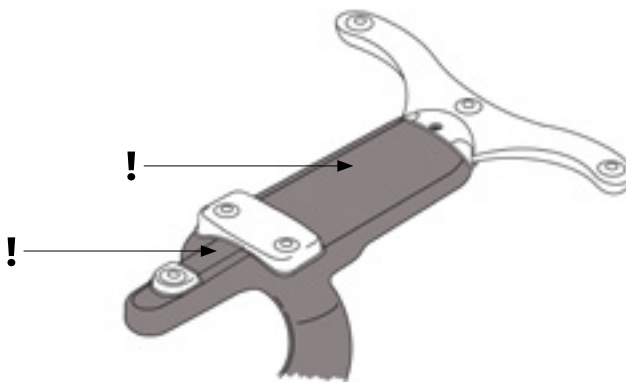10.5.3


10.5.4


## 10.6 Attaching the Bib

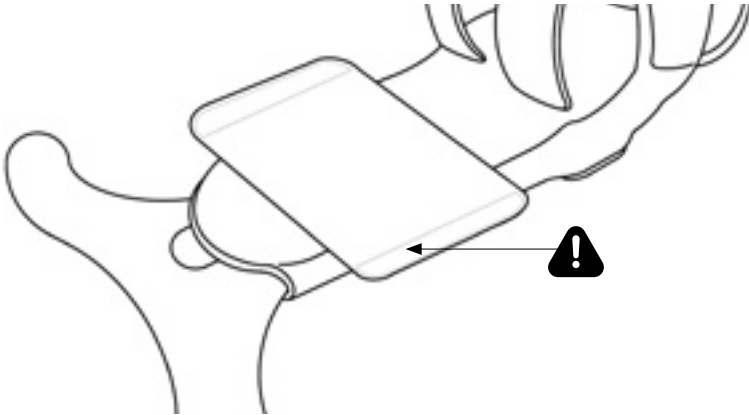10.6.1 Place a bib on the top surface of the product where the infant's head will be. Pull tightly at either side so there are no creases and secure by pressing firmly around each side.

10.6.2 Ensure the bib is smooth, secure and there is no excess material that could obstruct the infant's airways.

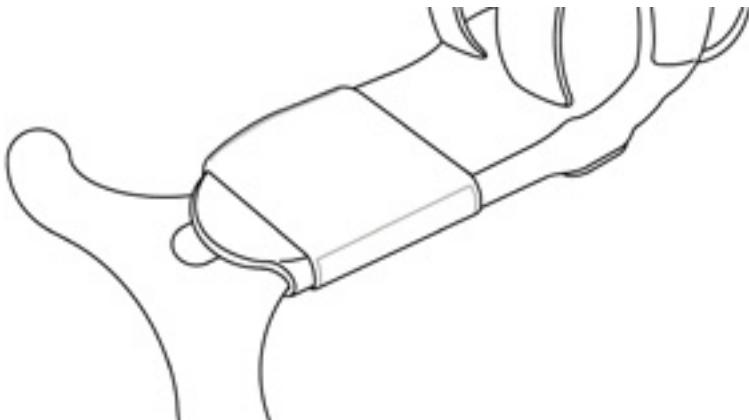10.6.1

10.6.2

## **10.7: Preparation and Set-Up**

We recommend using the product before feeding, or waiting at least 30 minutes after the infant is fed before using the product, in case of any digestive reflux. The infant should be in a calm and awake state.

We recommend that the infant is in a bodysuit with legs and forearms free, to allow for full range of movement.

The infant's feet should also be bare, with no socks or shoes on. This will allow the infant to grip the surface as they push against it.

### **Transferring the Infant into Joey**

10.7.1 Open the harness, middle strap and shoulder straps and lay them out flat as shown.

10.7.2 The adult should position themselves behind the product to ensure the infant is centrally aligned. Bring the infant to a prone position supporting their head and tummy while lowering them gently on the product. The infant should be placed tummy down with their face to one side.

Align the infants' shoulders slightly beyond the shoulder straps (approx. 2.5cm / 1")


10.7.3 Support the infant with one hand while you secure them in place. Pull the harness up between the legs onto the back. Ensure that the infants' legs can move freely and the feet can touch the floor. For smaller infants, the top of the harness can be tucked in on itself.

10.7.4 Attach each shoulder strap diagonally one at a time across the infant's body so they finish as a cross. Shoulder straps should be pulled firmly against the shoulder area of the infant creating a firm surface to push against.


10.7.5 Pull the right strap firmly across the middle of the infants' back. Then pull the left strap firmly across to the other side and press down the Velcro. Ensure the middle strap sits evenly across the middle of the infants back.

**NB.** The infant should be positioned with the head and neck flexed, the trunk well secured and the arms and legs free to move. The harness should be snug.

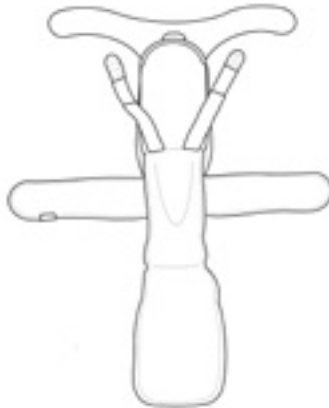To check stability and correct alignment you can give the infant a gentle sway side to side. If they seem at all unstable, remove the infant and begin this process again.

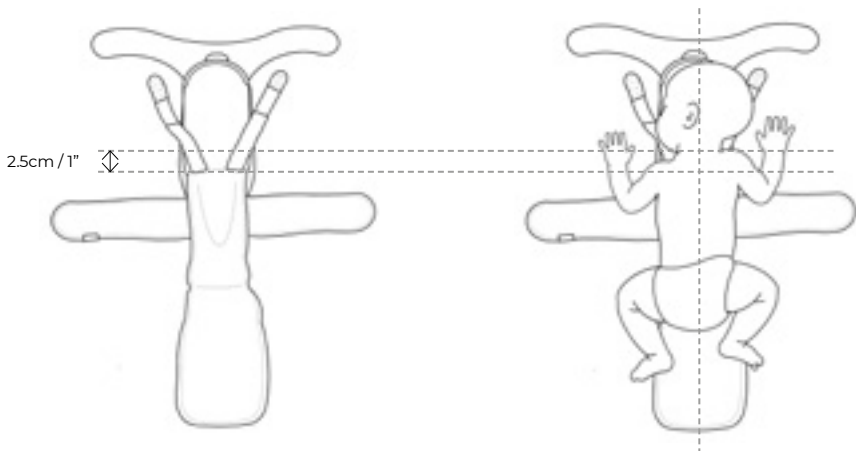10.7.1

10.7.2

10.73

10.74

10.75

## 10.8: Using Joey

Once the infant is securely positioned, give them some time to adjust to their new environment.

At the start, the adult can place the back of their hand behind the soles of the infant's feet, (horizontally) to help them push off. You can also place the palms of the infant's hands flat each time they take a step forward, to encourage weight-bearing through the arms.

The product is designed to encourage the natural development of reciprocal crawling movement (left leg, right arm, or right leg, left arm). Therefore, we advise you alternate your hand behind the infant, one foot at a time, until the infant begins to learn and initiate this movement independently.

### **During the session**

The adult can stand at the other end of the mat to encourage the infant to move forward (alternate the adult's position on the left and right side for infants with a head turning preference, to encourage rotation of the head).

You can also suggest toys to catch their attention and encourage forward movement.

Make sure the infant can comfortably turn their head to the side during training.

**NB:** It is common for the infant to cry when moving, especially before the age of 2 months due to the effort of moving.

If the infant shows signs of fatigue or genuine upset, you should remove the infant and try again later.

### **Duration of training**

A session of 5 minutes per day is recommended in the first sessions.

If well tolerated, the session can be extended to 10 minutes per day (2x 5-minute sessions). Rest breaks are encouraged.

Some infants may extend the use of product beyond 6-months, depending on their disability, condition or growth. You can continue to use the product until they outgrow this size.

*This can be identified when:*

- the infant's body is too long, and the head is protruding over the stabiliser area.
- the infant's legs are too long, and their knees are touching the mat/floor when they bend their knee/leg (inhibiting movement).
- the infant begins to show signs of weight-bearing on all fours.
- the infant starts independently crawling.

### **10.9: Disassembly**

To disassemble the product, reverse the steps from 10.2 – 10.6 respectively.

### **10.10: Additional Accessories**

As well as the accessories listed at the start of section 10, the following accessories are also available to purchase from the Daytot website at [www.daytotjourneys.com](http://www.daytotjourneys.com).

- Bibs (pack of x3)
- Daytot Activity Mat
- Spare Fabric Cover Set
- Spare Wipe Clean Cover Set

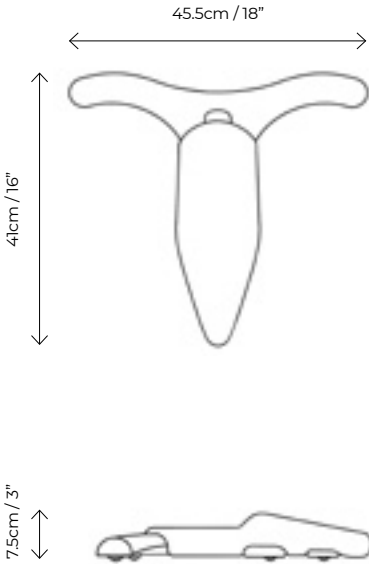
# SECTION 11: Technical Information

## 11.1: Product Dimensions


### Size 1

Size 1 Product Weight = 0.8kg / 1.8lbs


### Size 2

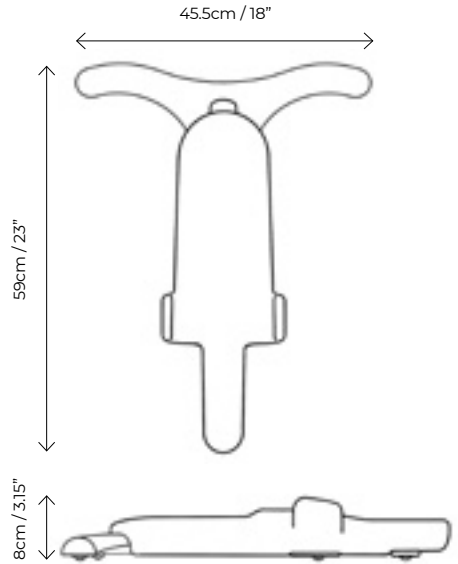
Size 2 Product Weight = 1.3kg / 2.9lbs

### Disassembled

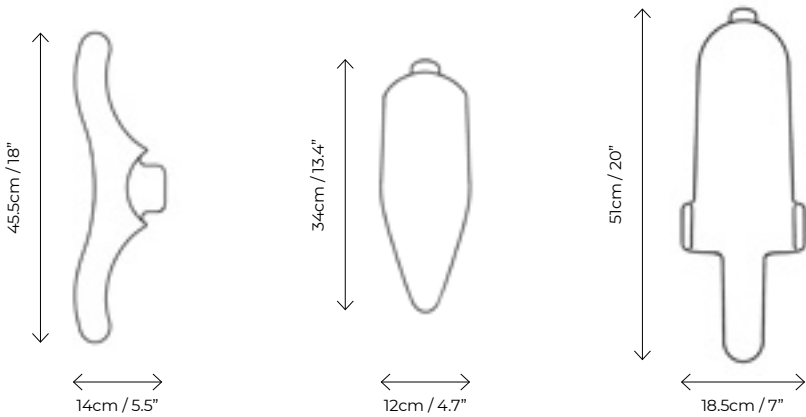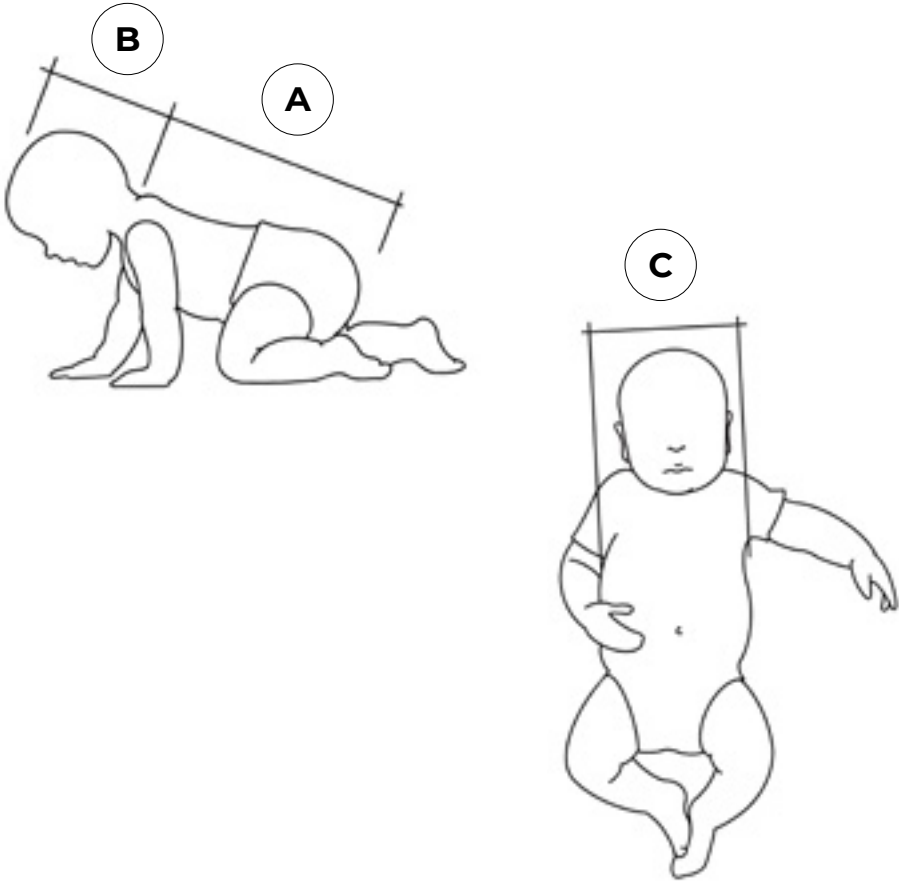**11.2: User Sizing – Key Dimensions to Measure**

**11.3: Joey Sizing Table**

<b>Dimension</b>	<b>Description</b>	<b>S1</b>	<b>S2</b>
<b>Age</b>	Approx.	0-2 months	2-6+ months
<b>Max User Weight</b>		Max 5kg / 11lbs	Max 10kg / 22lbs
<b>A</b>	Shoulder – Bottom	Max 24cm / 9.4"	Max 27cm / 10.6"
<b>B</b>	Top of Head – Shoulder	Max 17cm / 6.7"	Max 23cm / 9.0"
<b>C</b>	Chest Width	Min 11cm / 4.3"	Min 15cm / 5.9"

## SECTION 12: Table of Symbols

Symbol	Meaning
	Medical Device
	Read the Instructions for Use
	CE Mark
	Warning
	Caution
	Manufacturer
	Product Code
	Serial Number
	Max Occupancy Weight
	Date of Manufacture
	Contains Latex
	Unique Device Identification
	Machine Wash – water temperature 40°C
	DO NOT Tumble Dry
	DO NOT Iron
	DO NOT Bleach
	DO NOT Dry-Clean

## SECTION 13: Frequently Asked Questions

### **How do I know if Joey is suitable for my child?**

Joey is an early intervention crawler, designed to provide infants with an early experience and opportunity for independent movement from 37 weeks' gestational age (full term) to approx. 6 months old.

Please refer to our sizing table to check our recommended user dimensions and weights.

### **Are there any reasons not to use Joey?**

If the infant has extremes of tone or has had recent surgery (for spinal defects, for example), we recommend that you seek advice from your therapist and/or physician before using Joey. A full list of precautions can be found in Section 6 for the user manual.

### **How many times per day should I use Joey?**

This may vary depending on the infants' age, abilities, their mood, and the time of day. Be guided by the infant but avoid letting them become too fatigued as this may impact on their abilities in other daily activities.

The infant may enjoy using Joey frequently, in which case twice daily-use of 5-minute sessions may be appropriate. If the infant has extremes of tone or sensory issues, you may find that you need to go more slowly and can build up tolerance over time.

### **My child has extremes of high/low muscle tone. Are they suitable?**

If the infant has extremes of tone, they may find it difficult to use Joey for the full recommended duration of daily use. In this case, Joey can be used to help build strength and tolerance of this position which can be built up over time. If you are in any doubt as to your child's suitability, please consult with your therapist.

### **My child is unable to grip the floor with their feet. What should I do?**

We always advise using Joey with bare feet, as this will help the infant to grip the floor surface. If the floor surface you are using is too slippery, we advise that you try using the Daytot Activity Mat, or another flat, smooth surface in your home to see if this helps.

**My child is moving their legs, but their arms are fixed behind them. What should I do?**

We want to encourage the reciprocal movement of natural crawling where possible. If working with a child with low upper body strength or tone, you can support them by placing the palms of their hands, with each step they take.

Try alternating from working with the infant from behind, to working with them in front. Work with them to bring their arms by their side, with forearms or palms flat on the floor.

**My child's legs are fixed out behind them, like they're 'swimming' in the air. How can I encourage them to bend and flex their leg?**

This would suggest that the infants' hips are raised too high and is likely due to the infant being placed too far up on the product or, due to using the wrong size of insert.

*Follow these steps:*

1. Refer to the Sizing Table to check suitability.
2. Check the insert setting. If you have progressed from Size 1 to Size 2, you should use setting A. You should see the letter 'A' facing upwards. If you have just purchased Size 2, you should begin with insert A and check the amount of clearance between the infant's knee when flexed, to check if the insert is sufficiently raised. Refer to the sizing table for further guidance on insert sizing requirements.
3. Try moving the infant slightly further down the product to enable their feet to make contact with the floor surface. Ensure that their shoulders do not fall below the top of the shoulder straps as this may cause instability issues.

For further advice on sizing, please consult with the infant's therapist, or contact Daytot Customer Services.

**My child doesn't look steady, what should I do?**

The two outer stabiliser wheels may lift momentarily during use. This is normal. However, it is important to ensure that the infant is safe, secure, and comfortable in the product. Joey should only be used indoors on a flat, firm and level surface.

If you are in any doubt as to the stability of the infant or product, refer to *Section 10.7 'Preparation and Set-Up'* to ensure correct set-up and positioning of the infant on the product.

**How often should I clean Joey?**

We recommend sessional, daily, and weekly cleaning so that you get the most out of your product, now and in the future.

Always ensure that the product is fully cleaned before using with different users. Advice can be found in *Section 7 'Cleaning & Care'* of your user manual.

**What progress will my child make with Joey?**

Progress with Joey will vary from child to child. It depends on many things, such as the child's disability / condition, if there are any underlying (or associated) health conditions, and how often they use Joey.

We recommend that you speak to your therapist to help set appropriate goals and milestones for your child's development.

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## **Adaptive Imports, LLC**