



**ALLIED
MEDICAL**
LIMITED

USER MANUAL

Easylift Overbed Table

NRSN43541



USER INSTRUCTIONS

EASYLIFT OVER BED/CHAIR TABLE

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The Easylift Over Bed Table Range provides users with a firm, flat surface that is suitable for eating, writing, reading, etc. The single column construction and adjustable height makes them suitable for use with most types of beds and chairs. N43565, N43577, N43589 & N85195 are also suitable for use with wheelchairs.

2. SAFETY



Warnings:

- Do not exceed the safe working load of 25 kg (55 lbs) evenly distributed.
- Do not lean or sit on the table.
- Correctly position and adjust the table for safe and comfortable use.
- Do not place hot utensils, crockery etc. directly on the table surface. It has a maximum heat resistance of 65°C (149°F).
- Take care when moving the table over rough or uneven surfaces. Lift the table over large ridges and wide gaps.
- Take care with tilting table tops not to trap fingers between fixed and moving parts.



3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble. This product includes all tools needed for assembly.

3.1 Contents:



Figure 2

1. Table Top Assembly (Fixed or Split Top)
 2. Lift Column Assembly
 3. Base Assembly (Casters fitted)
- M8 x 55 mm Bolts x 3
 - M8 x 15 mm Cap Screws x 4
 - M8 Washers x 7
 - Split Washer x 3
 - Spanner
 - Hexagon Tool



Warning: Take care when handling the Spring-Loaded Lift Column Assembly. The column contains a compression spring that will open rapidly, in the direction shown (see Figure 3), if compressed during assembly. Handle with care.

Once assembled the weight of the Table Top ensures that the 'rapid expansion' does not occur during use.

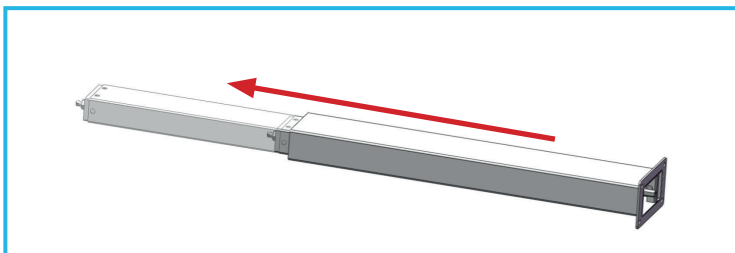


Figure 3

4. ASSEMBLY

4.1 To attach the spring-loaded column:

- Remove the protective plastic cap from the nut and bolt at the end of the column. See Figure 4.



Figure 4

- Put the Base Assembly on its side and locate the spring column in position. See Figure 5.

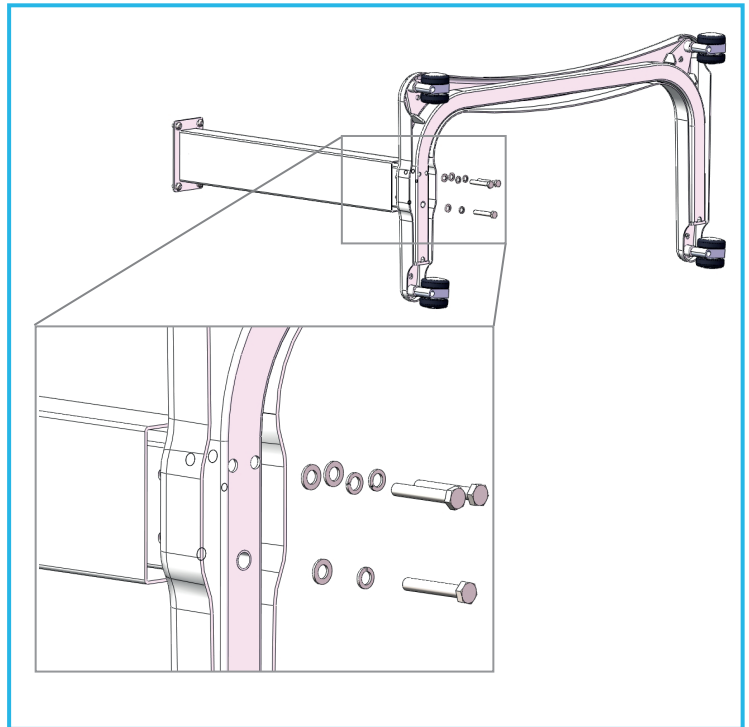


Figure 5

- Secure using three M8 x 55 Bolts, Split Washers and M8 Washers, see Figure 5.

4.2 To attach the Table Top:

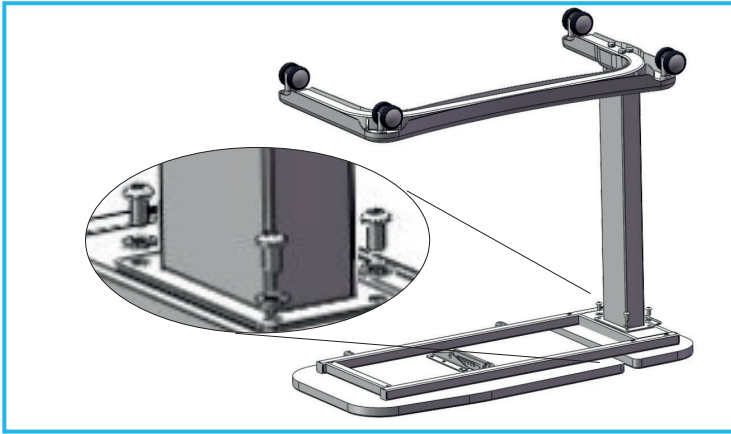


Figure 6

- Carefully put the Table Top upside down on the floor. See Figure 6.

Warning: Take care when doing this with N43589 & N85195 to not allow the hinged section to swing away from the frame. This could cause injury and damage the tilt mechanism.

- Locate the column and base assembly in position, see Figure 6.
- Secure using the M8 x 15 Cap Screws and M8 Washers.
- Allow the table to extend to the maximum height position before returning to its castors.

5. ADJUSTMENTS

5.1 Height

To **lower** the Table Top:

- Lift the lever (A) on the underside of the table near the spring loaded column, see Figure 7.
- Push down on the Table Top to lower its height.
- Release lever to lock in position.

To **raise** the Table Top:

- Pull the Table Top upwards. Hold the base down with your foot when lifting the top.



Figure 7

5.2 Tilt (Models N43589 & N85195 Only)

The tilt angle section of the Table Top will adjust to angles of 24°, 31°, 37°, 42° and 47°.

Take care not to trap fingers between the fixed and moving parts of the table when using the tilt mechanism.

To **increase** the tilt angle:

- Hold the edge of the table (B), see Figure 8.
- Lift the table top until the ratchet (C) engages at the position you need.

To **reduce** the tilt angle:

- Hold the edge of the table (B), see Figure 8.
- Lift the table top as far as it will go to release the ratchet.
- Lower the Table Top back to the flat, horizontal position.

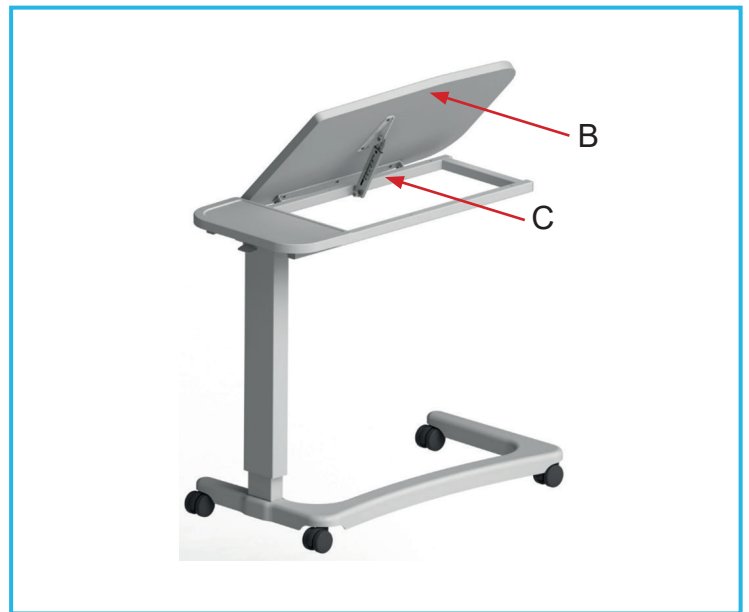


Figure 8

6. MAINTENANCE

Check the castors are securely fitted and free to rotate. Periodically, remove build-up of dirt and debris.

Check the height adjustment mechanism functions smoothly and that the table locks securely in position over the full range of height.

Check the tilt adjustment operates smoothly and locks securely in position.

Inspect product regularly for signs of wear. Report any suspected faults or damage immediately to the supplier and do not use the product.

7. CLEANING INSTRUCTIONS

Clean regularly, using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Allow product to fully dry before use/storage.

DO NOT use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

DO NOT use solvents or phenol based fluids to clean the product.

DO NOT use hot water. The table top has a maximum heat resistant temperature of 65°C (149°F).

8. PRODUCT DISPOSAL

Thoroughly clean and sterilise this product before disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

10. GLOSSARY

	Warning		Important Note
--	---------	---	----------------