


# USER MANUAL

## ELECTRIC TOILET LIFT SEAT



AML9402900000

# Product Accessories

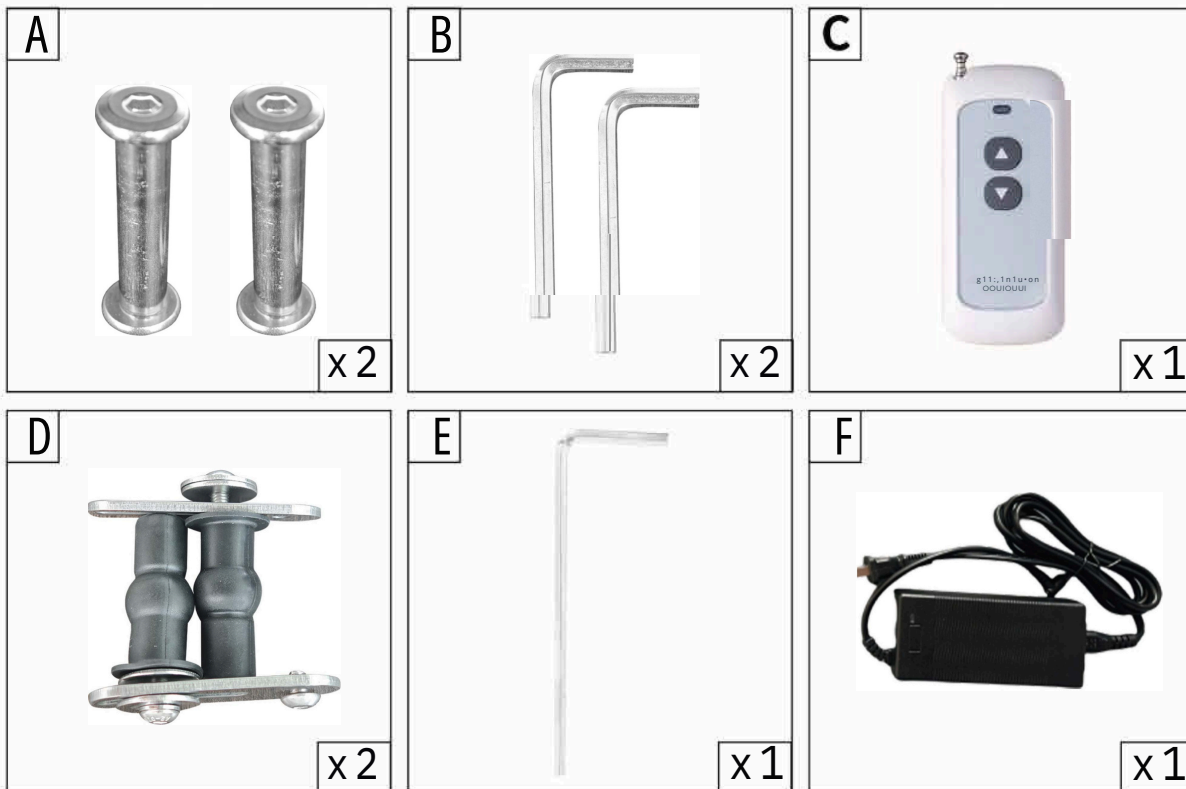

**A. Handrail locking screws x 2**

**D. Toilet fixing screws x 2**

**B. Allen wrench for handrails x 2**


**E. Hexagonal wrench inside the toilet x 1**

**C. Remote Control x 1**

**F. Charger x 1**


# Product Specifications

powersupply 110-240VAC, 50-60hz	Waterproof grade IP44
Working voltage 24v	Rated power 130w
Raise the angle Supports a 30 lift	Lithium batteries can rise and fall once full >120times
Installation requirements Toilet width < 45cm	Maximum load 136KG

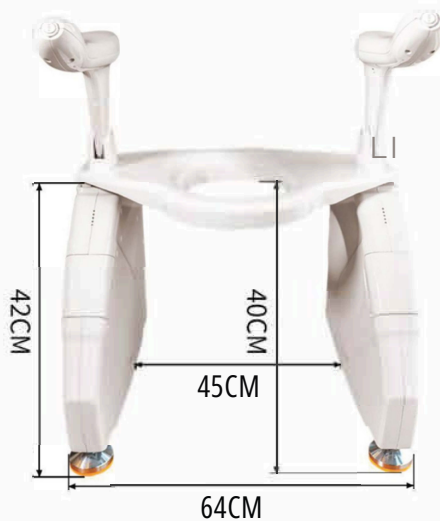

## Precautions before installation:


Please measure the width of the bathroom first to ensure it meets the installation standards of this product. The width of the bathroom body must be less than 45 centimetres, and more than 12 centimetres of space should be reserved on both sides to meet the installation requirements.

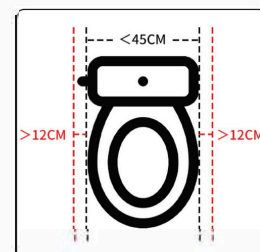


# Product Function




Fold-down armrest

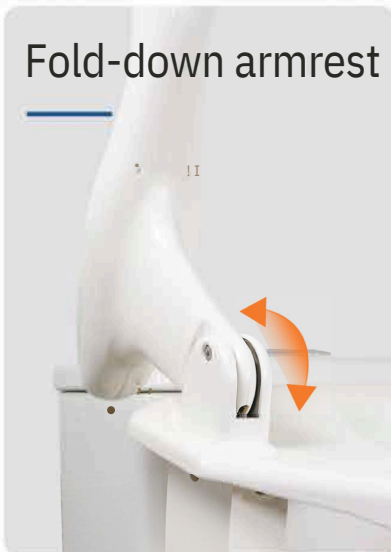

Battery display

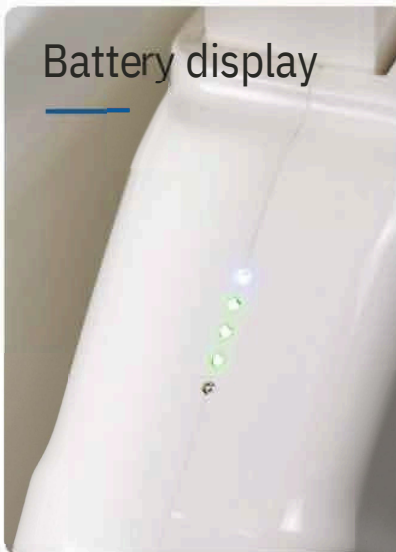


Adjustable  
Anchor screw

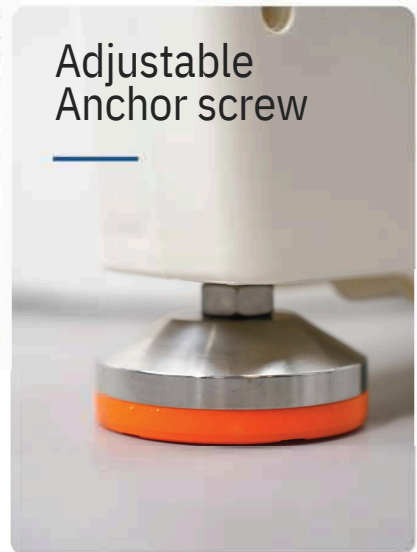

## Install the handrail



1. Align with the direction of the handrail snap shown in the diagram align and lockdown ward.

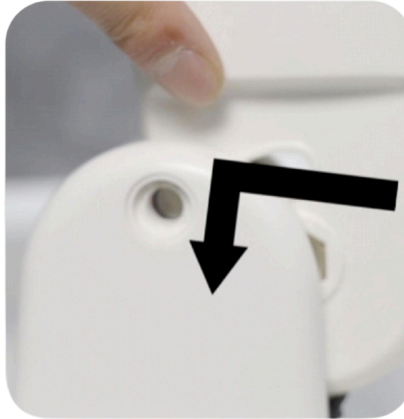

2. Gently shake and slide in to ensure the lock is in better condition.



3. Secure the handrail with screws above the main body of the machine.

## Handrail power cord

### Lift switch:



Connect the power cord



Connect the power cord


## Anchor Adjustment




The anchor bolts can be raised and lowered by 0 to 15cm. Adjust the height of the feet to match the height of your toilet




Align the iron plate with the toilet hole and secure it with screws

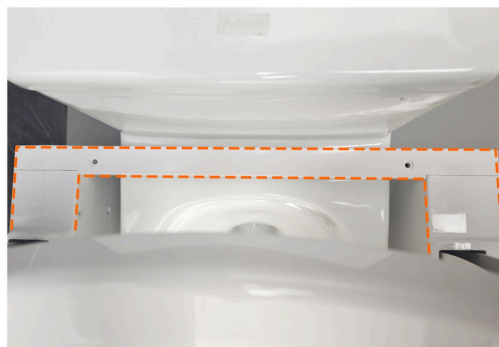

 Remember: Restore the cross-beam protective cover to its original position

# About Batteries

## Charging method:

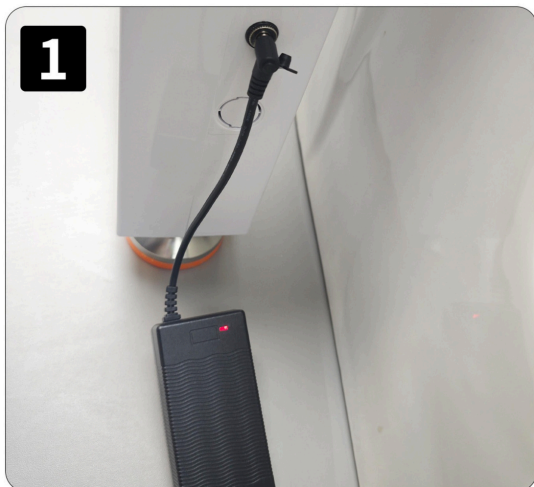

- This product is equipped with a large-capacity lithium battery and can operate independently without an external power supply
- Supports direct plugin operation, providing you with a more flexible user experience



- ① The blue light is on when it is fully charged
- ② The number of green lights represents the remaining power of the battery  
When only the red light is on, it indicates
- ③ that charging is needed. Please charge as soon as possible



- When the remote control is not in use for a long time, please turn off the remote control receiving switch. Keeping it on will continuously consume power
- Please pay attention to reasonable use to ensure the long battery life of the equipment

# Precautions

- 1 Before installation, please read the user guide carefully.
- 2 Users with limited physical conditions need to be supervised or assisted when using it.
- 3 Do not exceed the maximum load capacity of the equipment, which is 220 kilograms.
- 4 The handle is designed for easy support when standing or sitting. Please use both sides simultaneously.
- 5 Do not place your entire body weight on a single handle (each person is limited to 70 kilograms).
- 6 Do not stand on the product. It is not a climbing tool.
- 7 Do not rely your entire body weight on one side of the handrail. Instead, evenly distribute it on both sides, with each side bearing half of your body weight.
- 8 Overloading may cause damage to the handrail.
- 9 Before use, please confirm that all components have been correctly assembled and tightened.
- 10 Please stay away from heat sources.
- 11 Any serious incident related to this product must be reported to the manufacturer and the competent authorities immediately. Comply with the regulations of the member states where the user and the patient are located.

## Cleaning and Maintenance:

- The surface of this product is made of high-strength engineering plastic. It can be cleaned with a damp cloth. The main unit of this product requires no maintenance or lubrication. Try to avoid the push rod of the main unit getting wet.

# Warranty

**Your Allied Medical Lift is covered by:**

- 3 years of parts warranty
- 90 days of parts and labour

## **First 90 Days**

We'll help troubleshoot any issues and, if needed, coordinate with a local appliance repair technician. Labour and parts are covered during this time. Installation is not included.

## **After 90 Days (Up to 3 Years)**

We'll supply replacement parts free of charge (excluding grips, remotes, and toilet seats). Covered items include the lift's structure, actuators, batteries, and electrical components.

**For support, contact us on 0800 31 61 81 or [helpis@alliedmedical.co.nz](mailto:helpis@alliedmedical.co.nz).**