

Stealth Products®



**ALLIED
MEDICAL**



Lower Extremity Support

Gel Footplate Cover

Intended Use

The Gel Footplate Cover is intended for use as a gel padding system for any Quantum footplate. Its soft gel interior and fitted cover provide users with increased comfort and aid in the prevention of pressure sores and abrasions.

Installation and Use



Step 1:

Remove one of the covers from the packaging and separate the hook and loop straps so they hang freely. Position the cover over the footplate so the straps align with their respective slots.




Step 2a:

Feed the cover's front straps through both the footplate's front slots. (Only one side shown in illustration.)



Step 2b:

Feed the cover's back straps through both the footplate's rear slots. (Only one side shown in illustration.)

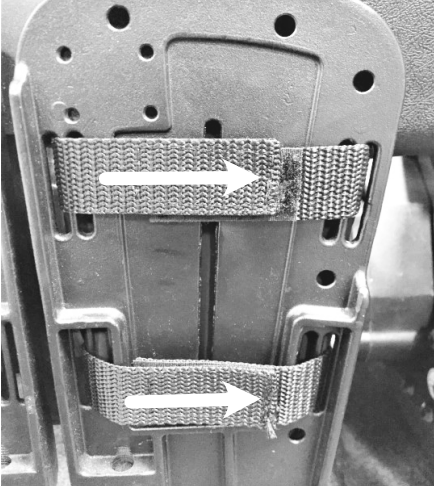

Step 3:

From underneath the footplate, pull the front straps tightly together and fasten them. Repeat for the rear straps. Make sure the cover is secure on the plate.

Repeat for the other cover.



WARNING

This product is designed to be fitted, applied, and installed exclusively by a healthcare professional trained for these purposes. The fitting, application, and/or installation by unqualified individuals could result in serious injury.

NOTICE

Remove gel from cover before washing. Failure to do so may result in damage to the product.

Care and Cleaning

We recommend that all products be examined on a regular basis for wear and function. Check the stitching on the cover and webbing for stress or damage. If loose, frayed or broken parts are discovered, discontinue use and contact your provider. Repairs should only be performed by a wheelchair service professional and with genuine Stealth Products parts, fasteners or items appropriate for the service required.

Cover should be cleaned with a damp cloth and a disinfectant. Ensure all cleaners are approved for finished steel, aluminum, plastic and upholstered surfaces.

DO NOT use bleach.

DO NOT tumble dry. Dry flat.

DO NOT iron.