

USER MANUAL


Glideboard Transfer Board



BH660140

buckingham glideboard

MADE IN UK

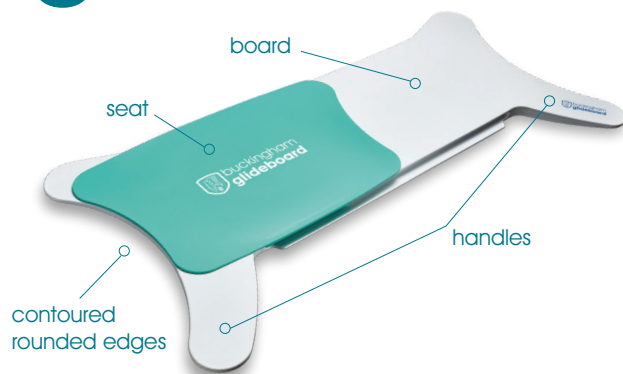

- Exciting new shape increases stability and safety
- Lightweight and multi-positional
- Easy to use around wheelchair arms
- The ease of use makes transfers easier and faster
- The moving seat reduces the effort for the both the user and care giver by over 31%


This is the Buckingham Glideboard and here's how to use it.

<p>1</p>	<p>Pick up your glideboard and insert one end under your thigh, making sure both handles of the board are in contact with your wheelchair, forming a stable base for you to transfer.</p>	<p>2</p>	<p>The other end should be flat and positioned on the other surface where you want to glide towards.</p>
<p>3</p>	<p>As you slide yourself onto the glideboard make sure that your hand holds the seat in place under your thigh – you can do this by spreading your palm across both the board and the edge of the sliding seat.</p>	<p>4</p>	<p>This way you can be sure that your body is directly on top of the seat, before you glide from your wheelchair.</p>
<p>5</p>	<p>Next, place one hand on the surface you are heading towards whilst keeping the other on your chair for greatest stability and glide across. Make sure it is stable.</p>	<p>6</p>	<p>To remove the board, gently tilt to one side and pull the board away from the body.</p>

Cleaning and Caring for your Glideboard

The Glideboard has been designed to give you years of trouble-free transfers between surfaces. To assist in this the Glideboard and seat should be cleaned regularly using warm soapy water and a sponge.

The seat easily detached from the board – simple place your fingers underneath the seat edge at the side of the board and pull up. The seat will snap off of the board allowing all surfaces to be easily and dried

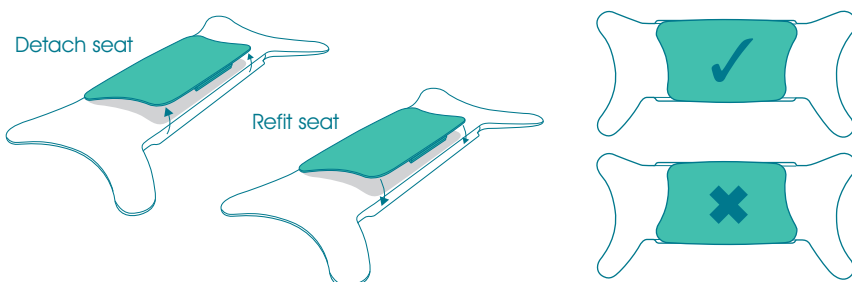

When you reattach, the seat make sure that it is the correct way round. If fitted the wrong way it could result in your skin or clothing being pinched which could be uncomfortable.

To refit the seat to the board simply take the seat and put one end of the seat in position on the board and push the other end down – the seat will easily “snap” firmly into position on the board.

When using your Glideboard take care to prevent food or other debris from getting under the seat between the seat and the board. Should this happen please clean both items before using.

The board can be used without the seat attached .

It is advised to wear undergarments when moving across the board to prevent any entrapment issues or skin shear.

If for any reason your board and seat become damaged you should stop using it for the intended purpose of surface transfers. Regularly inspect the board and seat for any cracks, fractures, weakness or any visible signs of damage.

Warranty – 12 months for any manufacturing defects.