

USER MANUAL

Home Helper Wheeled Trolley

NRSN85146



helpis@alliedmedical.co.nz | www.alliedmedical.co.nz | 0800 31 61 81

USER INSTRUCTIONS HOME HELPER TROLLEY - FLAT PACK

🕂 General Safety Warning 🛕

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

1. INTENDED USE

The Home Helper Trolley is designed to enable users to safely transfer items and meals around the home; the lower tray is set forward on the frame to allow users to walk naturally when pushing.



2. SAFETY

WARNING this trolley is not designed to be sat upon. WARNING although this item is used when walking, IT SHOULD NOT BE USED AS A WALKING AID.



3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.


3.1 Contents:

- 1. Frame Side Right x 1
- 2. Frame Side Left x 1
- 3. Tray x 2
- 4. Cross Tube Rear x 2
- 5. Cross Tube Front x 2
- 6. E-Clip x 4
- 7. Castor Assembly x 4

4. ASSEMBLY/INSTALLATION

Note this product does not require tools for assembly

4.1 Fitting Cross Tubes:

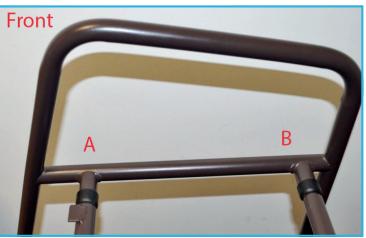

There are four tubes, two rear tubes (B) and two front tubes (A). The front tubes (with welded tabs) must be fitted in the correct orientation, with the tabs facing forward and upwards



To engage in the frame, press in the spring button and push in the tube until the spring button "clicks" into position and the button is visible through the hole in the frame.

4.2 Aligning the Side Frame:

Align the other frame side onto the four side tubes and engage the spring buttons as described above, ensure all spring buttons have clicked into place and are visible through the holes in the frame.


4.3 Fitting Trays to Frame:



With the tray sideways enter the frame and rotate into position. Engage the clips on the front of the tray onto the cross tube tabs, and then clip the rear edge of the tray onto the rear tube.

Note: Fit the lower tray first, repeat for upper tray.

4.4 Fitting Castors:

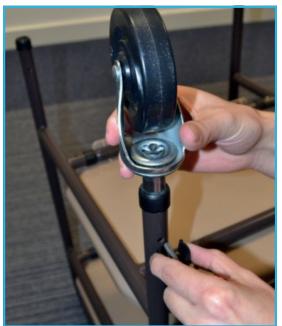

Fit the castors to the frame by pushing them into the bottom of the frame, align the holes in the frame with the holes in the castor leg and secure with E-clip. Turn product upright and check it is level and stable. *Note:* Ensure castors are set to the same height.

5. MAINTENANCE

Inspect the product regularly for signs of wear or damage, particularly the condition of the castors.

Check all fixing clips are inserted correctly and there is no excessive movement.

If you see or suspect a fault do not use – contact your dealer immediately.

6. CLEANING INSTRUCTIONS.

Clean regularly, use a general purpose, neutral detergent and warm water.

Do not use abrasive cleaners or cloths as this will damage the surfaces of the product.

Allow to dry thoroughly.

The trays are designed to clip on/off for cleaning. To remove, unclip the rear edge and lift clear. To refit, engage the clips on the front of the tray on the front cross tube and secure by clipping the rear edge of the tray onto the rear cross tube.

Check tray is level and secure before use.

7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors and Recycling Centres.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.