

# Battery Box Removal - Karma KP25.2

## Quick guide for safe battery removal

### 1. Power Off

- Turn off the wheelchair using the main switch on the joystick.
- Ensure the wheelchair is on a level surface to prevent movement.

### 2. Secure the Chair

- Engage the manual brake or keep motors engaged.

### 3. Locate the Battery Boxes

- The KP25.2 uses a dual battery box system located beneath the seat upholstery.



### 4. Release Securing Clips


- Find and release the clips on the back of the silver rails that hold each battery box in place. (Refer to diagram – callout #20 area)

### 5. Disconnect Power Cables

- Unplug the battery connector from the control system. Repeat for the second battery.

### 6. Remove the Battery Boxes

- Lift vertically and slide backward to remove. No tools required.
- Repeat for the second battery box.

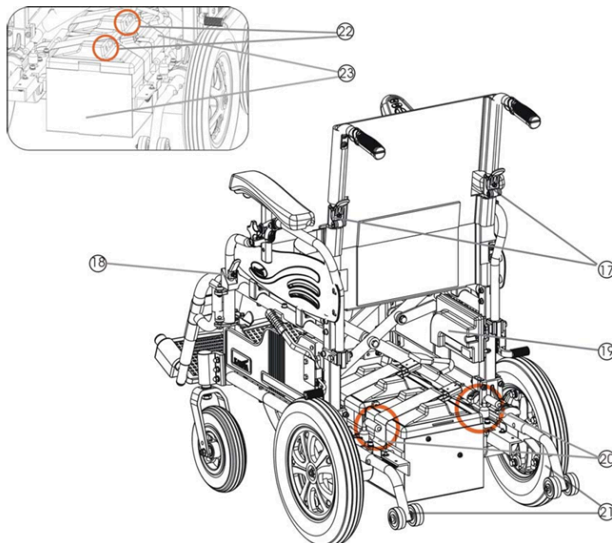

### Safety Notes

- Do not short-circuit battery terminals.
- Store batteries in a cool, dry place.
- Charge batteries only with the approved charger.

