


Karma Agile Elevating Legrests

The locking mechanism of Agile ELR design is different from the standard legrest. It is the same as current Flexx Junior (which has been changed to lockable design). Each ELR includes a screw set as shown in the drawing below. When installing the ELR onto an Agile, the screw sets must be removed from the ELR and reinstalled onto the frame.

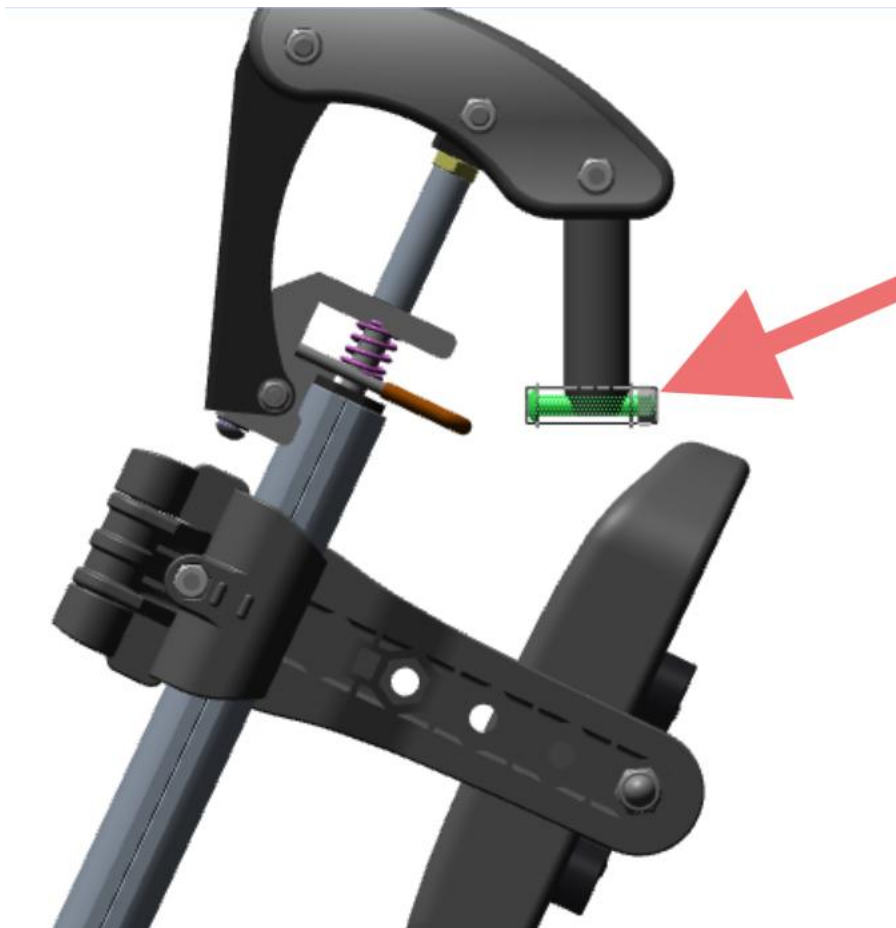

If the original screw set is missing, you may substitute it with standard M6 screws, washers, and nuts you have in stock.

Item No.	Description	Qty/Set
110803040003	Hex Nylon Insert Lock Nut-SI M6*P1*4t	1
110802040001	Flat Washer-SI M6*D13*d6.3*0.8t	2
110801040054	Hex Socket Type Binding Head Screw Full-thread M6*P1*35L	1