

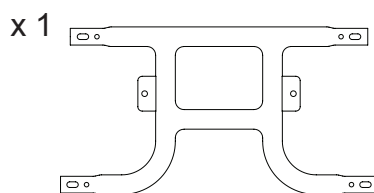
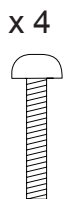
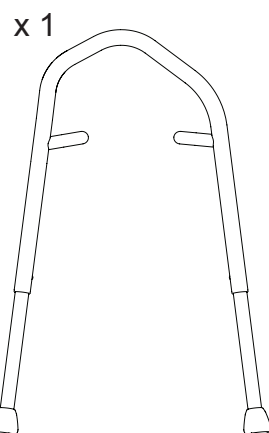


**TRUST**  
**CARE**

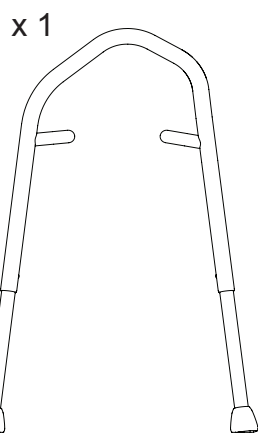





**LET'S ENJOY**

Article No. LEN101

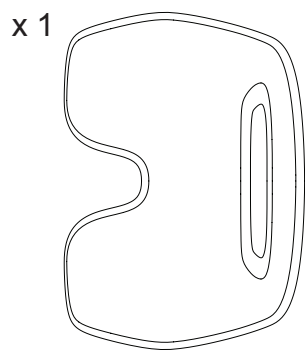

1

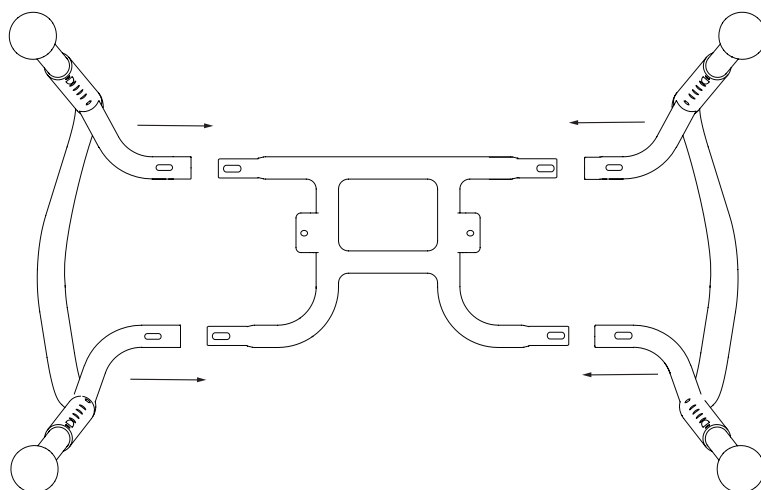

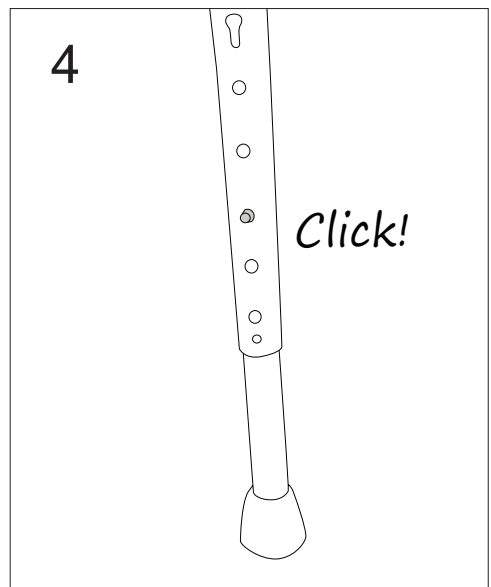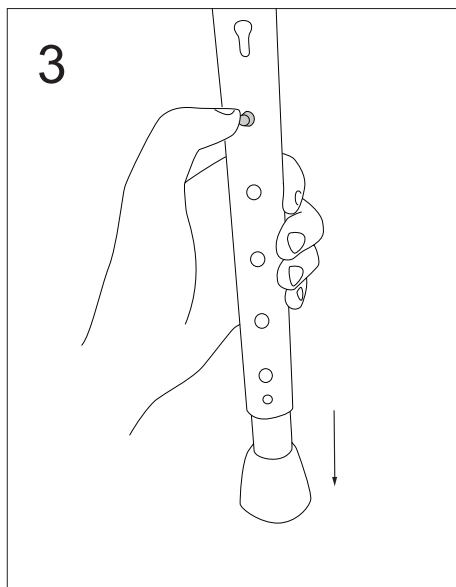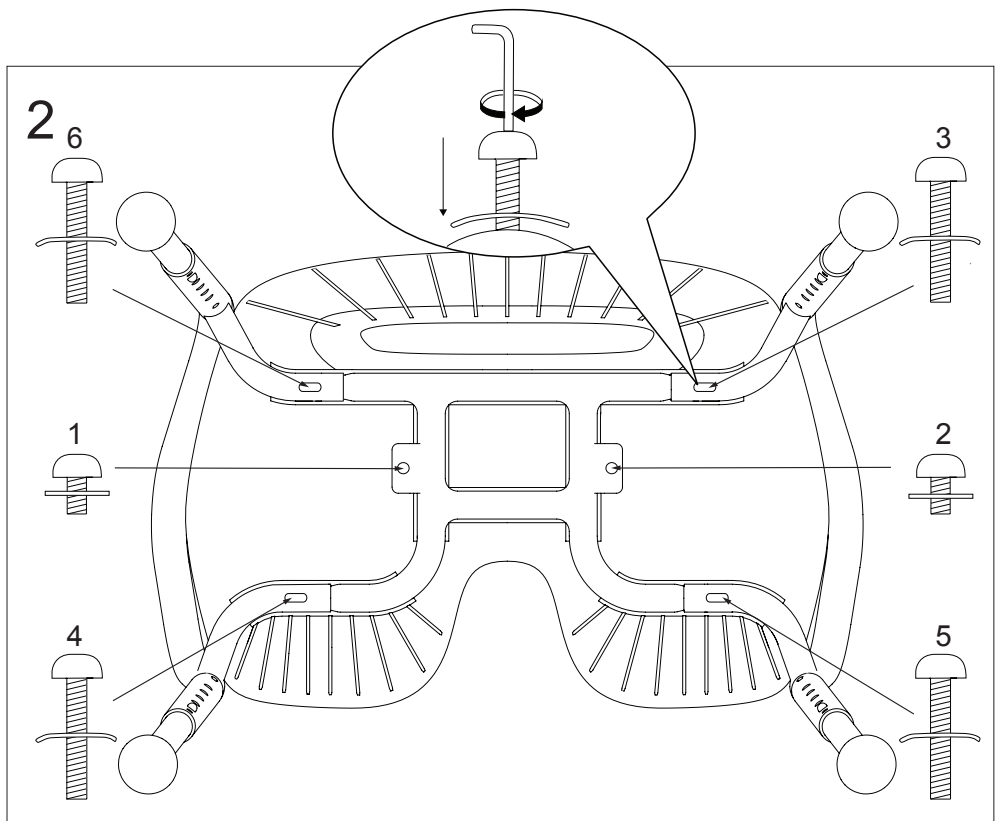

## ENGLISH

Let's Enjoy is a new bathstool with a modern design to be used in the bathroom and shower. It has a beautiful and stable construction for safe and secure use. The height of Let's Enjoy can easily be adjusted to the desired position with the buttons on the legs.

**Warning:** Do not stand on the bathstool.

**Warning:** A defective product must not be used.



**Warning:** Always adjust all of the legs to the same height to avoid falling accidents because of the stool becoming unsteady.

**Max user weight:** Do not overload. Maximum user weight is 150 kg.

**Cleaning:** Use detergent without solvent, pH 5 – 9, or 70% disinfection spirit.

**Decontamination:** Use alkaline detergent pH min 6, in max 85°C/185°F for no more than 3 min.

**Spare parts:** Use only original accessories and original spare parts from Trust Care.

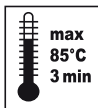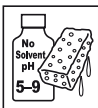

### Technical data:




**Material:** Seat ABS, Tube frame aluminium, Feet TPE

**Weight:** 2.4 kg

**Max user weight:** 150 kg

**Height:** Adjustable from 42 to 56 cm

**Colour:** Seat black, legs black, feet black

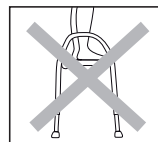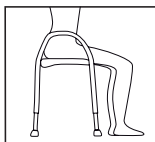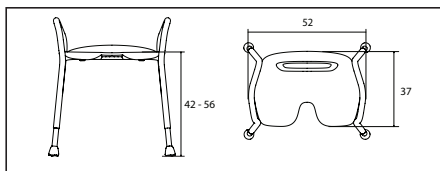

The stool is intended to sit on. Do not stand on the stool because of the risk of falling accidents.