



**ALLIED
MEDICAL**
LIMITED

USER MANUAL

Bedside Commode

MP20502



INSTRUCTIONS

To remove the bucket you must first remove the lid seat and lift the padded ring toward the backrest (as shown in the photographs). Now you can remove the bucket with ease. It comes with a handle for carrying and a lid.

MAINTENANCE

You must check the following:

- Check that the four leg height adjustment clips are properly seated.
- Check that the chair and its upholstery do not have any damage or damage. If you detect it, DO NOT use it and contact the dealer who sold it to you.

CLEAN

Wipe with water and/or mild, non-abrasive detergent. Dry the chair completely once finished.

WARRANTY

The warranty is extended for 2 years, according to the current Law of Guarantees that refers to all defects in materials or workmanship. Damage caused by natural wear and tear or improper use is excluded from the warranty.

In the event that you require a repair of your product, please contact the dealer who sold it to you. Through it, the factory will provide you with all the support of your warranty.



MODELOS:

Silla con wc CLUB (color azul) Comfortable chair with toilet

Built-in, its unobtrusive appearance conceals the function for which it has been manufactured. Designed for people who, for some reason, have difficulty move to the bathroom. Easy to clean and height-adjustable seat.



SERVICE toilet chair (grey) Toilet chair, with soft padding especially recommended for those who must stay sitting for long periods. Easy to clean, discreet design that blends easily with the home environment. Height-adjustable seat.



Common MEASUREMENTS for both models:

Total width: 60cm Total depth: 50cm Maximum weight: 100Kg. Seat height: 45-60cm
Width between arms: 45cm

PLEASE READ THE INSTRUCTIONS CAREFULLY BEFORE USING THE PRODUCT

Save these instructions. They may be useful to you later or to future users.

IMPORTANT SAFETY NOTICES

- Do not use if you detect that any part of the chair is in poor condition.
- It can't get wet. Do not use to shower or inside the bathroom.
- Be sure to adjust all four legs to the same height and check that you are on a flat, stable surface.
- Do not exceed the maximum weight indicated in the manual and on the product label, which is 100Kg.
- Asegúrese que el producto se adapta a las características y condición física del usuario.
- Make sure that the product adapts to the characteristics and physical condition of the user.

ASSEMBLY

If the packaging shows damage at the time of delivery, examine the chair for visible defects. If any damage is observed, please contact the dealer you are dealing with. You have purchased the product.

ASSEMBLY INSTRUCTIONS AND HEIGHT ADJUSTMENT

All four legs come unassembled. You need to adjust them to the right height for each user.

Remove the four clips (pins) located on the chassis and insert the leg into the tube, matching the hole in the chassis to the hole you want in the leg. Now put the clip back on and make sure it goes all the way through the tube. Perform the same operation on all four legs.



Be sure to adjust all four legs to the same height and check that you are on a flat, stable surface.

