

mouse

The **MOUSE** special needs stroller is designed for the youngest users who are unable to walk or sit in a correct position as a result of injuries, diseases or motor dysfunctions. The stroller is suitable for children at a height of **80 – 110 cm** and a maximum weight of **30 kg**.

Thanks to the backrest tilt and recline feature as well as adjustable torso and head pads you may easily adjust **MOUSE** to the required position and ensure the user with optimal protection and stability.

The shock-absorbing suspension on all four, large, gel-filled wheels together with the tilt-foot facilitate travel on uneven terrain. Modern and stylish design is an additional advantage of this stroller.

INDICATIONS

paralysis and paresis of lower limbs of various origins, paralysis and paresis of the trunk, weakening of postural muscles responsible for maintaining the correct body position, cerebral palsy, myelomeningocele, muscular dystrophies.



CONTRAINDICATIONS


In case of any contraindications to sitting position &/or passive verticalization stroller adjustments should be consulted with physiotherapist or physician-in-charge.



CE **MD** MEDICAL DEVICE - CLASS I

Colour.

  **UPHOLSTERY**
synthetic

 **FRAME**
spray painted / glossy



features. safety and comfort

TILT IN SPACE SEAT

simple mechanism



8°-23°

4 positions

RECLINING BACKREST

wide range of adjustments



16°- 88°

4 positions

ADJUSTABLE AND ANGLE TILT FOOTREST

simple and quick adjustment



84°- 173°

6 positions

SEAT MINIMIZING KIT

reducing insert



The SEAT MINIMIZING KIT in **MOUSE** is a comfortable, one-piece insert that allows you to reduce both width and depth of the seat.

STABILIZATION SYSTEM

adjustable pads



The head and torso support consists of two pairs of independent pads that are height, width and angle adjustable. Its wide range of adjustments and simplicity gives you a fast and effective adaptation to the user's individual needs.

LYING POSITION

wide range of backrest recline



With a fully adjustable seat and footrest you may quickly alter the stroller into a cozy bed perfect for a short rest or a nap.

REVERSIBLE SEAT

arrange your travel easily



The design of the stroller allows you to easily rotate the seat opposite to the driving direction. Thanks to this feature you can keep an eye on your child all the time.

FOLDING LIGHTWEIGHT FRAME

*ease of storage,
transportation and dismantling*



The construction of main elements allows you to dismantle Mouse into 3 separate units (frame, seat and wheels).

DRIVING SUPPORT

*comfort for the bumpiest
journeys*



The stroller is equipped with a shock-absorbing suspension on all four wheels and an adjustable rear suspension in the rear wheels, swivel front castor wheels with option to lock, central brake and a tilt-foot.

functional accessories. in practice



Canopy with window
torso / head pads



Seat minimizing kit



Adjustable footrest



Swivel front castor wheels
with option to lock



Quick release mount on all 4 wheels

accessories. wide range of equipment



seat minimizing kit



head / torso adjustable pads



abduction belts



adjustable wedge



therapy tray



safety barrier



tourist bag



shopping basket



rain cover



mosquito net

TECHNICAL PARAMETERS OF THE STROLLER

seat width: 24*/32 cm

effective seat depth: 22*/26 cm

backrest height: 52 cm

backrest width: 30 cm

distance between footplate and seat: 14,5 – 23 cm, 4 positions

*values are given with inserts

total weight: 17,2 – 23,2 kg

stroller length: 95 – 124 cm

stroller height: 122 – 134 cm

stroller width: 60 cm

IMPORTANT NOTE! The choice of a medical device appropriate for a specific disease entity and condition of the patient must be made with the help of a specialist doctor or physiotherapist. Before use, read the operating manual and the label or consult your doctor, as this medical device may not be suitable for you.