

PAWS

Installation and adjustment
of manual PAWS



REHASENSE



SETTING OF PAWS



1. Remove one of the gripping arms and adjust the gap so that they can be secured firmly with the locking handle. The grips can be adjusted for pipes between 19-32 mm by screwing the nut on the inside of the clamp in or out with a 13 mm socket.
2. Repeat the procedure with the other arm.



3. Fit one clamping arm into the width adjustment piece



4. Loosen all three adjustment screws for height, rotation in/out and angle with a 6 mm and 8 mm Allen key so that the entire fastening part is completely loose and flexible.



5. Drive the wheelchair into the gap and find a position to be able to lock the grip around the leg support (tube guard). When you can lock the grip, it means that all adjustment measurements are correct.
6. With the grip still locked, tighten all three screws to lock the position. Side one is now complete.
7. Uncouple the wheelchair and repeat the procedure on the other side.



8. Adjust the width so that the clamps are the same distance from the centre of the PAWS and lock with the quick-release locks. Lock PAWS to the wheelchair, loosen the locking ring screw on each side with a 4 mm Allen key.



9. Insert the locking rings towards the centre of PAWS and lock them in place. This means that the entire attachment unit will be in the correct position when, for example, you have removed them carrying PAWS in the car.