




NEW PARTS LIST

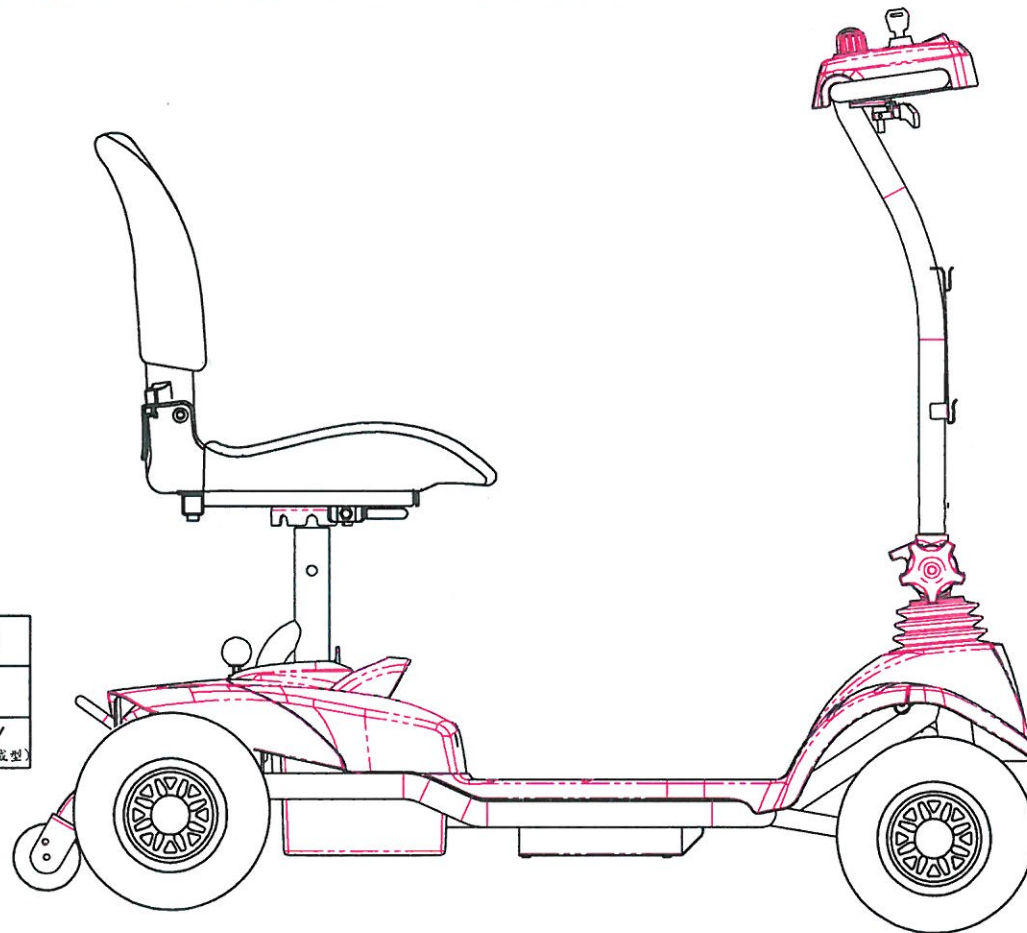

顏色 \ 版次	1	2	3	4	5	6
RD(紅色)				√ (黃色成型)		
BU(藍色)						√ (黃色成型)

Model NO. HS-118A(鋁車架)

分發單位		
營業一部		1
營業二部		1
品保部		1
生管課		1
製造二課		1
採購部		1
生產技術部		1
物料課		1
大陸上海廠		1
承認	審核	擔當
彭	黃	劉林


CT-RD-R-0021



(FIG. 1) STEERING

HS-118A

修訂日期:2021.12.24

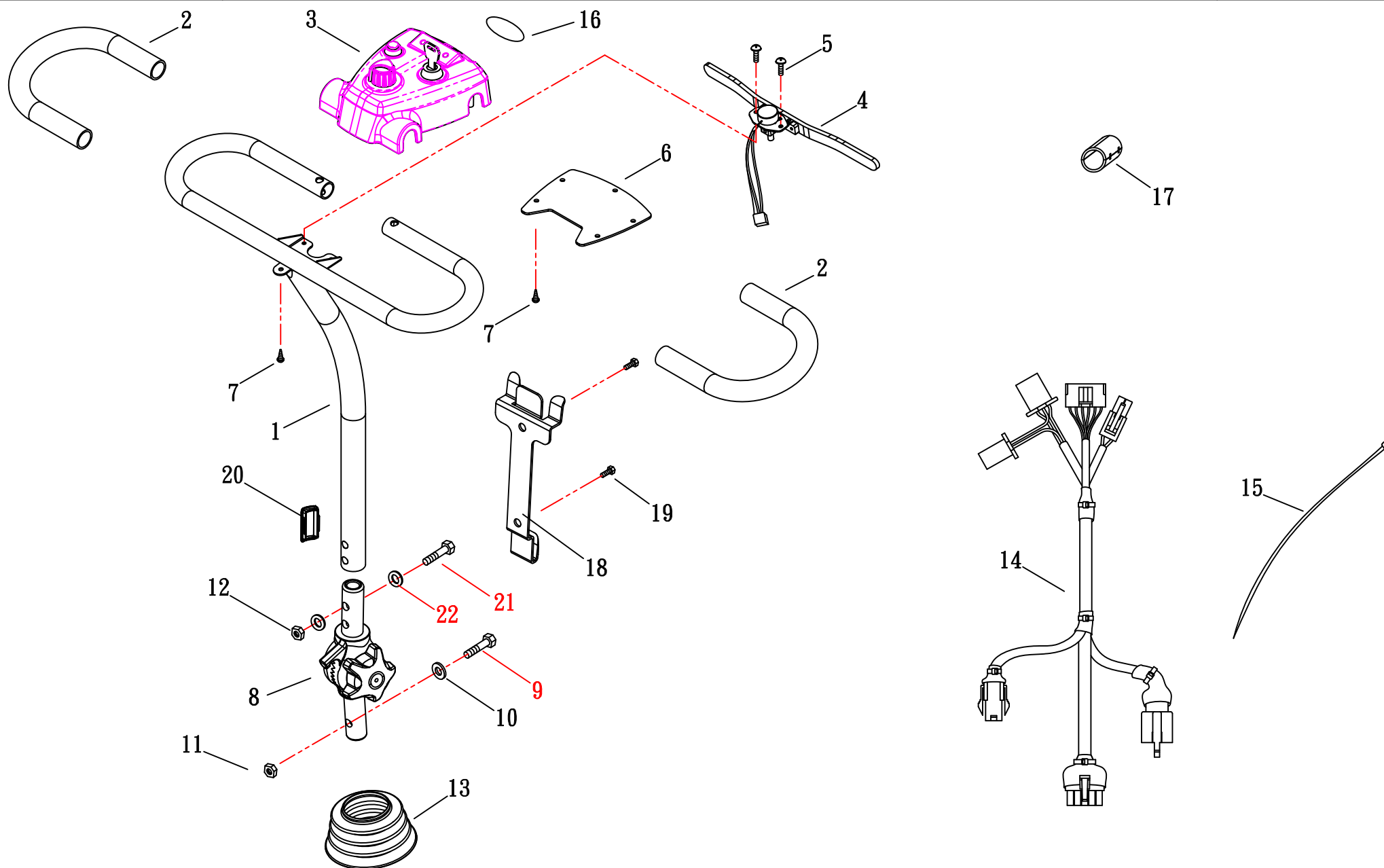




(FIG. 1-1) STEERING

HS-118A

修訂日期:2023.02.08

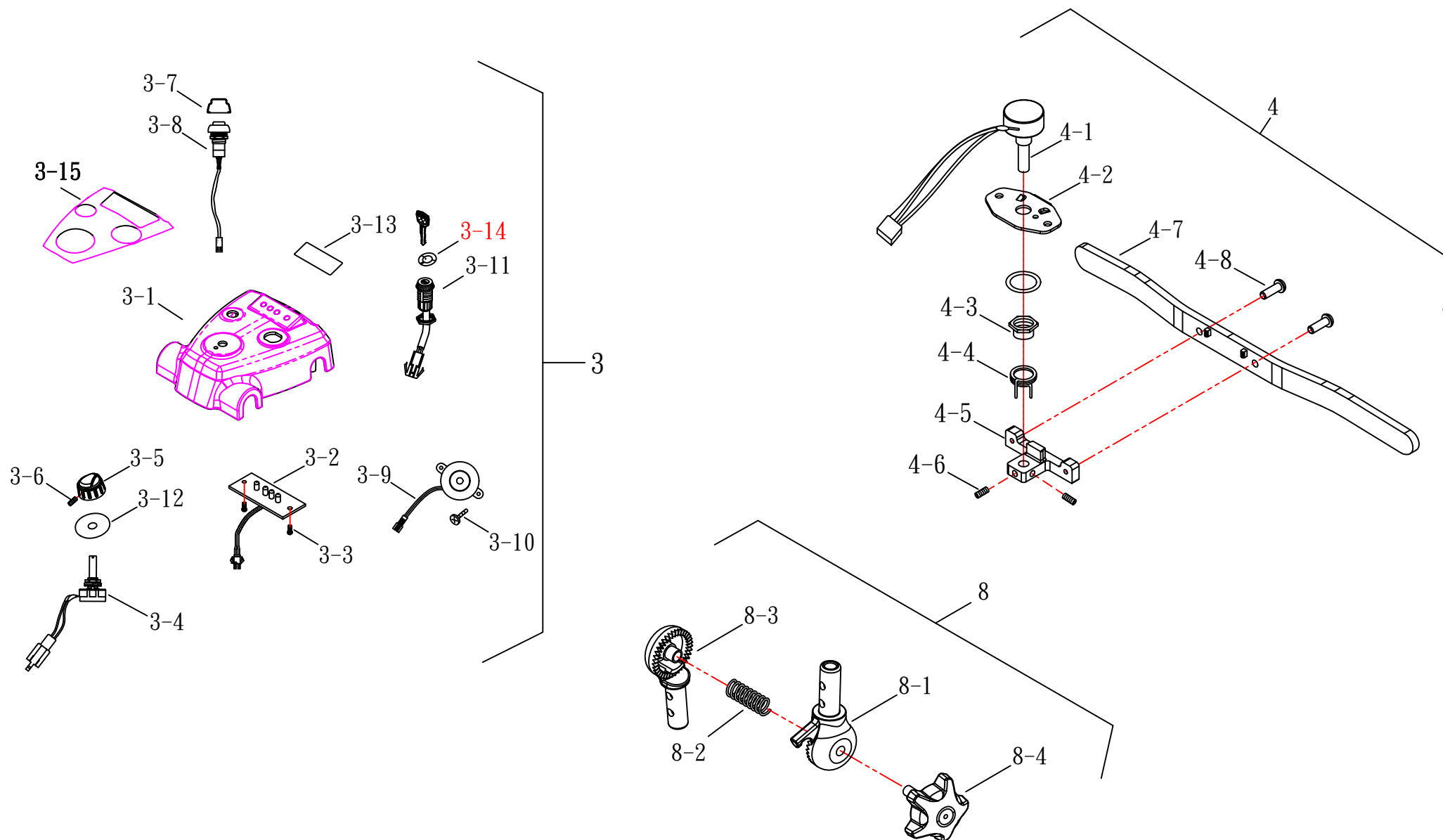




(FIG. 2) FRONT FRAME

HS-118A

修訂日期:2024. 01. 15

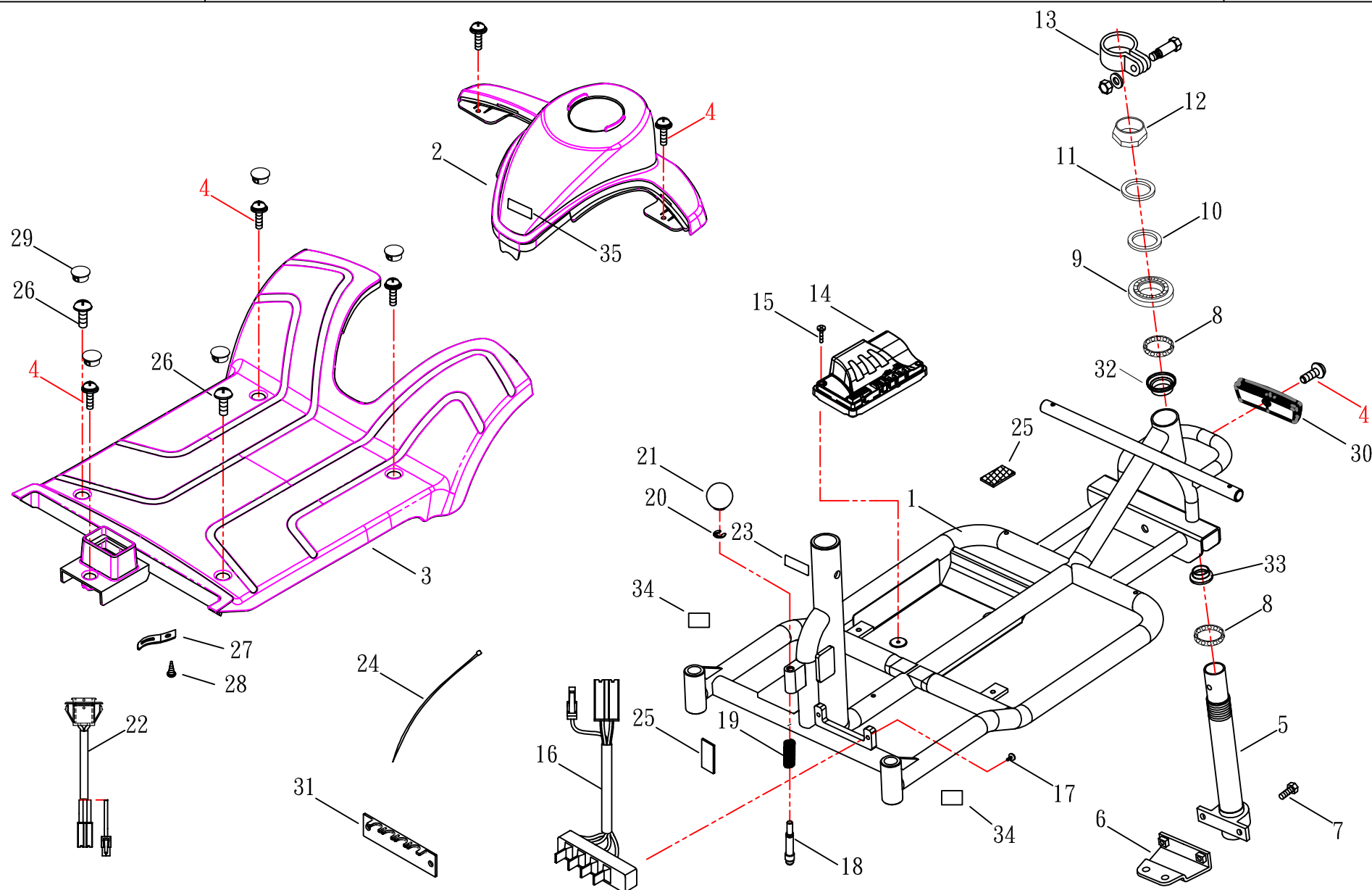


CT-RD-R-0022



(FIG. 3) REAR FRAME

HS-118A

修訂日期:2024.12.13

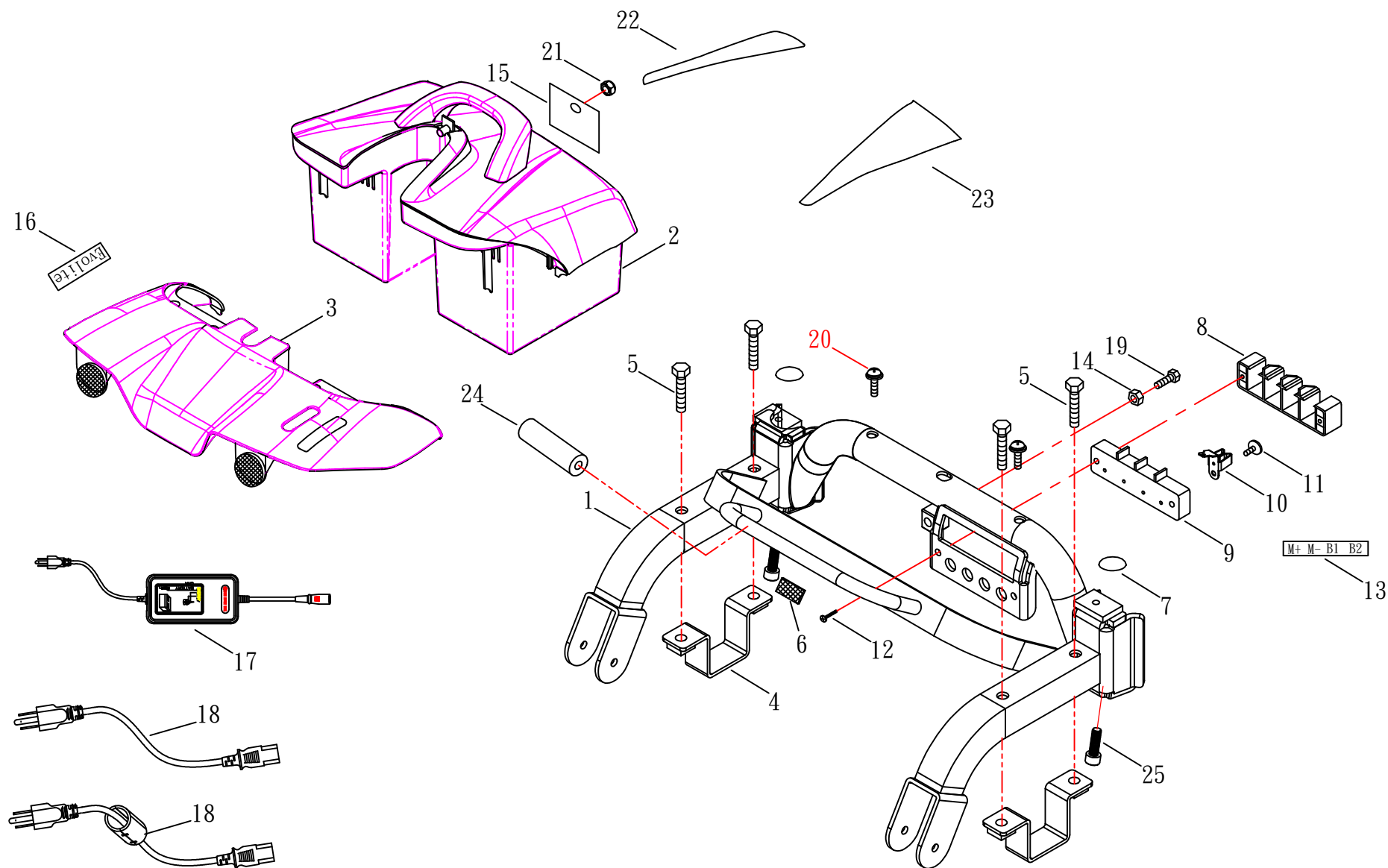




(FIG. 4) SEAT

HS-118A(CTM)

修訂日期:2024.05.07

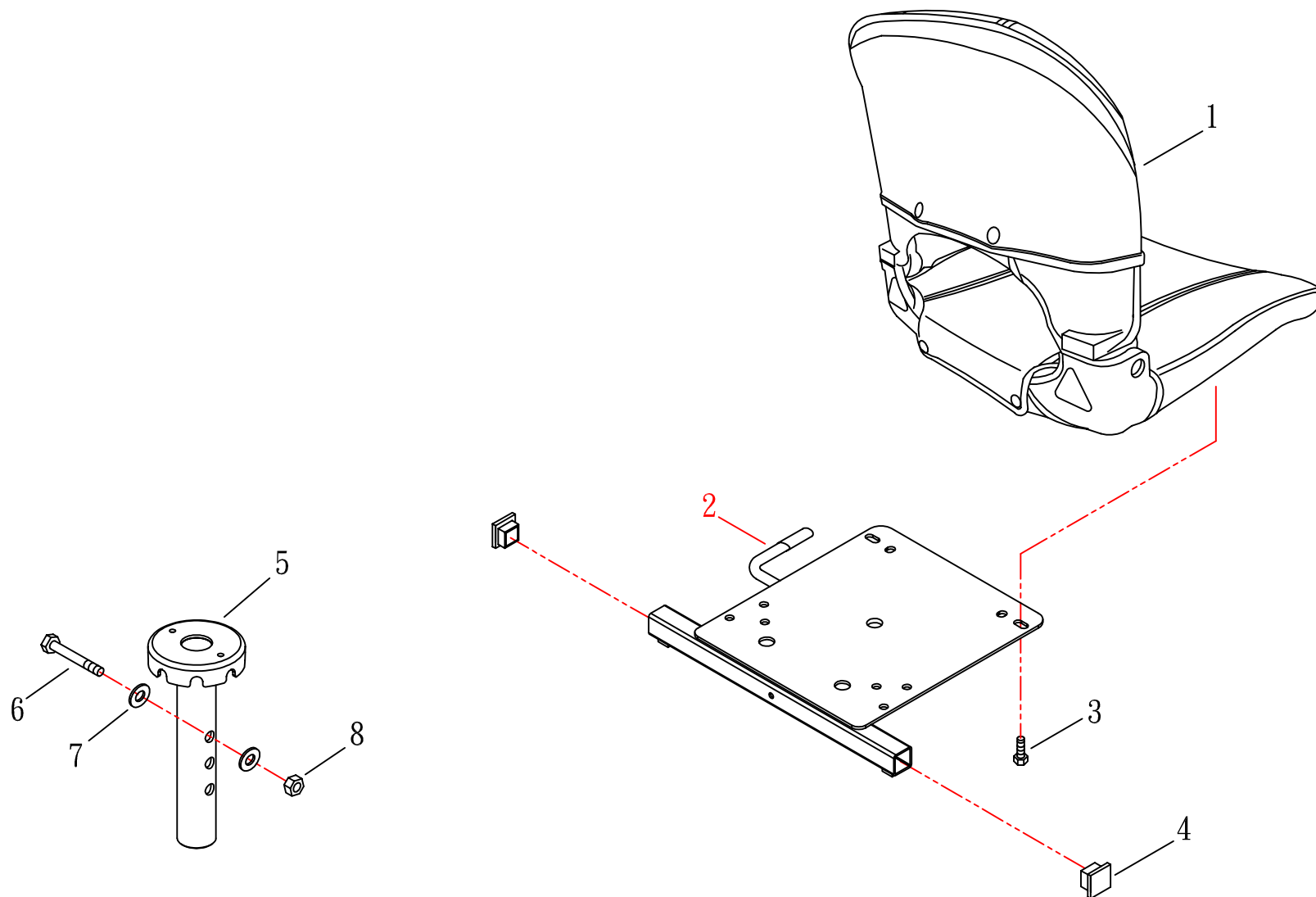




(FIG. 5) FRONT WHEEL

HS-118A

修訂日期:

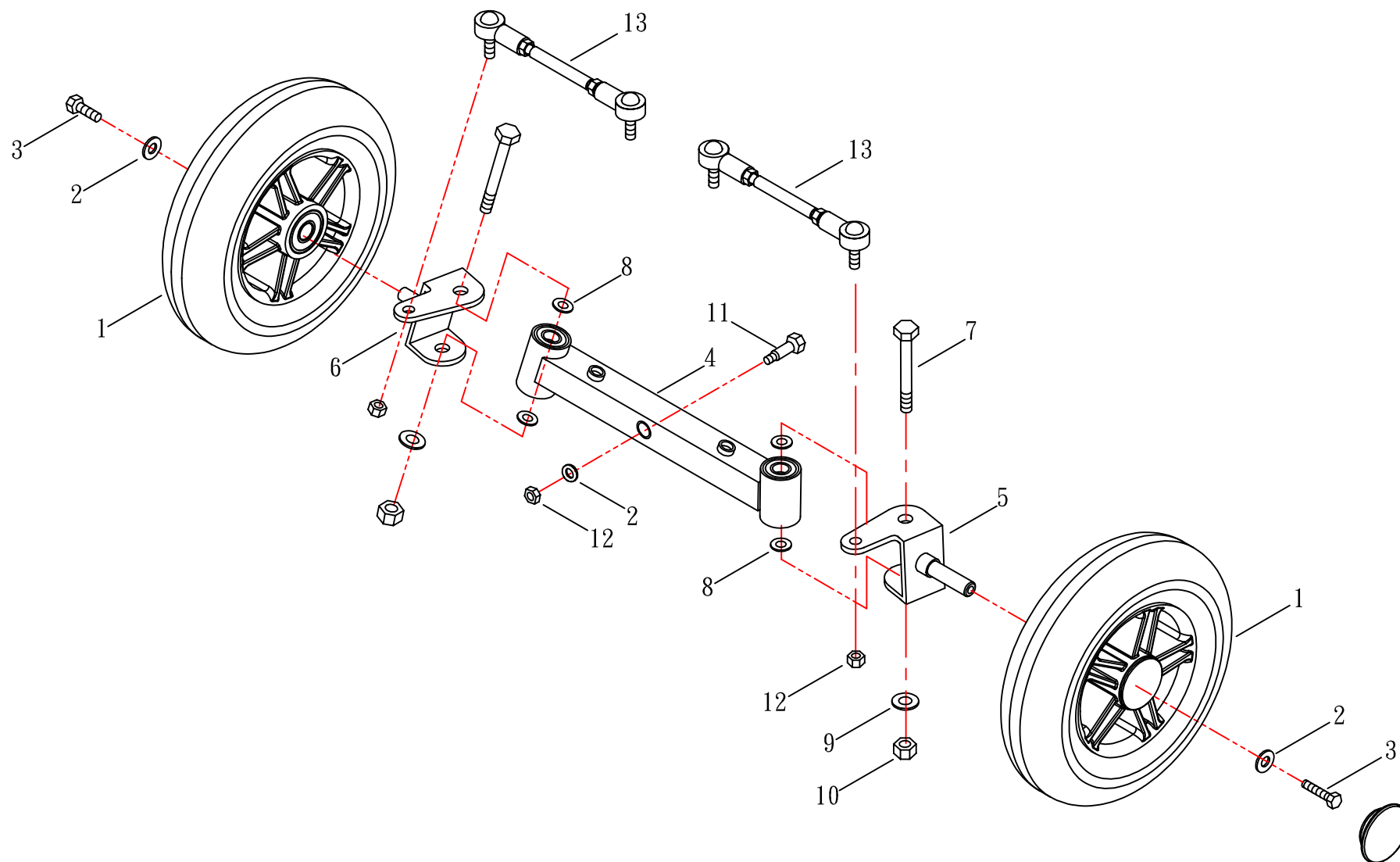




(FIG. 6) REAR WHEEL & MOTOR ASSEMBLY

HS-118、HS-118A、
HS-118A(UK18)

修訂日期:

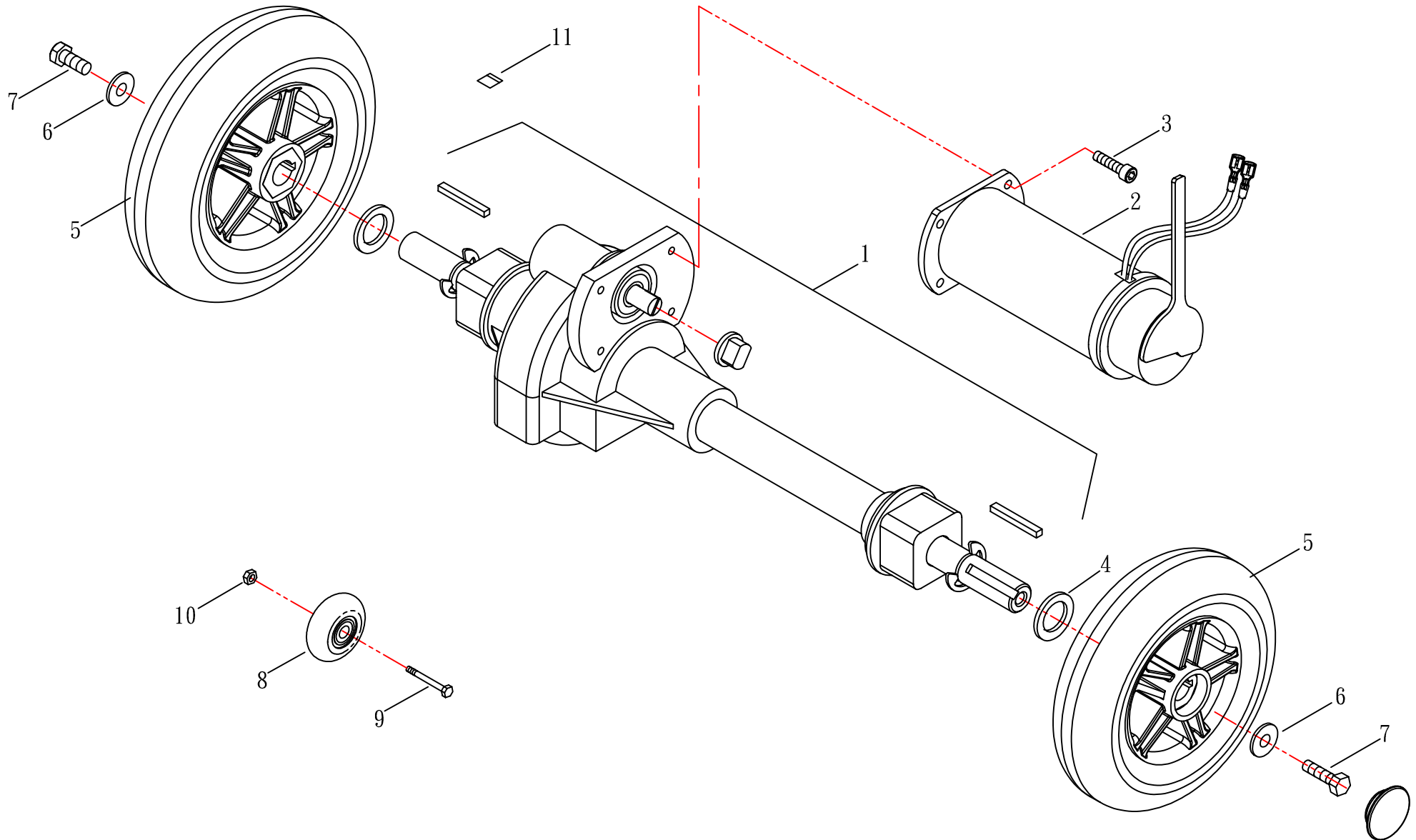




(FIG. 6-1) REAR WHEEL & MOTOR ASSEMBLY

HS-118、HS-118A、
HS-118A(UK18)

修訂日期:

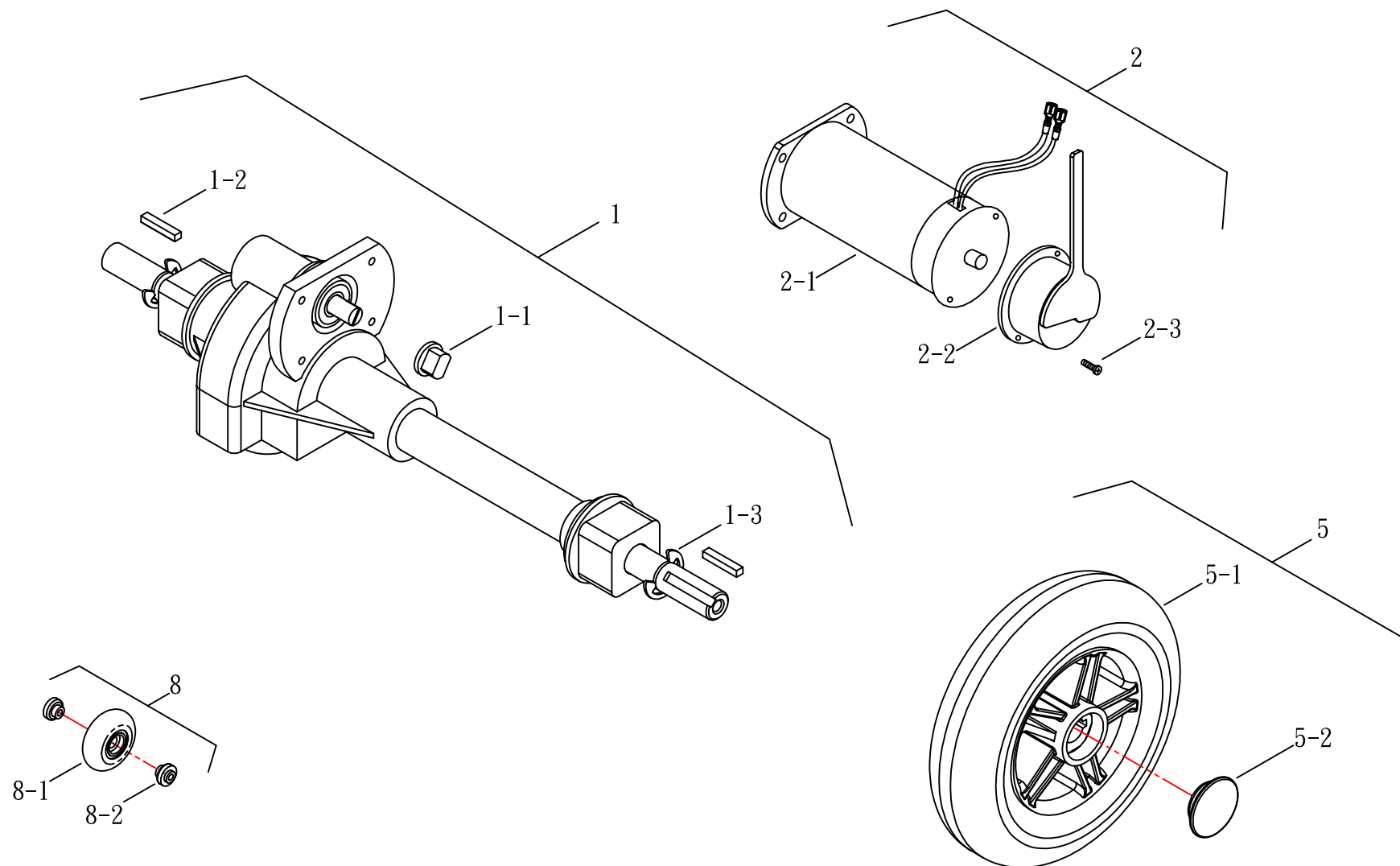



(FIG. 7) ACCESSORY

HS-118A

修訂日期:2023.07.04

