


Complete Swing-away Joystick Mount w/ Hardware				Mounting Hardware		
Ref	Part Number	Mounting Hardware	Seating Type	NE / NE+	Q-Logic	Contoured Seat Armrest Adapter
A1	ACC1908434	Q-Logic	Rehab			
B1	ACC1908435	Q-Logic	Contoured			
C1	ACC1908437	NE/NE+	Rehab			
D1	ACC1908438	NE/NE+	Contoured			
E1	ACC1908439	PG Drives*	Rehab			
F1	ACC1908440	PG Drives*	Contoured			

Please Note:


- The complete assemblies A1 thru F1 include the swing-away joystick mount, the applicable joystick mounting hardware, and adjustment instructions.
- The assemblies used with contoured seating also include the armrest adapter K1.
- PG Drives does **NOT** include Remote Plus.

Replacement Belt

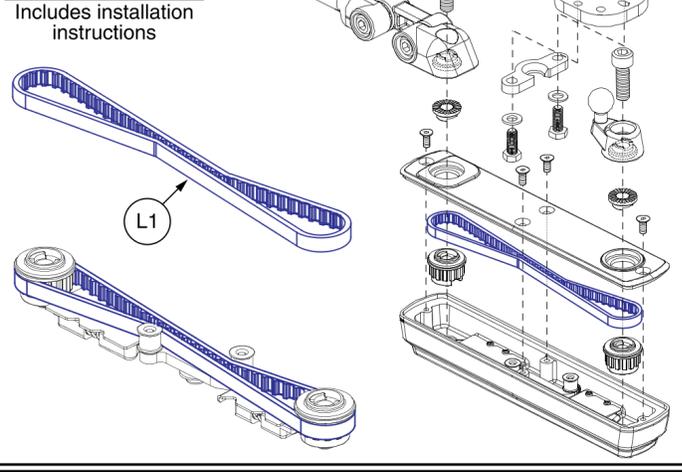


Adjustments

Reference Only. See the include instructions before making any changes.

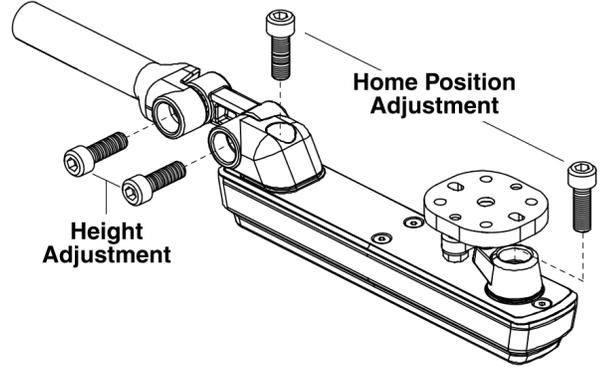


- Each pivot can be adjusted independently in 15 Degree increments to allow for a wide range of positions and angles.
- Two examples are shown. One for a possible home position and one for a possible height position.

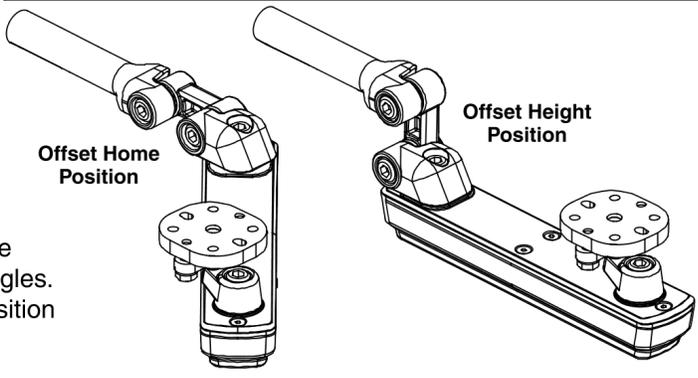

Swing-away Force Adjustment and Belt Tightening

Three set screws allow for adjustments to be made to the swing-away force and belt tension of the joystick mount. See notes on right for further details.

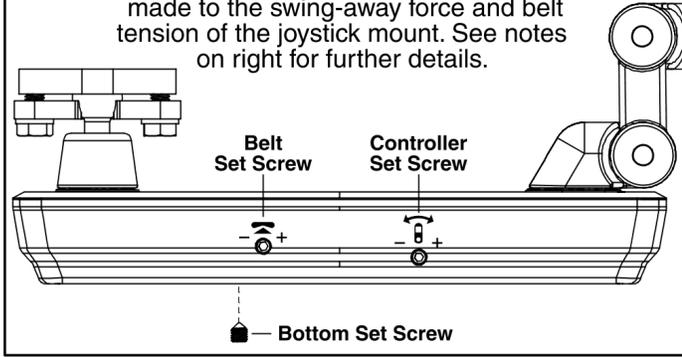

Set Screw	Notes
 Controller	<ul style="list-style-type: none"> - Adjusts the force needed to swing the joystick mount away. Force range depends on where the Bottom Set Screw is positioned. - Sets the force range of the Controller Set Screw. Removed sets the range from 3 to 6lbs. Inserted sets the range from 5 to 8lbs.
 Bottom	<ul style="list-style-type: none"> - Adjusts the internal belt tension. Turn inward 1/4 turn if the controller does not maintain the orientation when swinging away or if the controller spins. - Note: This ONLY needs to be adjusted if the belt is loose and should be performed rarely.
 Belt	

Item	Part No.	Name	Qty
A1	ACC1908434	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,Q-LOGIC	1
B1	ACC1908435	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,Q-LOGIC CONTOURED SEAT	1
C1	ACC1908437	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,NE / NE PLUS	1
D1	ACC1908438	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,NE / NE PLUS CONTOURED SEAT	1
E1	ACC1908439	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,VSI / VR2 / GC3 / R-NET	1
F1	ACC1908440	JOYSTICK MOUNTING BRACKET,ASSY,SWING AWAY,VSI / VR2 / GC3 / R-NET CONTOURED SEAT	1
G1	DWR1111C002	SPACER,CURTIS NE/NE PLUS MOUNT BRACKET,C-1111-002	2
H1	SCRPPH1384	SCREW, PAN HEAD, TYPE H, M5X0.8 X 20, ISO 7045, CL 4.8, BZ	4
I1	SCRPPH1279	SCREW, PAN HEAD, TYPE H, M5X0.8 X 16, ISO 7045, CL 4.8, BZ	4
J1	SCRPPH1384	SCREW, PAN HEAD, TYPE H, M5X0.8 X 20, ISO 7045, CL 4.8, BZ	2
K1	MEC146133	ARMREST ADAPTER ASSEMBLY	1
L1	KIT2103246	TIMING BELT SERVICE ASSEMBLY, SWING AWAY JOYSTICK MOUNT	1