



**ALLIED
MEDICAL**
LIMITED

USER MANUAL

Shower Stool With Handles



NRSF20524



USER INSTRUCTIONS

SHOWER STOOL WITH HANDLES

General Safety Warning

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

Maximum User Weight



1. INTENDED USE

For use in a bathroom or shower, to provide a stable seated surface for users with reduced mobility who are unable to stand for long periods.



2. SAFETY

Warning: The maximum user weight must not be more than 160 kg (25 st).

Do not attempt to stand or climb on the product.

Ensure all legs are set at equal height to avoid rocking and instability.

3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try and assemble.

4. ASSEMBLY/INSTALLATION

Product arrives fully assembled.

4.1 HEIGHT ADJUSTMENT

Remove the 'E-clip' and slide the leg to the desired position.

Align the holes and re-insert the 'E-clip'.

Repeat for all four legs.

Note: Ensure the floor surface is level and even to avoid rocking.

5. MAINTENANCE

Check that all 'E-clips' are inserted correctly and that there is no excessive movement.

Inspect the product regularly for signs of wear or damage, pay particular attention to the condition of the ferrules.

Any suspected faults or damage should immediately be reported to the supplier and the product taken out of use.

6. CLEANING INSTRUCTIONS

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution and rinse thoroughly with fresh water.

Allow the product to dry fully prior to use/storage.

7. PRODUCT DISPOSAL

Thoroughly clean the product before disposal.

Product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through local Trade Waste Collectors.

8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 24 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover abuse, interference or tampering with any part of the products. Issues arising from fair wear & tear and customer misuse are not covered.

This guarantee covers domestic use only. If the product is used in commercial or semi-commercial settings such as care homes, schools, rental or communal accommodation etc, the guarantee is valid for one year only. Consumable parts are also guaranteed for one year only.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.