

SKI PAD^{CE}

(Part of the “SKI” range of products)


The Ski Pad is designed for use in places where space is limited, such as congested hospital wards and buildings with narrow corridors or single width fire doors.

Features:

- Durable Vinyl coated base material enables the Ski Pad to move over virtually any interior and exterior surface.
- 5cm foam pad provides protection against bumps and bruises.
- Seat belt style buckles and webbing
- Foot Pocket
- Wall mountable, wipe clean bag

Using the SKI PAD:

- In an Emergency the Ski Pad is quickly removed from its wall mountable storage bag.
- The patient can then be transferred on to the Ski Pad (this can be performed either from bed/floor/chair).
- Place feet within the foot pocket.
- If pillows are available place these on the individuals leg and chest area (for comfort once strapped in).
- Secure the individual in place on the pad using the seat belt style cross straps.
- Using the end pulling handles the individual can be moved to a safe area. When going down stairs it is important to pull the individual feet first.

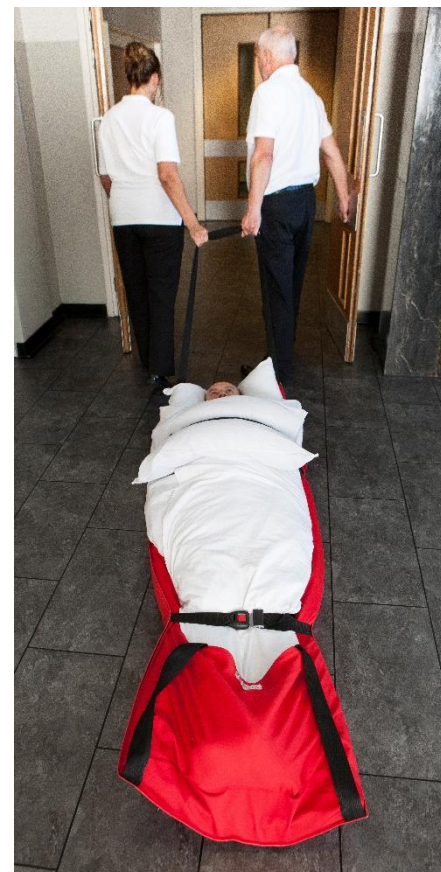

Specifications:

Packed: 67cm x 64cm x 20cm (2 or 3 strap)

Unrolled: 197cm x 61cm

Padding: 5cm thick Weight: 5kg

Safety tested to: 160kg

Safe working load: 120kg (for two people)

Product codes:

Two Strap: SKIP001

Three Strap: SKIP002



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

1 Kingsthorpe Business Centre, 63 Studland Road, Kingsthorpe, Northampton, NN2 6NE, UK.
Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553

