

SKI SHEET ^{CE}

(Part of the “SKI” range of products)



The SKI SHEET is designed for mattress evacuation, of non-ambulant patients, from places such as hospital wards in the event of a fire or other emergency. Permanently and unobtrusively fixed to the underside of the patient's mattress, the Ski Sheet is instantly accessible when needed

Features

- Easy accessible pockets for the storage of the cross straps.
- Elastic corner straps for attaching securely to the underside of the mattress.
- Pulling handles at each end.
- Seat belt style metal buckles and webbing.

Attaching the Ski Sheet:

- Simply lay the Ski Sheet on the base of the bed and place the mattress on the top.
- Concertina the belts and store within the sheets easy to access pockets.
- Attach the sheet to the mattress using the sheets elastic corner loops.
- Tuck in the end pulling handles at the head and foot end

THERE IT STAYS - READY FOR IMMEDIATE USE!



Using the Ski Sheet:

- Place pillows on the patient's chest and legs for maximum patient comfort and security once secured.
- Pull the belts from the storage pockets (located next to the corner elastics).
- Fasten the buckles over the patient.
- Tighten the straps as tight as possible creating a U-shape.
- Slide the mattress off the bed.
- Move to a safe area (even down stairs) without the need for heavy lifting – let gravity do the work!



www.spectrumhealthcare.co.uk

Email: sales@spectrumhealthcare.co.uk

Telephone: +44 (0)1793 762928

1 Kingsthorpe Business Centre, 63 Studland Road, Kingsthorpe, Northampton, NN2 6NE, UK.
Spectrum Healthcare (UK) Ltd - Registered in England No. 06197553





Laundering Instructions

MACHINE WASHABLE

DO NOT BLEACH

DRIP DRY or TUMBLE DRY ON A LOW HEAT

Washing: When laundering does become necessary the buckles on the belts should be fastened, and a temperature of up to 80°C can be used.

To protect the buckles it is recommended that the sheets should be washed in a bag.

Thorough rinsing will ensure that no detergent deposit is left.

A disinfectant may be added to the final rinse if required.

Drying: A low temperature is recommended to prevent the nylon material from becoming brittle and shrinking.



Specifications:

Packed: 27cm x 10cm x 13cm

Unrolled: 196cm x 85cm

Safety tested to: 160kg

Safe working load: 120kg – for two people

Material: Nylon

Weight: 1kg

Product codes:

SKISH01 (Standard)

SKISH02 (modified with a positioning handle situated at each pocket – As picture)

**ONSITE DEMONSTRATIONS AND TRAINING PACKAGES ARE AVAILABLE
CONTACT US FOR MORE INFORMATION**