

ALUMINIUM TRI-WALKER

User's Manual



Impo International Ltd.
Braitrim House,
98 Victoria road, London,
UK. NW10 6NB

Aluminium Tri-Walker

Please read carefully and
retain for future reference



The Aluminium Tri-Walker is suitable for everyday indoor use. With a sturdy with frame, it is portable and offers stability and security to the user needing assistance mobility.

All Aluminium Tri-Walkers are manufactured to the highest standards and carry the CE mark. This product is classified as a Class 1 Medical Device.

Features

- ① Folding mechanism
- ② Tightening Handles
- ③ Removable Tray/Basket
- ④ Brake levers and locks
- ⑤ Removable Bag
- ⑥ Cable Brakes
- ⑦ 19cm Solid caster wheels
- ⑧ Front caster wheel



Folding mechanism
(in lock position)



The Aluminium Tri-Walker is guaranteed for a period of 1 year from the date of purchase or delivery. The warranty is not transferable.

Please contact your dealer if your Tri-Walker becomes defective or is not operating properly during the warranty period.

The manufacturer will not accept liability for damage caused by any misuse, unauthorised modifications or non-observance of the instructions set out in this manual.

The guarantee excludes parts which are subject to normal wear and tear during this period including the wheels, caster and plastic parts.

CLASS 1 MEDICAL DEVICE

MODEL	
NUMBER OF WHEELS	3
HANDLE HEIGHT	82 - 97cm
OVERALL WIDTH	65cm
OVERALL DEPTH	60cm
FOLDED DEPTH	24cm
UNIT WEIGHT	7.2kg
MAXIMUM USER WEIGHT	110kg
WHEELS DIAMETER	19cm

This product is designed to assist a user with limited mobility to walk and carry small items on flat surfaces. The maximum user weight is stated on page 6 of this manual in the specifications section.

The intended user should possess the necessary physical, visual and cognitive skills to operate the Tri-Walker safely. If you have any doubt, please consult your dealer/prescriber.

It is important that the user reads and understands the safety advice given within this manual before attempting to operate the product to avoid possible injury to the user or bystanders and avoid damage to the Tri-Walker or surrounding property.

Safety Checklist

DO:

- Read this manual carefully before using your Tri-Walker.
- Ensure the brakes are in the locked position if leaving the Tri-Walker unattended.
- Check the caster and wheels are secure before use.
- Check the brakes are functioning properly.
- Check the handlebar height tightening handles are secure before use.
- Check the Tri-walker is fully unfolded and the folding mechanism is locked before use
- Take care when carrying the folded Tri-Walker. Keep hands away from the folding mechanism to prevent the risk of pinching.

DO NOT:

- Use the device other than for its intended purpose.
- Sit on the Tri-Walker.
- Hang objects on the brake levers or hand grips. This could affect brake operation and/or stability of the Tri-Walker


Assembly

1. Remove all packaging, ensuring that the wheels are free from the protective wrapping
2. Insert the handlebars into the upright tubes of the main frame and adjust to the required height (see section 'Adjusting the Handlebars' on page 4). Ensure that the brake cables extend to the wheel on the same side as each handlebar.
3. Insert the tightening handles and turn in a clockwise direction until secure. Ensure that the tightening handles are on the outside of the Tri-Walker.

fig. 1



4. Check that the brakes are functioning correctly before use.
5. Fully open the Tri-Walker using the handlebars and ensure the folding mechanism is in the lock position.
6. Place the basket onto the front of the rollator ensuring it is properly secured with both hooks and fit the tray on top.
7. Use the press studs on the bag to secure it to the four tubes of the Tri-Walker.

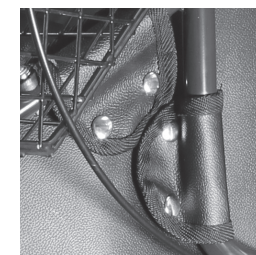

To adjust the height of the handlebars, unscrew the tightening handle in an anticlockwise direction and remove. Adjust the height of the handlebars to suit the user, then reinsert tightening handles.

Tighten by turning the tightening handles in a clockwise direction until secure. Ensure that the tightening handles are on the outside of the Tri-Walker.



To engage the brakes squeeze the brake levers upwards towards the handles.



To lock the brakes, push the brake levers in a downward direction until a click is heard. To release, pull the brake levers in an upward direction until a click is heard.

- For your health and comfort, make sure that you adopt a comfortable posture and that you can operate the braking mechanism with ease.
- Ensure that the handlebars are at equal heights and securely locked in position.
- Ensure that the Tri-Walker is fully folded out and the folding mechanism is in the locked position before use.
- Ensure that all three wheels are stable and are on the ground before use.
- Ensure the braking mechanism is adequate and test before using.
- Ensure that the latest health and safety practice is used for carrying and lifting the product.

Care and Maintenance

- Ensure the Tri-Walker is used only as an aid for mobility.
- Check the moving parts (e.g. wheels, brakes & height adjustment) regularly.
- Regularly test the brakes to ensure they are functioning properly and are properly adjusted.
- Regularly check that all fastenings (screws, nuts and bolts) are securely tightened.
- Regularly check that the front caster wheel swivels freely. Do not use the Tri-Walker if it does not.
- Regularly check that the folding mechanism is functioning properly and is in the locked position when the Tri-Walker is in use.
- Only use an approved dealer to carry out any maintenance or repairs on the Tri-Walker.
- Clean with a damp cloth or sponge and warm soapy water. Domestic anti-bacterial surface cleaner may be used if required.

Note: Care should be taken not to leave any cleaning residue on the hand grips which may leave them slippery when used.

Do not use scouring pads or scrapers as this may damage the surface coatings of the product

Folding and Storing

1. Remove the tray and basket and ensure the bag is empty.
2. Pull the black lever in the direction shown to unlock the folding mechanism.
3. Continue to fold the rollator using the handlebars.
4. To allow the Tri-Walker to stand upright during storage, leave a gap of at least 5cm between the rear wheels when folding.



Aluminium Tri-Walker