

# User Manual



## 4-Wheel Rollator

### Important Note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and serviced on a regular basis. This manual reflects the latest product developments. Hebei Healthplus Medical Device Co., LTD has the right to implement changes to this type of product without any obligation to adapt or replace similar products previously delivered.

The warranty on this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.



## Parts Description



## Specification

	R726	R728
DIMENSIONS	624x615x(777-932)mm	640x620x(830-980)mm
HEIGHT ADJ.	777-932mm	830-980mm
SEAT SIZE	357x305x35mm	360x310x35mm
PRODUCT WEIGHT	5.8kg	6.3kg
CASTOR SIZE	6"	7.2"
WEIGHT CAPACITY	135kg	135kg

## Specification

### Step 1

Remove the Rollator and all components from the packaging. Lay out the Rollator.

### Step 2

To unfold the Rollator, lift using the lifting strap which is located under the seat. Lower the Rollator down onto its wheels and push down on the central bar to fully open.



### Important!

Always ensure the side braces are fully down before using.

### Step 3

Insert the handlebar into the tubing ensuring the brake cable is to the outside of the Rollator frame. Select the required height and secure with the tightening handle supplied. Turn lock wise to tighten and anticlockwise to unscrew. Repeat for the other handlebar.



### Step 4

Remove all packaging from the backrest. Position the backrest ends into both brackets, push in both backrest 'buttons' and fully insert into the brackets. Ensure the 'pins' are fully located in the holes. To remove the backrest, pull both 'buttons' out and remove the backrest from the brackets.





## Adjusting Handlebar Height



To adjust the height of the handlebars, unscrew the tightening handles on either side of the Rollator turning anti-clockwise and remove. Adjust the height of the handlebars to suit the user, re-insert the tightening handles and tighten by turning clockwise to secure (as shown in the picture). Ensure that the tightening handles and brake cables are on the outside of the Rollator.

## Brake Operation

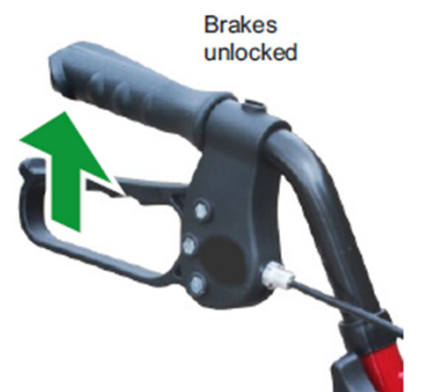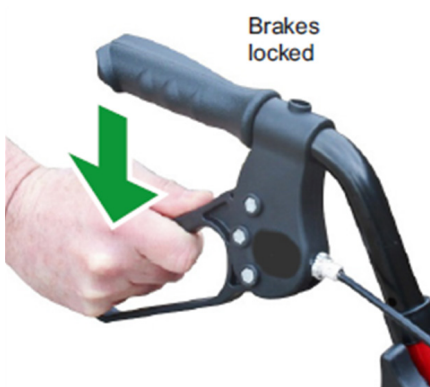

### To operate the brakes:

Squeeze the brake levers up and hold in position to engage the brakes (as shown by the green arrow). Release the levers to release the brakes.

### To lock the brakes:

Push down on the brake lever until it locks and stays in down position (as shown by the green arrow).

### Important:

Ensure that the brakes are locked on when sitting on the seat.

### To unlock the brakes:

Pull up on the brake lever until the lock removes (as shown by the green arrow).

## Folding and Transporting



### To fold the Rollator for transporting:

Whilst holding the Rollator handle, lift-up the seat and lift the Rollator using the lifting strap. As you lift off the ground, the Rollator will fold.

#### Note:

It is recommended that the brakes are locked when folding and storing.

### To unfold the Rollator:

Lift the Rollator using the lifting strap which is located under the seat. Lower the Rollator down onto its wheels and push down on the central bar to fully open.

## Care and Maintenance

- Ensure the Rollator is used only as an aid for mobility.
- Ensure the Rollator is kept clean. Wipe down the Rollator with a damp cloth on a regular basis. Do not use any abrasive detergents. Solvents can damage the coating and surface of plastic parts.
- Check the moving parts (e.g. – wheels and height adjustment) regularly.
- All the fasteners (screws, nuts, and bolts) are checked regularly to ensure they are securely tightened.
- Check that the front castor wheel/s swivel freely. Do not use the Rollator if they do not.
- Only use an approved dealer for any maintenance and repairs on your Rollator.



Phone: 0800 31 61 81

Fax: 09 415 1686



Email: [helpis@alliedmedical.co.nz](mailto:helpis@alliedmedical.co.nz)

[www.alliedmedical.co.nz](http://www.alliedmedical.co.nz)



**ALLIED  
MEDICAL**  
LIMITED