

Assembly Instructions



AML20107 - Height Adjustable Shower Stool with Arms

Check you have all the parts:

1x	Seat frame
4x	Legs (3 x 6 holes) (1 x 12 holes – used to change the angle of the seat)
1x	Seat
2x	Screws
4x	Plastic washers

Tools required:

1x	Phillips screwdriver
----	----------------------

Instructions

1. Turn seat frame upside down (to make installation of legs easier).
2. Place legs on the chair, pushing the button down to slide and lock into place, repeat for all, the height can be adjusted later for legs.
3. Place frame on its side.
4. Using 1 x screw and 1 x washer slide into the frame, repeat for the other screw.
5. Place another washer on top of the screw and line up to the seat.
6. Use the screwdriver to tighten the screws.
7. Completed Assembly.

Guidance for Use

Turn upright, to adjust push the buttons on the legs and slide up and down to suit.

Maximum Weight Capacity: 113kg

