



**ALLIED
MEDICAL**
LIMITED

USER MANUAL

Corner Shower Stool

AMLCSS



Intended use of the product

The triangular shower stool is a medical device which intended use is to give stable and easy seating support when people with functional limitations use the shower.

Indications

Bath benches and seats are overlays for regular bathtubs, indicated for use as a seating device by people with functional limitations, persons in wheelchairs and other people with limited mobility of the lumbar spine, knees, or hips, and for all low and standing back up from bathtub's bottom while performing everyday personal hygiene. Use indicated above is prohibited.

Contraindications

Product is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The device must not be used if the user has problems with impaired perception.

Areas of use

The product can be used in home care, health and long-term care facilities, and hospitals.

Medical incidents and unwanted events

In case of any unwanted events related to the product, please contact your retailer or Peak Care

Safety Requirements

- Make sure that this manual has been read by all persons using the device
- The manufacturer assumes no responsibility for damage and/or injury resulting from failure to follow the operating instructions
- Only use the product in flawless condition
- If defects or errors are detected, you must immediately contact the retailer
- Follow all manufacturer's recommendations and warnings
- Be aware of the information on the product label
- Only use the product for the described purpose
- Don't stand on the device
- Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications
- Make sure legs are the same height
- Do not exceed maximum weight capacity of 120kg
- Use the device on a flat, smooth surface
- Always check the stability of the device before use

Parts supplied

Each product package should contain: 1 triangular shower stool top, 3 height-adjustable legs, 1 user manual.

Preliminary check

Check the contents of the box against the above list. If at this stage you discover that a part is missing or damaged, contact your retailer immediately.

Product Description

The triangular shower stool provides a stable and easy seating support for use in the shower. It has an aluminium frame with height-adjustable legs and a triangular seat made of white plastic.

Height adjustable legs

To adjust the height of the stool, press the pin-button on the stool leg and slide it to desired setting until the button pops into the corresponding hole. Repeat for other two legs.

WARNING! Make sure all three legs are set to the same height.

Assembly

To assemble the stool, insert a leg into the base of the stool until the pin-button pops into a hole, and repeat for the remaining 2 legs.

Technical Data

	Item No. 302069
Dimensions (Width x Height)	43.7 - 45.8cm x 46.6 - 56.6cm
Weight	2.155kg
Max. User Weight	120kg

Care & Maintenance

Normal dirt can be removed with standard cleaning products and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

Storage: We recommend storage in a dry place at a temperature above 0, away from intense sunlight.

Transportation

The device should be transported in a stabilised and secured way with no risk of sudden or dangerous movements during transport.

Most common problems and solutions

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
The shower stool is not stable	The legs are not inserted correctly into the upper tubes, or are adjusted to different heights	Make sure the legs are inserted into the tubes correctly and/or adjust them to the same height

Guarantee

Peak Care offers a 12 month warranty for damages caused by errors in production or material errors. The warranty is not valid but non-intended repair or usage of the product. Parts subject to normal wear and tear are not covered by the warranty, unless wear caused by a manufacturing fault. In the event of product damages covered by the warranty, please notify your retailer directly. The warranty does not cover transport costs and does not include compensation for personal injuries for non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.