Dictus Band

An orthopedic aid for people
with Foot Drop
Art. No. DS-02

Dear Customer,

Thank you for buying the Dictus Band orthopedic aid. We hope that it will work to your satisfaction. To avoid unnecessary problems and trouble, please read the instructions below carefully before using this orthopedic aid.

Fig. 1

INSTRUCTION

Use

The Dictus Band, an orthopedic aid, is mainly intended for those suffering from foot drop. The active part is a latex elastic band, which keeps the front of the foot in an upward position. (Fig. 1)

Parts

The Dictus Band DS-02 includes: (Fig. 2)

- Ankle Strap, consisting of two layers of leather, closed and tightened with hook and loop straps
- Nylon D Ring
- Latex Elastic Band
- Stainless Steel Hooks
- Mono Hook

Fig. 2

Mounting

The ankle strap is placed on the top of the user's sock (not directly on the skin). The opening and tightening function is managed with the two hook and loop straps. The latex elastic band is looped through the D ring and onto the hooks, which are attached through the shoelace holes (Fig. 3). The distance between the D ring and the hooks determines the lifting ability of the latex elastic band. Position hooks into the shoelace holes to achieve the best performance.

Fig. 3

Alternate Mono Hook attachment

method (Fig. 4)

The mono hook replaces the stainless steel hooks used above and are inserted between the laces and the tongue of the shoe. The plastic hook is positioned along the lacing path. Slip the hook up through the laces at a point that best provides the desired performance. The hook opening should be facing the toe of the shoe. The latex elastic band is simply attached to the extending hook.

Fig. 4

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Precautions and Care Advice

When using the Dictus Band, the user should periodically check that all parts are correctly mounted. The latex elastic band has a seam, which should be placed in the middle and top of the locking loop at the D ring. Any other positioning may lead to premature breakage of the latex elastic band. Do not expose the latex elastic band to prolonged direct sunlight, since this will shorten its life. The latex elastic band is subject to normal wear and tear. It should be replaced as necessary, or at least once every three months. A spare latex elastic band is supplied with your original package.

PLEASE OBSERVE

Spare latex elastic bands should be kept in a dark and cool place.

The ankle strap must be cleaned with a damp cloth only. No other cleanser is to be used.

The user should also assure that the D ring, hook and loop straps and fastening hooks are always in good condition.

WARNING!

There is a risk of falling if the latex elastic band or any other part breaks or comes loose, causing the foot to no longer stay in an upward position.

Spare Parts

Spare elastic bands and fastening hooks can be ordered from your retailer.

Warranty

The Dictus Band DS-02 comes with a 1 (one) year warranty by Erimed International KB. The warranty is granted only if the user has followed the instructions and care advice and not exposed the Dictus Band to exceptional strain.

PLEASE OBSERVE

The Dictus Band package includes small parts which should be kept away from small children.

This product is not a toy.

The Dictus Band must not be used for purposes other than those described in this product's operation instructions.

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