

# USER INSTRUCTIONS

## FREESTYLE GRAB & GO / WALKING STICK

S08847	Grab & Go - Short 816mm / 32"	S17253	Walking Stick - Short 816mm / 32"
S08859	Grab & Go - Medium 865mm / 34"	S17265	Walking Stick - Medium 865mm / 34"
S08860	Grab & Go - Long 915mm / 36"	S17277	Walking Stick - Long 915mm / 36"

### ⚠ General Safety Warning ⚠

Please make sure that you read and follow these instructions carefully. Keep these instructions in a safe place for future reference.

## 1. INTENDED USE

The Freestyle Grab & Go and Freestyle Walking Sticks have been designed to enable comfort and support while walking or manoeuvring from one place to another. The Grab & Go Walking Stick has the additional benefit of the grabber feature to enable the retrieval of light to mid-weight items, such as keys, wallets and glasses.



Freestyle Walking Stick

Freestyle Grab & Go

## 2. SAFETY

**Warning:** The maximum user weight must not be more than 135Kg (21st).

When moving be aware of slipping hazards & uneven surfaces.

Inspect the product regularly for signs of wear & damage.

Check for loose or cracked handles, worn/split/damaged ferrules, bent or damaged tubes and joints.

## 3. UNPACKING

Remove the packaging and check all parts carefully. Contact your supplier immediately if parts are missing or damaged and do not try use.

## 4. IN USE

### 4.1 Walking Stick - S17253, S17265 & S17277

Use as a regular walking aid to support you while walking or manoeuvring.

### Maximum User Weight




### 4.2 Grab & Go Walking Stick - S08847, S08859 & S08860


Use as a regular walking aid to support you while walking or manoeuvring.

#### 4.2.1 Using the Grabber & Jaw

1. Pull the handle lever to open the grabber jaw.

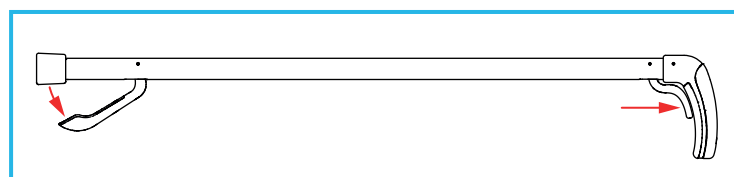

2. Place the item you want to pick up between the grabber jaw and ferrule.

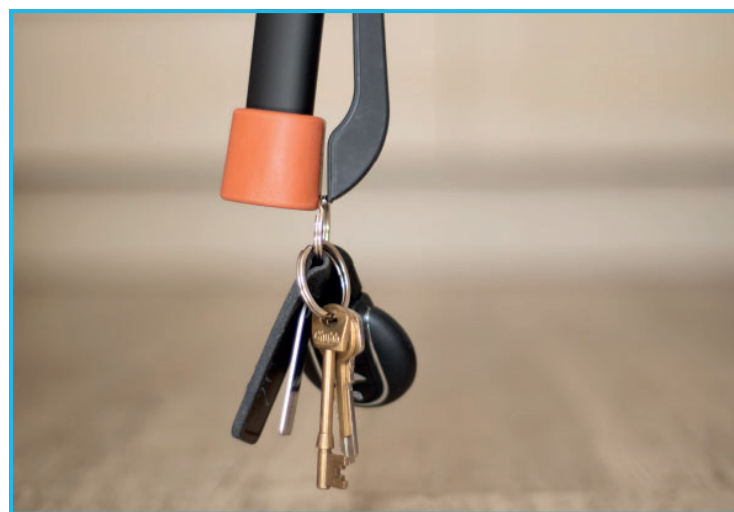

3. Release the handle lever to close the grabber jaw.
4. To remove the item, pull the handle lever to open the grabber jaw.

**Note:** For heavy items use the handle to slide the stick through your other hand to reduce the leverage.

## 5. CLEANING INSTRUCTIONS.

Clean regularly using a general purpose, neutral detergent and warm water.

Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.

Allow product to fully dry prior to use/storage.

## 6. ACCESSORIES & SPARE PARTS

S14896: Grab N Go Walking Stick - Replacement Ferrule.

For more details please visit our website or call Customer Services on the details below.

## 7. PRODUCT DISPOSAL

This product must be thoroughly cleaned/sterilised prior to disposal.

This product is manufactured from metal & plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 8. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; fair wear & tear, abuse, misuse, interference or tampering with any part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at:

[devices.regulatory@mhra.gov.uk](mailto:devices.regulatory@mhra.gov.uk)

**UK  
CA** NRS HEALTHCARE Sherwood House,  
Cartwright Way, Forest Business Park, Bardon  
Hill, Coalville, Leicestershire, LE67 1UB, UK.

**CE** NRS HEALTHCARE 4C Altona Road,  
Blaris Industrial Estate, Lisburn, Co  
Antrim, Northern Ireland BT27 5QB

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.

Please visit one of our websites to view our full range of products:

[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)