IDENTIFICATION SHEET

Final inspection:

Name:

Signature:

Identification tag / Product type (write up):

EN

DE

Warranty conditions:

1. On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
2. The warranty has to be applied in the business organization, where the product was bought, or in authorized service repair shop.
3. **Properly filled Certificate of Warranty and a copy of acquiring document have to be advanced when applying the warranty. Do not pull down the identification card. It has to be readable and undamaged for the whole warranty period.**
4. Any defect that has occurred must be repaired immediately after finding it. The user is fully responsible for defects resulting from the continued use of the product without the original defect being repaired.
5. The product has to be used entirely to the purpose, which it is determined for, the user has to follow the instructions mentioned in the instructions for use. Producer / distributor is not responsible for defects caused by incompetent handling in conflict with the instructions for use.
6. The product has to be used, treated, maintained and stored properly according to directions mentioned in the instructions for use.
7. The warranty proceedings can be initiated, if the product under claim has been sent complete in original package, carefully packed and secured against harm. The owner is obliged to advance the product under claim properly cleaned up, hygienically flawless due to common hygienic principles. The owner cannot ask the seller to do basic maintenance or cleaning of the product during claim proceedings.
8. Every defect has to be claimed immediately
9. The warranty is forfeit if the defects were caused by forbidden overloading, accident, violence or incompetent intervention into the product or incompetent handling.
10. Warranty period cannot be interchanged with using period, i.e. period, for which, when used and treated properly including maintenance, the product, in respect of its characteristics, given purpose and difference in construction, can be used properly.
11. The engineering solution does not guarantee the ideal direction trails.

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Date of sale (day, month, year): _____ Name (block letters), phone number, stamp and signature of the seller: _____

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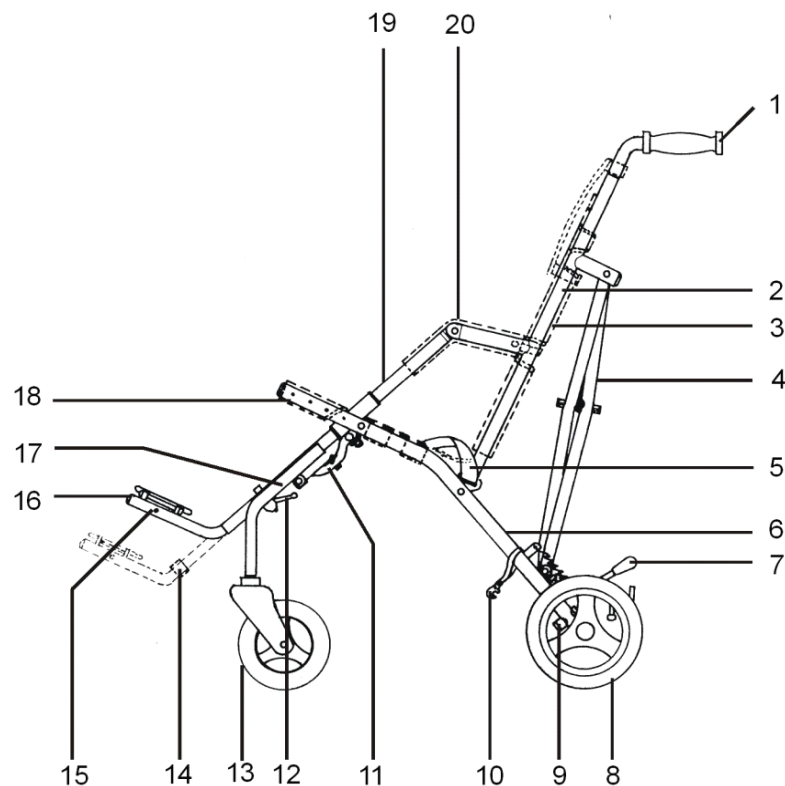
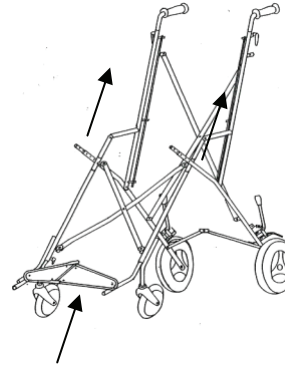
Pic. 3

Dear customers,

Rehab-Stroller CORZINO are designed for short stay of an occupant. With the design and especially unique construction are ranked among the best from their category. Prior the first usage of the buggy, please, read carefully this user guide. In case you are not fully sure about the proper usage of the product after reading of the user guide feel free to contact your local dealer.

Retain the user guide for future reference.

..... SAFETY AND WARNING NOTES:

WARNING! Keep fingers out of any moving parts when operating with the stroller.

WARNING! Never leave your child unattended! Even if the belts and brakes are in use.

WARNING! Check before use that all locking devices are properly locked and all safety equipments are properly used.

WARNING! Check before use that seat unit is properly locked in the chassis interface.

WARNING! The seat unit is not suitable for a child younger 6 months.

WARNING! This product must not be used while jogging or in-line skating.

WARNING! The seat unit must not be used in motor vehicles as the car seat.

ATTENTION! Read this manual carefully before the first use of the stroller, as well as

keep it for further reference. Safety of a child might be in danger in case you will not follow the instructions and safety notices mentioned I the user guide.

- please read this instruction booklet fully prior to use
- for safety reason it is necessary to check prior using of buggy, that rear brace (seat brace - size 38), safety levers are properly tightened and the spring button of the seat and handles quick releases are properly positioned
- expert medical advice regarding the suitability of this product must be sought before use
- the attendant should be familiar with the operation of the buggy - if in doubt, stop and read this instruction booklet
- before setting off, ensure the child is comfortable and that clothes and rugs etc. are tucked in, so they do not catch in the wheels
- always apply the parking brakes when the buggy is stationary and when getting in and out the buggy
- never tip the buggy forward at a kerb
- the buggy MUST NOT be used as a seat for a child while traveling in motor vehicle or bus
- the attendant must take precautions against fire risk. In buildings, check the location of fire exits and ensure procedures are understood.
- do not let children play with or hang on the buggy
- The stroller is not designed to be driven down the stairs.
- do not use your buggy in a manner for which it is not designed
- PATRON Bohemia a.s. cannot accept responsibility for the use of accessories other than those supplied by this firm
- during the drive keep full attention, to avoid any dangerous situation or collision
- do not allow the child to climb unassisted into and out the buggy
- always apply the parking brakes when getting in and out the buggy
- do not overload this buggy with additional children, goods or accessories other than those supplied by PATRON Bohemia a.s.
- ensure the child always wears a correctly fitted and adjusted safety belt. The attendant must know how to fit, adjust and release accessories
- do not stand on the footrest
- never leave child unattended in the buggy, even for the shortest time
- by the using and manipulation with buggy act accordingly not to cause any injuries to the occupant, yourself or any other person
- PATRON Bohemia a.s. is not responsible for any losses and injuries caused by incorrect manipulation and using of the product.

USER GUIDE

Opening the buggy - stand buggy on wheels, unlock storage clip, grasp one handle and push down on the opposite seat tube. Press down firmly on seat tubes to ensure the buggy is completely opened. Use foot to lock the rear brace into a straight position. Wrap the armrest padding around the armrests. Mount the footrest on the buggy. (Lock the seat brace - size - 38)

Footrest

install - put on the floor almost decomposed braked stroller and release both safety levers. Take the footrest, press spring button placed at the rear top of the tube, unlock the locking pin and pull both tubes into the both footrest attachments until they are locked in the lowest position by the rear spring button. height adjustment - release both safety levers, pull both spring buttons of the footrest and adjust it into desired position. Tighten both safety levers to secure the footrest in position.

uninstall - put on the floor almost decomposed braked stroller. Place the footrest into its lowest position, press the rear spring buttons, unlock the locking pin and pull out the footrests both side tubes at the same time.

Brakes

lock - pull the brake lever down to lock the wheel

unlock - pull the brake lever up to unlock the wheel

Five point**Deploy 5 points. belt on the stroller**

Shoulder straps should be punched with a buttonhole in the backrest at the desired height according to the size of the child, we strap the straps through the inner space in the cover to the top buttonhole, in which we strap through the backrest and in the back ensure the clasp clasp. Straighten the side straps with a metal buckle in the side of the back of the backrest on the structure and secure it with a buckle clip.

Deploy 5 points. Belt for baby - sit baby in carriage, shoulder straps to child on shoulders, secure with center buckle. We adjust the length of the straps using ladder clasps and pull-out buckles in the back of the backrest.

Getting into the Buggy - ensure the parking brakes are on, to make access easier, the footrest can be taken off; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the buggy steady); place child carefully into the buggy, fasten safety belt and fit

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zabrzdzenie kočíka

the footrest. The child must always feel comfortable.

Getting out the Buggy - ensure the parking brakes are on; if possible, place the back of the buggy against a wall or immovable object (alternatively, another helper may hold the pushchair steady); unfasten safety belt and lift the child carefully.

Negotiating the Kerb or Single Step

down - take the rear wheels to the edge of the kerb, hold the handles firmly and tip the buggy slightly backwards. Balance on the rear wheels, lower the buggy down the kerb on its rear wheels. Never tip the buggy forward at a kerb.

up - take the rear wheels to the kerb. With a firm grip on the handles, tip the buggy backwards and place the rear wheels on top of the kerb. Position the front wheels at the kerb bottom and lift to the upper level.

Negotiating Steps

- make use of ramps and lifts when they are available

- where possible steps should be avoided, if there is no alternative to negotiating steps then ensure at least one helper is available to give assistance

- do not attempt to take the buggy up or down a flight of steps unless you and your helper are sure you have control of the buggy and can take the weight of its occupant

- during this operation, the attendant and helper must be well balanced and have secure grip on the buggy

- the attendants hands must be inside the handle straps at all time

- take rests in regular intervals

- do not lift the buggy with footrest

going down steps - position the buggy squarely with the front wheels on the top step, tilt the handle backward and push the buggy forward until the rear wheels are at the top step; the helper stands in front, steadies and lowers the buggy by gripping the front frame; take the weight and lower the rear wheels down the step, continue step by step to the bottom. (picture no. 1)

Going up steps - position the buggy squarely with the rear wheels against the bottom step, tilt the buggy backward and raise to the next step; the helper stands in front, steadies and lifts the buggy by gripping the front frame; continue step by step to the top. (picture no. 2)

Folding the buggy - unfasten the dry zipper of armrest upholstery, unlock rear brace by pushing up on center of brace. (Unlock the seat brace - size 38) Grasp one handle and pull up the seat tube with other hand. Make sure to cover the backrest is curved toward the stroller to prevent damage to the spacer folding mechanism. Lock the storage clip. Is not necessary to uninstall the footrest for folding the buggy. (Picture no.3)

Upholstery - the seat, seat reinforcement, backrest and backrest extension are not detachable. You could clean them by warm water and mild detergent.

armrests - undo the velcro and button to uninstall the armrests upholsteries.

Wash by hands on max. 40°C with mild detergent.

Seat reinforcement - you could increase or decrease the seat rigidity by adjustment of the seat reinforcement. Lengthen the belts of seat reinforcement to decrease the rigidity. Shorten the belts of seat reinforcement to increase the rigidity. Never change the seat width by adjusting of the seat rigidity.

MAINTENANCE

- your buggy has been designed to a high standard and with correct use and maintenance will give many years of trouble free service
- regularly inspect locking devices, catches, footrest, joints fixings to ensure they are secure and in full working order
- brakes, wheels, tires are subjects to wear and should be replaced if necessary by a repairer approved by the buggy supplier
- we recommend that the buggy is checked by an approved repairer every 12 month
- We recommend that after 24 months visit a service department.
- if the buggy is damaged or in need of repair, the approved repairer will be able to advise on what is required.
- Alternatively, PATRON Bohemia a.s. Customer department is available for product information
- only PATRON Bohemia a.s. replacement parts should be used on this buggy
- should parts of your buggy become stiff or difficult to operate (or not later than every 6 month), ensure that dirt is not the cause, apply a light application of a spray lubricant e. g. WD 40 (do not use oil or grease). Ensure that the lubricant penetrates between the components but do not over spray as this may encourage the accumulation of dirt. Wipe off any access spray. Regular lubrication of moving parts will extend the life of your buggy and take opening and folding easier.
- make sure that you are familiar with the operation of the buggy. It should always open and fold easily. If it does not, do not force the mechanism - stop and read the instructions.
- wheels, plastic and metal parts may be sponged clean with warm water and mild detergent.
- the seat fabric can be washed by hands (max. 30°C).
- never clean with abrasive, ammonia based, bleach based or spirit type cleaners.
- after exposure to damp conditions, dry off with a soft cloth and leave fully open in warm environment.
- store your buggy in a dry, safe place
- Check and tighten the screw connections regularly.
- If there are used control cables (brakes, telescopes, tilt), check it regularly and, if necessary, set up and oil them.

PATRON Bohemia a.s. reserves the right to make design changes to any of their products without prior notice as a part of their continuous development program.

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Alternativ steht Ihnen PATRON Deutschland GmbH für Produktinformationen zur Verfügung
Es sollten nur Ersatzteile der PATRON Deutschland GmbH verwendet werden
Änderungen auf allen Produkten der Firma Patron Deutschland GmbH vorbehalten