




2 Easy Steps to Independence

1 Sitting Down

- A. Sit Comfortably in the center.
- B. Start deflating by pressing the “Down” arrow button.
- C. **You Can Now Sit Down Comfortably.**

◆ NOTE

One “Down” arrow depression will lower one layer. Continuous pressing of the “Down” arrow will deflate air from the next layer automatically.

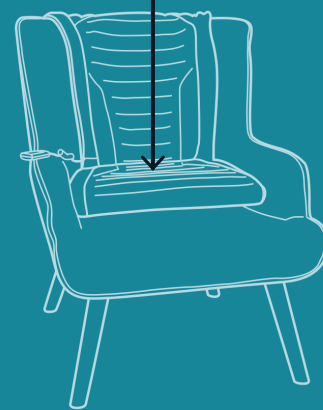

2 Standing Up


- A. Sit upright.
- B. Place your feet close to the chair.
- C. Spread your feet.
- D. Inflate the cushion by pressing the “Up” button.
- E. **Now you can easily Stand Up.**

◆ NOTE

One depression of the “Up” button will inflate one layer. Continuously pressing the “UP” button will inflate next layer automatically.



i Get to know the Remote control

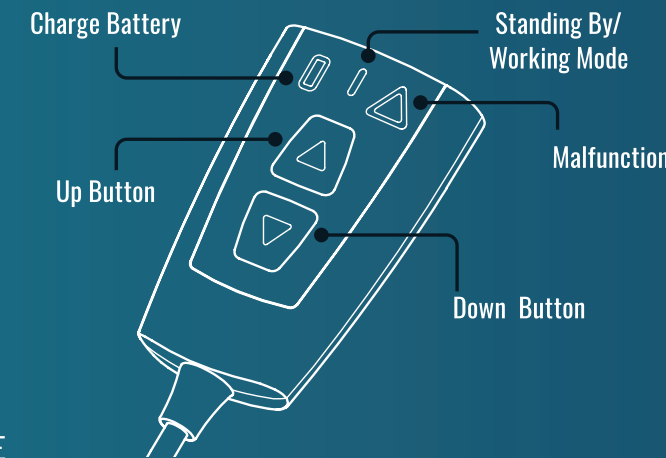

◆ NOTE

Standby mode - flashes slowly (one flash/sec) | Working mode - flashes quickly (5 flashes/sec) | Malfunction reset - press the two control buttons continuously for 10 secs. If that doesn't help, please contact us at – info@sitnstand.com


Placing the SitnStand

- A. Select an appropriate seat. It is advisable to select a steady and stable chair that does not easily move. An armchair is preferable.
- B. Position the **SitnStand** as illustrated below.



Charging the Battery

- A. Remove battery by sliding it towards you.
- B. Connect battery to the charger as illustrated.
- C. When the “Red” light on the charger changes to “Green”, the battery is fully charged.
- D. Slide the charged battery into its place.

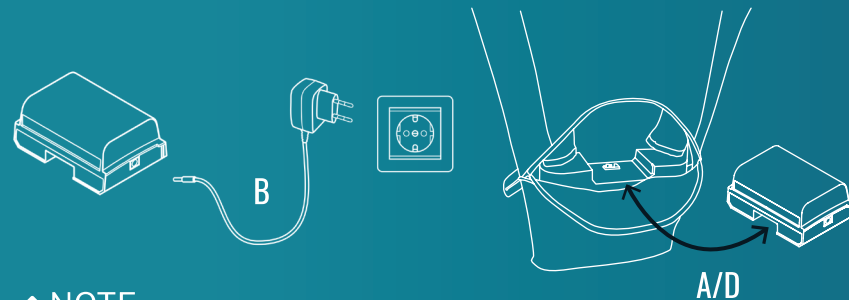

◆ **NOTE**
When using the battery for the first time, it should be charged for at least 12 hours.

Moving is Easy

- A. Hold both handles.
- B. Lift the **SitnStand** up from the seat.
- C. Place the **SitnStand** on the new seat.
- D. Sit down.




Keep This Card

Congratulations for buying the SitnStand!

Keep it folded in the battery pocket

We hope that the SitnStand will offer you greater independence for many years, and allow you to move freely wherever you go, without the concern of where to sit and how to stand up.

The SitnStand is a quality product. We have invested time and effort to ensure it is delivered to you in prime condition to serve you for many years. If for any reason you experience any discomfort during use, please do not hesitate to contact us.

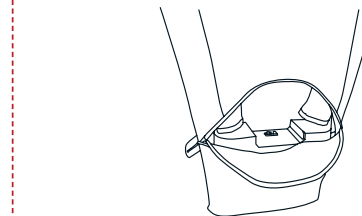

SitnStand
Portable Smart Rising Seats

Tel. +972-77 437 1228

email info@sitnstand.com

web www.sitnstand.com