

Mobile or Non-Mobile Wooden Sitter Base

ASSEMBLY MANUAL

Model # 77XXXXXX

UM-SWB-06-22LIT

Revision 06/2022

Helpful Tools



Large phillips screwdriver



Crescent wrench



Casters & Brackets



2x Angled Brackets

2x "L" Brackets

Bracket Hardware

8x Bracket Bolts
8x Bracket Lock Nuts




Non-Mobile Hardware Kit

4x Glides
4x Lock Washers
4x Nuts



WARNING: To avoid unintended motion, lock caster brakes when not in use.


01 Sitter Sizes 1,2, and 3

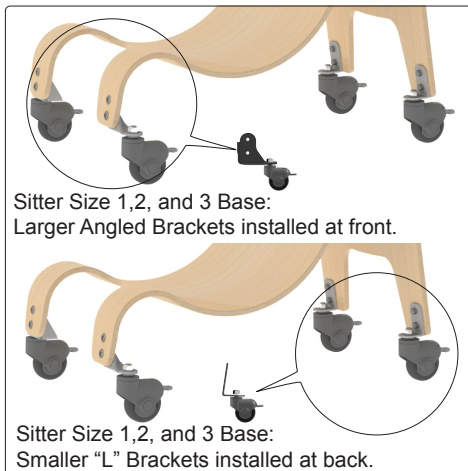

Sitter Size 1,2, and 3 Base:
Larger Angled Brackets installed at front.

Sitter Size 1,2, and 3 Base:
Smaller "L" Brackets installed at back.

02 Sitter Sizes 4 and 5

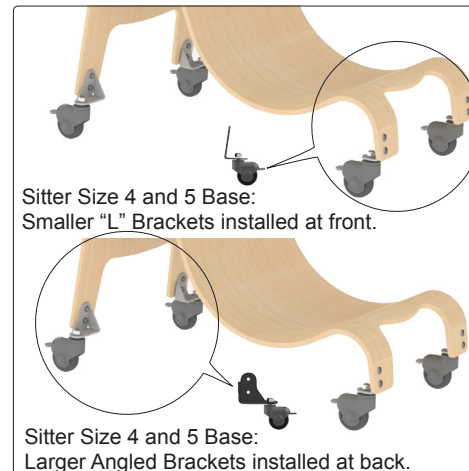

Sitter Size 4 and 5 Base:
Smaller "L" Brackets installed at front.


Sitter Size 4 and 5 Base:
Larger Angled Brackets installed at back.

Install Brackets x4



03 Install each Bracket to Wooden Base as shown in Steps 1 and 2 using supplied Bracket Bolts and Lock Nuts. Note: Maintain tight bolts.

Non-Mobile Only: Glides 4x

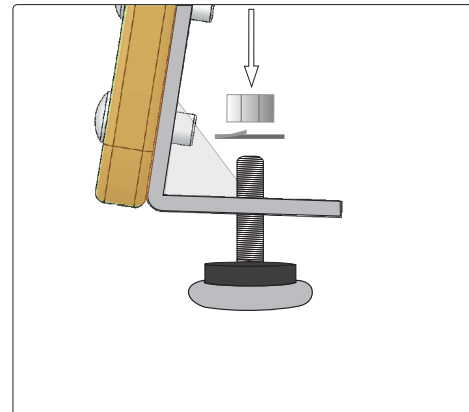

04 Feed Non-Mobile Glide up through Base Bracket. Install Lock Washer then tighten Nut. Note: Maintain tight bolts.



Global Carehab
HOLMSTRUPGAARDEJ 11
8220 BRABRAND
DENMARK
+45 24 65 55 50



© 2022 Bergeron Healthcare
15 South Second St.
Dolgeville, NY 13329
1-866-529-8407

