35218
SELECTION
ULNAR DEVIATION

SELECTION

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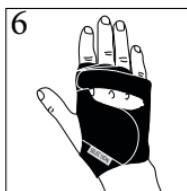
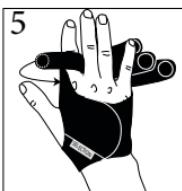
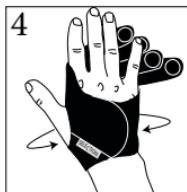
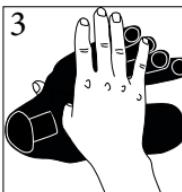
Carefully read this instruction sheet before use!

Indications / Intended Use

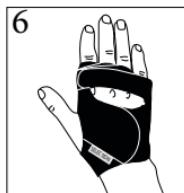
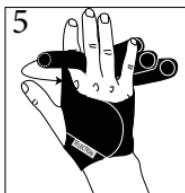
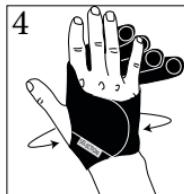
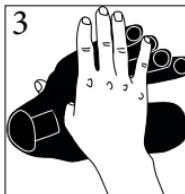
Textile hand/finger-orthosis for management of Ulnar Deviation. Corrects alignment of joints of the fingers to improve extension, strength and function of the hand and thereby can provide reduced pain and joint stiffness. For patients with Arthritis. Ulnar deviation is likely to be a characteristic for those with various forms of inflammatory diseases, such as Rheumatoid arthritis. Can also be used for realignment and stabilization in people with joint laxity and muscle weakness. Can be combined with a Selection Wrist orthosis.

Contraindications

Fixed contractures.

Adjustments

This product should only be altered by a medical professional person. Minor adjustments can be done during fitting. It's possible to cut in material without frying.

Warnings

Ensure that the straps are not too tight. Do not use when the product is wet or on wet skin. Check your skin daily for irritation. At first sight of skin irritation remove the orthosis and contact your treating clinician.

Material Specifications

This orthosis is made from nylon, polyester, lycra. This product is not flame retardant. Latex free.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the orthosis is complete and without damage.
2. Check that you have the correct size and side.
3. Place the thumb into the hole.
4. Fasten the splint at the back of the hand first.
5. Fasten each finger strap in between the fingers, one at the time, starting with the index finger.
6. Adjust until you are satisfied with the position of the fingers.

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