

# The ProForm NX™ Cushion




# ProForm NX™

- A cost-effective, customizable solution from an off-the-shelf product
- For clients with challenging positioning and postural support requirements
- Incorporates Air-Foam Floatation technology with modifiable components
- Components can be easily modified using common everyday tools
- HCPCS Code E2624/E2625 Skin Protection and Positioning Wheelchair Seat Cushion, adjustable




# Contoured Base

- Easily modifiable with standard, everyday tools
- Abductor Laterals
- Medial Thigh Separator
- Pre-ischial Bar
- Deep Ischial Pan

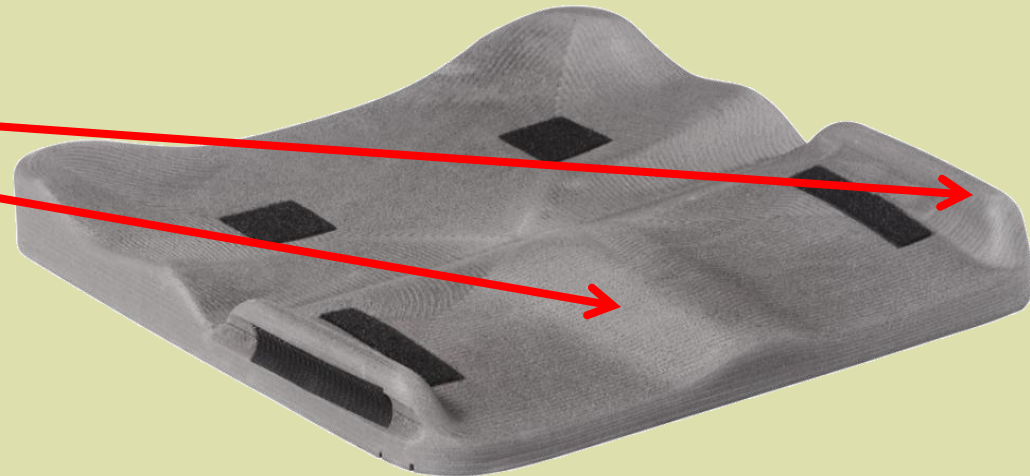

# Contoured Base

- Easily modifiable with standard, everyday tools
- Adductor Laterals
- Medial Thigh Separator
- Deep Ischial Pan
- Pre-ischial Bar

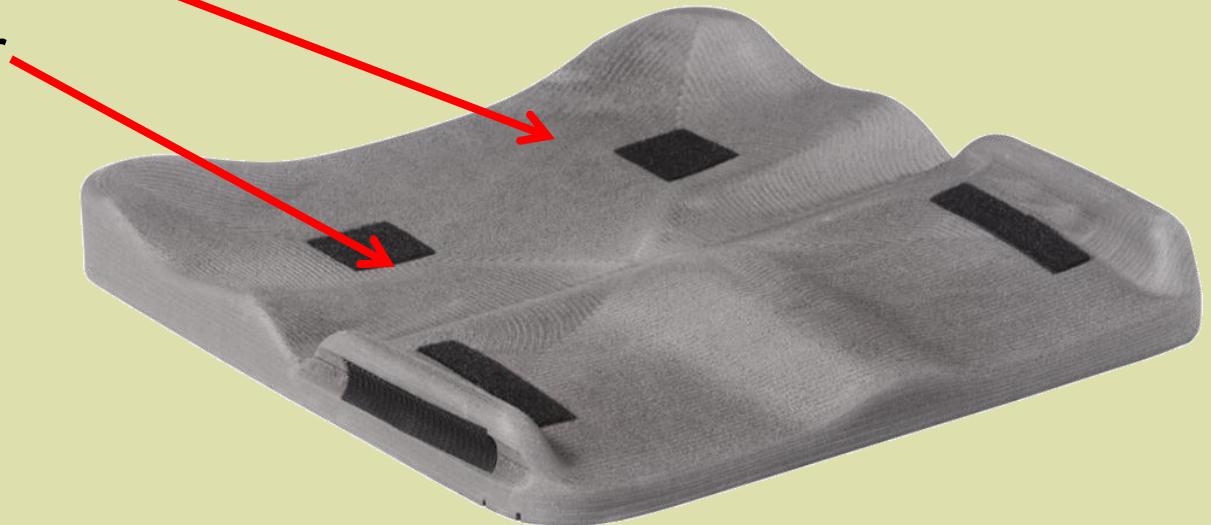


# Contoured Base

- Modification Guides
- Flex Hinge



# Modifiable Wedge

- Modifiable Wedge can be cut to any length for desired amount of “dump” or “squeeze”
- Because the Contoured Base is hinged, the seat-to-back angle does NOT change when Modifiable Wedge is utilized
  - The pelvis remains neutral while unloading the ITs and loading the thighs
- Can be placed inside the cushion cover

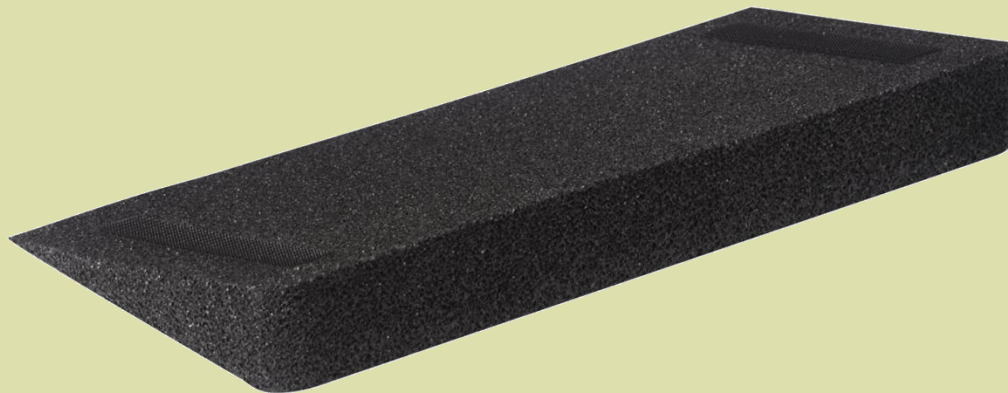

# Air-Foam Floatation Cushion

- Self-inflating, air and foam cushion
- Two-way valve releases air to immerse client into the cushion
- Dual-chambered cushion
  - Patent pending technology
  - Provides accommodation of pelvic asymmetries
- Single-chambered cushion
  - Provides symmetrical pelvic and postural support



# Modifiable Thigh Cushion

- Closed-cell foam provides postural support and comfort through the thigh region
- Contours over the medial thigh separator and adductor laterals
- Score lines on bottom for easy modification in 1" and 2" increments
- Hook and loop patches for secure attachment to the contoured base





# Cover

- Available in mesh and incontinence versions
- Machine washable
- Non-skid surface on bottom
- Reticulated foam inside promotes air exchange and improves pressure distribution
- Hook and loop aligned on the bottom allows for cinching of the cover when making modifications



# Modifying the ProForm NX™

- Modify Contoured Base and Modifiable Wedge using common blade or electric knife
- Cinch cover using provided hook and loop
- Alter Modifiable Thigh Cushion using common blade or electric knife



# ProForm NX™ cushion is the seating solution for:



Pelvic Obliquities

# ProForm NX™ cushion is the seating solution for:



Upper Leg Length Discrepancies

# ProForm NX™ cushion is the seating solution for:



Wind-Swept Deformities

**ProForm NX™ is the seating solution for:**



CVA/ Unilateral Scooters

# ProForm NX™ cushion is the seating solution for:



Above Knee Amputees (AKA)

# ProForm NX™ cushion is the seating solution for:



Symmetrical sitters requiring extra support



# ProForm NX™ cushion is the seating solution for:



Clients with limited hip flexion

# ProForm NX™ cushion is the seating solution for:



Clients with hot spots