
# Seating Accessories

VARILITE™ accessories let the seating specialist customize a client's seating system.

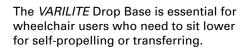





The VARILITE Rigidizer, or solid insert, counters the effects of sling upholstery. For some wheelchair users, sling seating promotes internal rotation of the LEs, sacral sitting, posterior pelvic tilt, loss of natural lumbar lordosis, and obliquity of the pelvis. Such poor positioning can result in discomfort and long-term problems. A Rigidizer offers an easy and cost-effective solution to the problems of sling seating.

VARILITE Rigidizer

Every wheelchair user is an individual. VARILITE Cheat Sheets™ let the seating specialist modify a seating system's overall fit, feel and function to meet each client's specific needs.



### **Drop Base**

	14 x 16	16 x 16	16 x 18	18 x 16	18 x 18	20 x 20
	(35 x 40 cm)	$(40 \times 40 \text{ cm})$	(40 x 45 cm)	(45 x 40 cm)	(45 x 45 cm)	(50 x 50 cm)
	04388	04288	04308	04278	04378	04258
Fits Chair Width:	13 - 14 in.	15 - 16 in.	15 - 16 in.	17 - 18 in.	17 - 18 in.	19 - 20 in.
	(33 - 35 cm)	(38 - 40 cm)	(38 - 40 cm)	(43 - 45 cm)	(43 - 45 cm)	(48 - 50 cm)

#### Rigidizer

12 x 12	12 x 14	14 x 14	14 x 16	16 x 16	16 x 18	16 x 20
(30 x 30 cm)	(30 x 35 cm)	(35 x 35 cm)	(35 x 40 cm)	(40 x 40 cm)	(40 x 45 cm)	$(40 \times 50 cm)$
04360	04390	04260	04380	04280	04300	04290
17 x 17	18 x 16	18 x 18	18 x 20	20 x 16	20 x 18	20 x 20
(43 x 43 cm)	(45 x 40 cm)	(45 x 45 cm)	(45 x 50 cm)	(50 x 40 cm)	(50 x 45 cm)	$(50 \times 50 \text{ cm})$
04389	04270	04370	04240	04320	04340	04250

## Cheat Sheets™

10 X 10
(25 x 25 cm)
02102

# Seating Accessories



VARILITE Drop Base is made from durable and lightweight 3/8-inch (1 cm) PVC. Our L-J brackets are hardened steel for superior strength and durability. The hardware allows 4 inches (10 cm) of height adjustment and 15 degrees of angle adjustment-great for changing the seat-to-floor height or seat-to-back angle.

Finger locks secure the Drop Base to the wheelchair and allow for quick removal. VARILITE Drop Base fits wheelchairs that are 13 to 20 inches (33 to 50 cm) wide, with seat rail diameters of 7/8 or 1 inch (2.2 or 2.5 cm).



VARILITE Rigidizer is made from durable 1/4-inch (6 mm) plywood and is available in pediatric to adult sizes. It can be placed between the cushion and upholstery or, preferably, inside the cushion cover (we round the edges and sand the surface so there is no risk of splintering or puncturing). With the Rigidizer zipped inside the cover, the user benefits from improved sitting balance, more neutral pelvic position, greater spinal extension, and overall improved comfort. All VARILITE cushion covers are sewn with room for a Rigidizer.

VARILITE Cheat Sheets are adhesive-backed sheets of closed-cell foam. Each package contains four pieces of foam measuring 10 x 10 inches (25 x 25 cm). Cheat Sheets can be cut with a blade or scissors and applied to a variety of surfaces.

#### Suggested Cheat Sheet Applications:

Simply cut the shaped desired, remove the adhesive backing and apply the Cheat Sheet to a clean, dry surface. Layer as needed and round off corners and edges. Apply a final, smooth layer.

A popular use for Cheat Sheets is customizing a *VARILITE* Pro-Form™ contoured base. Cheat Sheets are used to build up the medial thigh separator and laterals, smooth cut edges, and improve contact for overall pressure distribution.

#### Other Cheat Sheet Uses:

- · Build contours on planar systems
- Modify back systems for kyphotic posture
- Pad or contour headrests, laterals, hip guides, ASIS bars, armrests
- · Smooth sharp edges on wheelchairs or accessories





CE